
EN, This transit wheelchair Neptuno model of Mobiclinic is designed even for the occasional use of one user with 100 kg maximum. It can be used indoor or outdoor and can be manually moved forwards and backwards. This transit wheelchair Neptuno is crafted of the finest quality and it meets the EC standards.
Folding and unfolding the wheelchair:

Weight capacity: 100kg.

To unfold the chair, follow the steps below:

1. Remove the chair from the bag and raise the backrest.
2. Grasp the aluminum bar between the rear wheels and pull the seat belt until you hear a click to secure the ends.
3. Snap the safety levers on both sides.
4. Lower the armrests on both sides.
5. Fold out the aluminum footrest and the chair will be completely unfolded.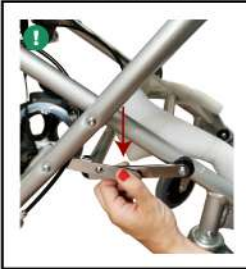

! To fold the chair, do the same thing in reverse, always making sure that the side folds are down.

MAIN ASPECTS TO TAKE INTO CONSIDERATION

The user or caregiver must check the security of the wheelchair often.

1. Please, check the hand grips and ensure that they are properly fixed and secure.
2. To ensure a correct adjustment, please, check the blockers and make sure that the big wheels are blocked when the brakes are activated.
3. Check all the screws, nuts, and joins to prove that they are correctly fixed.
4. In case of noticing other conditions that can affect the security of the user, please, stop using this wheelchair.
5. Do not stand on the footrests when getting out of the chair.
6. Keep this wheelchair away from fire.
7. Make sure that the brakes are activated before sitting down or standing up from the wheelchair.
8. Avoid bending forwards in the wheelchair without having both feet on the floor. This is especially important for those users with plasters.

How to activate and deactivate the braking system of the Neptuno wheelchair:

Image 1. To temporarily brake the chair, press the brake lightly. To advance, release the brake.

Image 2. To activate the locking system, press the brake down. To unlock it, lift the brakes upwards and you can continue to drive.

CLEANING AND MAINTENANCE

To keep your product clean, you can wipe down the surfaces with a damp cloth, never wet! After that and as a part of your product cleaning, be sure to dry everything thoroughly. Do not use solvents, bleaches, synthetic detergents, wax glazes, sprays or other abrasive products which may damage it.

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY CO., LTD
SUNGO EUROPE B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.