ENGLISH

CAUTIONS

Read carefully these instructions before the patient uses this raised toilet seat Alcalá and keep them for future reference.

The raised toilet seat Alcalá is designed for the sitting on the toilet seat and should never be used for any other use different to the intended one. Do not use this product as a staircase, do not stand up on it and do not allow children to play with it.

You should check regularly and before its use that the fixing elements are firmly secured. Check carefully the state of the components of the equipment. In the case that one of them appears not to be in good condition of use you should substitute it by an original spare part.

Height: 10 cm.

Maximum patient weight: 150 kg.

ASSEMBLY AND INSTALLATION

1. Take all the components off the packaging. Verify that all the accessories are included.
2. Take one armrest and place it in its lateral orifice on the posterior part of the raised toilet seat, insert and coil the screw between the armrest and the raised toilet seat. Repeat the same procedure with the other armrest.
3. Adjust the 3 limits of the raised toilet seat to your toilet seat, tighten the screws until it stays stable.

CARE INSTRUCTIONS AND MAINTENANCE

Wash and dry the seat after each use. Remove the dirtiness with a wet cloth and neutral soap and then dry. Do not use aggressive or abrasive chemical cleaners as they can damage the seat.

At the end of its lifetime, get rid of the seat in a responsive way.

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**BESTCARE INTERNATIONAL LTD**
LOTUS NL B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.