## ESPAÑOL

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## ENGLISH

### Parts listing and images

1. Bolts
2. Washers
3. Key
4. Backrest
5. Backrest supports
6. Seat
7. Seat support

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### How to make the assembly

1. Open the packing and take out all parts from the polybag.
2. Put the washer in the screws.
3. Place the seat on a chair with upside down and place the seat supports on the slots. Put the short bolt into the hole in the center where two leg pipes cross and lock the bolt with the tool.
4. If your model is Olivo or Marisma, you must assembly the backrest. There are two holes on each end of the backrest supports. Check the end with bigger distance between two holes and place this end on to the seat supports and lock it with bolts.
5. Put the shower chair on the floor and put the backrest and backrest tubes together with the bolts.

**Remark:** Lock all bolts with the key available from the tool kit. Weight capacity: 100kg.

Care and maintenance: clean the product only with soap and water, do not use any chemical substance. Protect from unfavourable weather.

Intended for domestic use, please do not use in wet environment or permanently outdoor if you want to extend their service life.

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### Indications

- It is designed for the old, the patient, the weak, and disabled, as a shower chair.
- To be used indoors on a flat, smooth surface.
- Check regularly that all nuts and bolts are tightened securely. And make sure all four legs are touching the floor at the same level.
- Weight capacity 100kgs.

### Tips and precautions

- Unless it is specified, do not adjust, assemble or disassemble the shower chair with a user seated. Only adjust, assemble or disassemble the chair when the chair is not occupied.
- Do not tilt your body out of the chair, to avoid the chair turning over.
- Do not change your sitting position when you head back, otherwise it might lead to chair turning over.
- Push the seat on both side parallel, to stand up. Do not push only one side.
- After using the shower chair, clean it with dry cloth. Avoid direct contact with corrosive chemical.

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Imported by/Importado por: Grupo R. Queraltó S.A.