ESPAÑOL

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ENGLISH

This device consists of two parts: the mattress and the pump. The pump uses a small, quiet and energy efficient compressor. The control panel is simple and very user friendly. The mattress relieves pressure thanks to its alternating air cells that inflate and deflate every 12 minutes, approximately. It is known that constant pressure on a bone protuberance is the main cause of bedsores. The continuous movement of the mattress reduces the constant pressure and improves blood flow.

Notice to user and/or patient:

Any serious incident that has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Use:

1. Place the mattress with the hose end towards the foot of the bed.
2. Use the included hooks to securely hang the pump on the footboard.
3. Connect the air hoses from the mattress to the pump.
4. Connect the pump to an electrical outlet and make sure the power cord is safe from possible hazards.
5. Turn on the power with the switch on the control panel of the pump. The pump will begin to inflate the mattress.
6. Once it inflates, adjust the mattress using the dial on the compressor.

Indications:

This product is intended:

- To help and reduce the incidence of pressure ulcers while optimizing patient comfort.
- For long term home care of patients suffering from pressure ulcers.
- For pain management as prescribed by a physician.
- If needed, replace the fuse with one marked as: T1A 250V.

FRANÇAIS

Power input: ~220V 50Hz, 0,1A.

- Maximum weight: 140 kg.

Contraindications:

- This equipment is not suitable for use in the presence of flammable anesthetic mixture with air or with pure oxygen or nitrous oxide.
- Do not turn or flip patients who have bone fracture or spinal cord injury.
- Do not smoke near the pump.
- Keep the pump away from hot surfaces.
- There is a risk of explosion if used near flammable anaesthetics.

Cleaning and maintenance

1. Make sure the system is disconnected from the electrical supply before cleaning.
2. Do not submerge in water.
3. Do not dry clean.
4. Do not use bleach or products that contain it.
5. Do not iron.
6. Do not sterilize in an autoclave or use phenol for the cleaning process.
7. Do not clean the air cells, clean with mild soapy water or disinfectant.
8. Clean the compressor with a dry and soft cloth.
9. Environmental conditions:
 - Operating conditions:
Room temperature: From 10°C to 40°C
Relative humidity: From 10% to 75%
 - Storage and shipping conditions:
Room temperature: From -18°C to 43°C
Relative humidity: From 10% to 95%

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Imported by/Importado por: Grupo R. Queralto S.A.
Guangdong Yuehua Medical Instrument Factory Co., Ltd.
SUNGO Europe B.V.