


**1 ES Piezas / EN Parts listing / FR Pièces / PT Partes / IT Pezzi /
 DE Teile / NL Onderdelen / SWE Delar / PL Części / DK Dele**

**2 ES Montaje / EN Assembly / FR Assemblage / PT Assembleia / IT Montaggio /
 DE Montage / NL Montage / SWE Montering / PL Montaż / DK Samling**


A - Asiento / Seat / Siège / Sede /
 Sedile / Sitz / Seat / Sittplats /
 Siedzenie / Sæde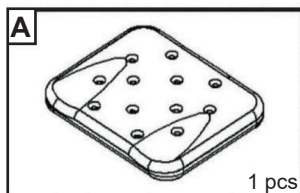


B - Respaldo / Backrest / Dossier / Apoio /
 Schienale / Lehne / Steun / Stöd /
 Wsparcie / Støtte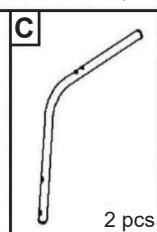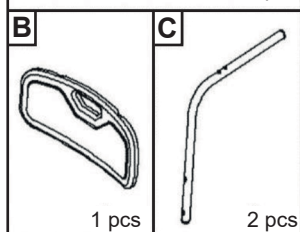



C - Barras para el respaldo / Backrest supports /
 Barras pour le dossier / Barras de encosto /
 Barre per lo schienale / Stützstangen /
 Rugleuningstangen / Ryggstödsstänger /
 Pręty oparcia / Stænger til ryglæn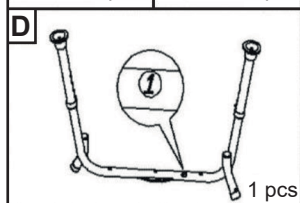



D+E - Barras para el asiento / Seat supports /
 Barres pour le siege / Barras de assento /
 Barre per il sedile / Sitzstangen /
 Zitstangen / Stånger för sätesstolpar /
 Drażki do siedzenia / Sædestænger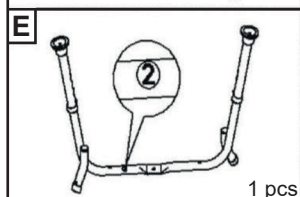






H+I - Reposabrazos / Armrests / Accoudirs /
 Apoios de braços / Braccioli / Armlehne /
 Armsteunen / Armstöd / Podłokietniki / Armlæn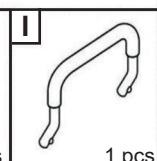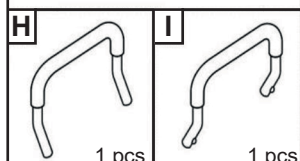

J - Tornillos con arandelas / Bolts with washers /
 Vis avec rondelles / Parafusos com arruelas /
 Viti con rondelle / Schrauben mit Unterscheiben /
 Schroeven met sluitringen / Skruvar med brickor /
 Śruby z podkładkami / Skruer med skiver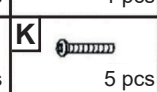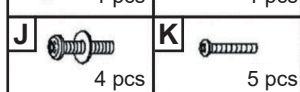

K - Tornillos / Bolts / Vis / Parafusos /
 Viti / Schrauben / Schroeven / Skruvar /
 Śruby / Skruer

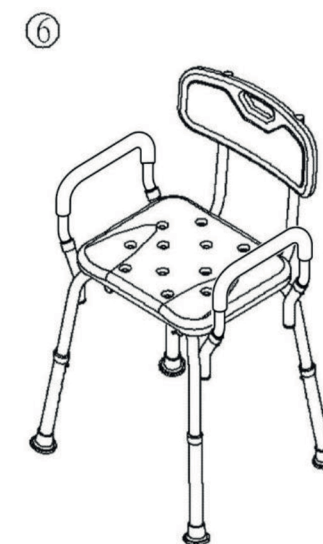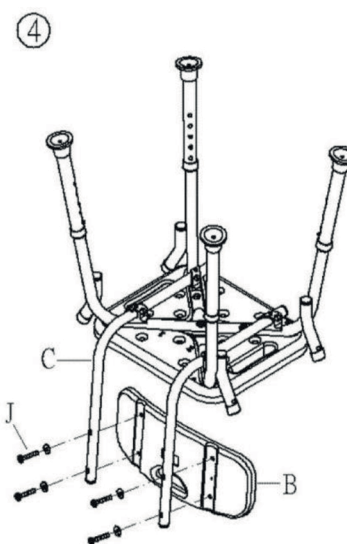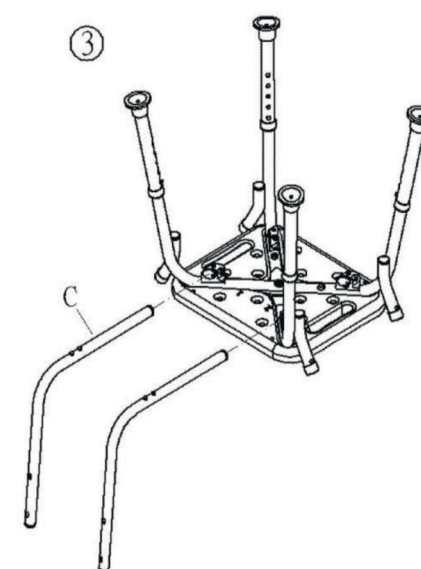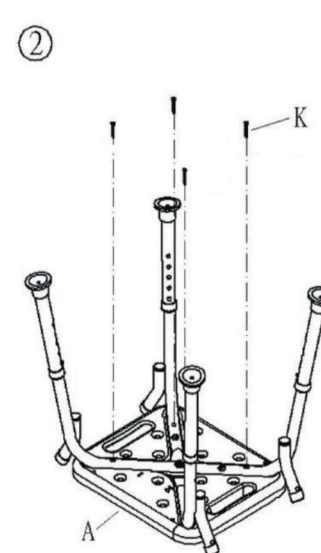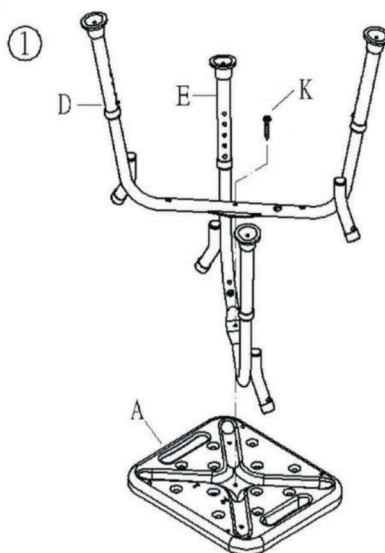ESPAÑOL

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ENGLISH

Instructions for use:

To adjust the height of the chair, press the adjustment plunger on the leg and slide it to the desired height. Also make sure that the adjustment plunger is positioned securely in one of the holes. This process should be repeated for all four legs and all legs should be at the same height for your safety.

Warning to the user and/or patient:

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications:

- It is designed for the assistance of patients, elderly and/or disabled persons.
- It should be used indoors on a flat, smooth surface.
- Check regularly that all nuts and bolts are tightened securely. And make sure that all four legs are touching the floor at the same level.
- Maximum weight: 100 kg

Contraindications:

- Only adjust, assemble or disassemble the wheelchair when the wheelchair is not occupied.
- Do not lean your body out of the wheelchair, to prevent the wheelchair from tipping over.
- Push the seat from both sides to stand up. Do not push on one side only.
- Do not stand on the wheelchair.

Cleaning and maintenance:

- Remove dirt with a damp cloth and neutral soap and then wipe dry. Do not use harsh chemicals or abrasive cleaners to avoid damaging the product.
- After use, wipe the shower stool with a dry cloth. - Avoid direct contact with corrosive chemicals.

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