MANUAL DE INSTRUCCIONES / INSTRUCTION MANUAL / MANUALE D'UTILISATION

Bastón trípode / Tripod cane / Canne tripode GLORIA

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ENGLISH

Description

- Cane with folding seat, practical and easy to use. Includes a curved support handle, covered with foam, a round PP padded seat and a tripod design for extra stability. Supports a weight of up to 90 kgs.
- Seat diameter; 21 cm.
 - Total height of folded product: 84 cm.
 - Height of open seat (distance from the ground to the seat): 50 cm.

Warnings and precautions

- Do not exceed the maximum load capacity of the product.
- Use with caution. To avoid tipping hazard, do not grasp handles with wet hands, as the may slip and lose their grip.
- Use extreme caution when using on wet and slippery surfaces.
- The canes are intended only as a walking aid and are not designed to support full body weight.

Instructions for use

- The cane is already assembled and ready for use. Still, check the condition of the components before use.
1. Never deploy nad use your seat cane on a slippery, soft or uneven surface, also not in bathrooms or moving vehicles.
 2. Before using the cane, check that all parts are properly connected and secured, if there is any sign of cracking or breakage of the cane, it should not be used.
 3. Always make sure that all the screws and bolts are securely fastened before use and check the stability of the pole.
 4. Make sure the folding legs are fully extended and on a secure surface. All three legs should touch the ground simultaneously.
 5. Straddle the seat facing the handle with your hands on the handle to maintain balance while the product is unfolded, Sit in the centre of the seat in a stable position. Do not rock or drag the chair. Do not use only two legs or a single chair leg for support.
 6. Do not use the handle for support when standing or sitting.
 7. Do not hang heavy objects on the handle.
 8. Do not sit sideways, recline with your back to the handle or stand on the chair.

Notice

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications

Walking sticks are used as a support device: for patients with musculoskeletal disorders or elderly people;to facilitate the movement of disabled and sick patients with injuries and diseases of the lower extremities using orthoses and prostheses; for the elderly in order to relieve the joints; for patients during the rehabilitation period after surgery; for rehabilitation after limb fractures

Contraindications

The very old and very young may not possess the needed torso strength and coordination to use crutches successfully.Any musculoskeletal or neurological condition that reduces the mobility, strength, and sensation can potentially be a contraindication to the short term or long-term use of crutches. Inability to coordinate body movements, dizziness, drowsiness.

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JIANLIAN HOMECARE PRODUCTS CO., LTD.
Lotus NL B.V.
Imported by/Importado por: Grupo R. Queralto S.A.