ESPAÑOL

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ENGLISH

Please read carefully this direction to know your new rollator well before use. Hércules model has a weight capacity of 100kg. The followings are steps of using rollators:

1. Lock the brakes on the rollator by pushing the brake handles down.
2. Place both feet side-by-side in the middle of the rollator frame.
3. Release the brakes by pulling the brake handles up.
4. Grasp the hand grips on either side of the rollator.
5. Push the rollator forward to a comfortable distance. Your toes should be even with the back wheels of the rollator.
6. Step forward toward the inside of the rollator frame with your weaker leg.
7. Step forward with your stronger leg while supporting yourself with the hand grips. Both feet should be side-by-side between the wheels of the rollator. Continue walking by repeating Step 5 and Step 7.

Please kindly insert the horizontal frame in the right direction, and you need to insert the both side at the same time. Do not insert one side, and then the other one.

Tips and Warnings

Keep a part of your body, like your toes, within the rollator frame at all times. When standing from a sitting position, push up from the seat instead of pulling up on the rollator handles. When turning, take small steps and keep your hips parallel to the front bar of the rollator. Avoid twisting your body away from the rollator. Don't sit on the rollator seat and don't use this device as a temporary wheelchair. The rollator can tip over causing injury. Ensure all wheels of the rollator are on level ground before walking. Don't hunch forward over the rollator. Don't use the rollator on an escalator or stairs.

Notice

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications

An aid for the movement of people with disabilities.

Intended for rehabilitation after various types of injuries.

Help facilitate the movement of the elderly.

Disorders of the musculoskeletal system.

Active expansion of the motor regime after long-term illnesses, injuries and the postoperative period.

Contraindications

The product is not designed for use by people with severe balance problems or functional perceptual disturbances.

A walker may be contraindicated for patients with limited upper extremity function, motor control, cognitive function, or balance deficit but this determination should be made by the issuing practitioner who can weigh the risks and benefits for the individual patient.

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JIANLIAN HOMECARE PRODUCTS CO., LTD.
Lotus NL B.V.
Imported by/Importado por: Grupo R. Queralto S.A.