ESPAÑOL

Descripción

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ENGLISH

Description

Bathtub board mounting design, easily removable. Includes a rotating round seat panel made of high strength materials, a seath panel with holes to drain water from the surface and reduce the risk of slipping, a handle for added safety and a soap holder.

Assembly

1. Tighten the screws firmly into the threads the handle should be facing the wall.
2. Place the bench on the bathtub.
3. Make sure that the inside walls of the bath are flat so that the bench can be securely fastened.
4. Adjust the position of the fastening screws to the inner width of the bathtub.
5. Tighten all screws securely.

Instructions for use

- The bathtub board supports a maximum weight of 100 kg. Do not overload the bench.
- The bath board pad helps to make it lightweight, easy to use and safe to access. Suitable for shower or bathing.
- Use the bench with care. Sit on it with the help of the handle and lift and insert your feet one at a time, never at the same time. Sit comfortably in the center of the bench.
- There are openings in the bench to ensure the water flow during use, thus reducing the possibility of slipping while sitting on it.
- For safety reasons, never sit on the edge of the bench that protrudes beyond the tub.

Notice

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications

The bath chair is an auxiliary product intended for people with musculo-skeletal disorders and people with disabilities. It facilitates the daily activities related to the maintenance of personal hygiene for all people for whom the standard use of a shower or bathtub is a problem.

Contraindications

Use of the bath chair is not appropriate in the case of perception disorders, extreme imbalance, inability to sit or a body weight above 100kg.

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Imported by/Importado por: Grupo R. Queralto S.A.
JIANLIAN HOMECARE PRODUCTS CO., LTD.
Lotus NL B.V.