MANUAL DE INSTRUCCIONES / INSTRUCTION MANUAL / MANUALE D'UTILISATION

Andador / Rollator / Déambulateur TRAJANO

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ENGLISH

Instructions of use

1. Lock the brakes on the rollator by pushing the brake handles down.
2. Place both feet side-by-side in the middle of the rollator frame.
3. Release the brakes by pulling the brake handles up.
4. Grasp the hand grips on either side of the rollator.
5. Push the rollator forward to a comfortable distance. Your toes should be even with the back wheels of the rollator.
6. Step forward toward the inside of the rollator frame with your weaker leg.
7. Step forward with your stronger leg while supporting yourself with the hand grips. Both feet should be side-by-side between the wheels of the rollator.

Tips and Warnings

When standing from a sitting position, push up from the seat instead of pulling up on the rollator handles. Apply the brake before sitting and make sure the brake is applied before standing. When turning, take small steps and keep your hips parallel to the front bar of the rollator. Avoid twisting your body away from rollator. Don't use the rollator on an escalator or stairs. Don't place heavy objects in the basket.

Assembly

1. Open the box and remove the rollator from the protective plastic.
2. Fold the back legs and insert a wing nut in each one to secure them. You will find these in a small bag.
3. Use the other two nuts for the holes in the front wheels.
4. Unfold the rollator and adjust the handlebars to the desired height.
5. Place the bag.

Notice

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications

An aid for the movement of people with disabilities. Intended for rehabilitation after various types of injuries. Help facilitate the movement of the elderly. Disorders of the musculoskeletal system. Active expansion of the motor regime after long-term illnesses, injuries and the postoperative period.

Contraindications

The product is not designed for use by people with severe balance problems or functional perceptual disturbances. A walker may be contraindicated for patients with limited upper extremity function, motor control, cognitive function, or balance deficit but this determination should be made by the issuing practitioner who can weigh the risks and benefits for the individual patient.

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Imported by/Importado por: Grupo R. Queraltó S.A.
JIANLIAN HOMECARE PRODUCTS CO., LTD.
Lotus NL B.V.