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# INSTRUCTION MANUAL ELECTRIC SCOOTER PISCIS

**NOTICE**

Notice to the user and/or patient: Any serious incident that has occurred in connection to the product must be communicated to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**INDICATIONS**

- Only one user can ride the scooter.
- The lowest speed must be used when carrying out the following actions: Going up, going down or crossing a slope or sloping terrain and also on uneven ground, ramps and soft surfaces.
- Keep away from flammable objects while charging the device.

**CONTRAINDICATIONS**



- Never try to go up or down a slope when the surface is slippery or there is a risk of skidding.
- Do not ride on a slope over 6 degrees.
- Avoid shifting the center of gravity when turning left or right.
- Direct exposure to rain or humidity will stop the scooter from functioning properly, it may cause the scooter to oxidise.
- Never put the scooter in neutral when you are on a slope.
- Do not try to modify or manipulate the scooter in regards to its mechanism or structure.
- Do not ride the scooter on stairs or escalators.
- Do not use the device while it is charging.

**OPERATING YOUR SCOOTER**

- Before operating your scooter, check the following:
  - Remove rear shroud and connect the battery wire.
  - Free-wheeling device on D
  - Speed control knob All 5 LED for is at the turtle picture.
- Sit on scooter and turn on key. The Power Reserve Indicator light should be lit. The self-diagnostic warning light should not be blinking.
- When your hands rest comfortably on the handlebars, the thumb control levers should be within easy reach. The right lever moves the scooter forward, the left one moves it backward. When you release both levers the scooter will stop.
- Practice driving where there are no obstacles. Start at the slowest.
- If the scooter stops, locate the circuit breaker (A). Press it and try the scooter again.

**IDENTIFICATION OF PARTS**

Before attempting to drive this scooter on your own, it is important that you familiarize yourself with the controls, and how to operate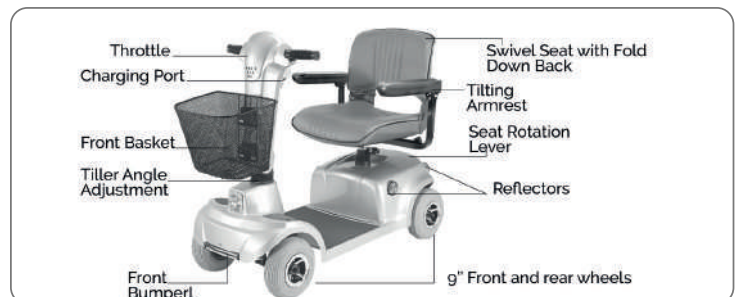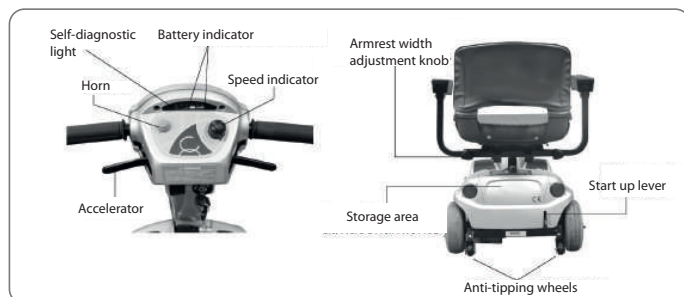

## FUNCTION OF PARTS

### Control Panel:

#### 1. Speed Control Knob

The image of rabbit means fast or high speed.

The image of turtle means slow or low speed.

Before driving the scooter, set the speed to low speed by turning the knob towards turtle image. Drive in high speed when encounter an up slope, and drive in slow speed when encounter a down slope.

#### 2. Self-Diagnostic Warning Light

Flashing light indicates there is a problem within scooter.

#### 3. Power Reserve Indicator

The 5 LED are red. When all 5 LED are lit, the batteries are fully charged. When the number of lit LED decreases, the batteries need to be recharged.

#### 4. Throttle

Pushing the right Throttle moves the scooter forward. Pushing the left Throttle moves the scooter backward. Releasing both engages the automatic brake. This is also your accelerator. The further you depress them, the faster you go.

Please keep safe distance when stopping to prevent any danger.

### At base of Steering Tiller and Seat:

#### 1. Tiller Angle Adjustment:

Push downward on the lever to adjust.

Pull up to lock Tiller Angle Adjustment at comfortable angle.

#### Seat Rotation Lever

Push lever back to rotate seat.

Push forward to lock seat in position.

#### Armrest Width Adjustment Thumbscrews

Loosen the two Thumbscrews to adjust the arm width.

Tighten again to lock the desired position.

### Rear Body:

#### 1. Rear Cover

Protects batteries from damage.

#### 2. Anti-tip wheels

Helps keep the scooter from tipping over.

#### 3. Free-Wheeling Lever

When lever is in the N (Neutral) position, the scooter can be moved without power. When lever is in the D (Drive) position, the scooter can be driven. Normal position is D. You will find some stickers next to the lever, they will help you to know the Mode of your scooter.

If you would like to change from Manual Mode to Automatic Mode, you must turn off the scooter. After changing the Mode, you will be able to turn on again your scooter.

### Keep in mind these rules:

-Release Throttle and allow scooter to stop completely before changing from forward to reverse or reverse to forward.

-Use the scooter only where it would be safe to walk.

-Use low speeds for reverse, downhill, ramps, curbs, or uneven surfaces.

Hill climbing: You may need to use a higher speed. Turn to lower speed before going downhill.

Down slopes: To proceed down steep slopes slowly, set speed control in proximity of the turtle. This enables driver control, as the closer the speed control is set toward the turtle, the slower it will become.

### ELECTROMAGNETIC INTERFERENCE AND WARNINGS

Powered wheelchairs and motorized scooters may be susceptible to electro- magnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such a radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones. The interference (from radio wave sources) can cause the motorized scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the motorized scooter control system.

1. Do not operate hand-held transceivers (transmitters-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the motorized scooter is turned ON.
2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
3. If unintended movement or brake release occurs, turn the motorized scooter OFF as soon as it is safe.
4. Be aware that adding accessories or components, or modifying the motori- zed scooter, may make it more susceptible to EMI.

20 volts per meter (V/m) is a generally achievable and useful immunity level against EMI (as of May 1994). The higher the level, the greater the protection. This product has an immunity level of at least 10 V/m.


### CHARGING THE BATTERIES

**Batteries must be charged before using the scooter for the first time, it should be charged at night and recharged after each day use.**

1. Insert battery charger cord into Charging Socket on battery pack.
2. Plug other end of power cable into a standard electrical wall outlet.
3. The charging indicator will normally be red or yellow at this point.
4. Allow batteries to charge until charging indicator turns green.

After indicator turns green, unplug battery charger from scooter and wall outlet. If at any time battery charger light flashes green over 40 minutes, this indicates abnormal charging occurred, so you should check if charger plug is correctly positioned or if the scooter is turned off.

If the problem continues, reset circuit breaker in (A).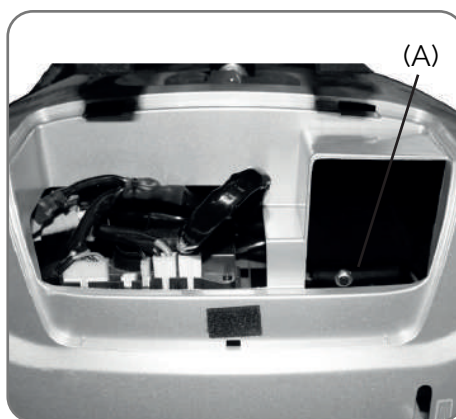

The meter gives an approximation of battery strength. Three coloured lights illuminated indicate batteries are fully charged. When only the red light remains on, the batteries need to be charged immediately. Charge the batteries after each trip.

If the scooter is not used for some time, batteries may lose their charge. Batteries should be charged at least once a month.

Batteries must be fully charged in order to not hurt their lifetime. If not, faults may result.


## DISASSEMBLING YOUR SCOOTER

The scooter is easy to disassembly and you can transport it by car. Tools are not necessary. To assemble the scooter, you can repeat the disassembly directions in reverse.

1. Remove seat by releasing Seat Rotation Lever and then lift Off.
2. Remove Rear Shroud on the Scooter
3. Remove Battery Fixing Velcro
4. Detach Battery Power Plugs.
5. Remove two Batteries.
6. By removing the Front Basket and Adjusting the tiller down by using the Tiller Angle Adjustment.
7. There are 5 main parts after disassembling the scooter


### Fuse Replacement

1. Remove the front cover screws (R), the arrow indicates the direction. (Caution, the screws have different sizes).
2. Then open the box where the fuses are located (R1) and replace it with a backup fuse (R2), the fuse must be of the same rating.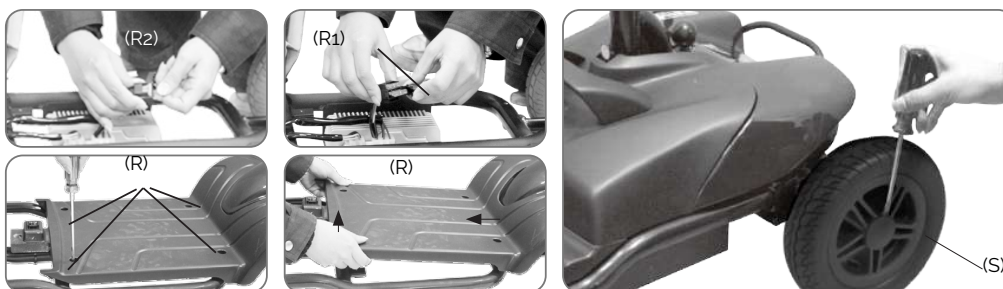

## CLEANING AND MAINTAINING YOUR SCOOTER

If your scooter is dirty, use a damp or lightly soapy cloth to wipe it down. Do not use running water to wash or rinse the scooter in order to protect the electrical parts. Polish with an automotive liquid polish.

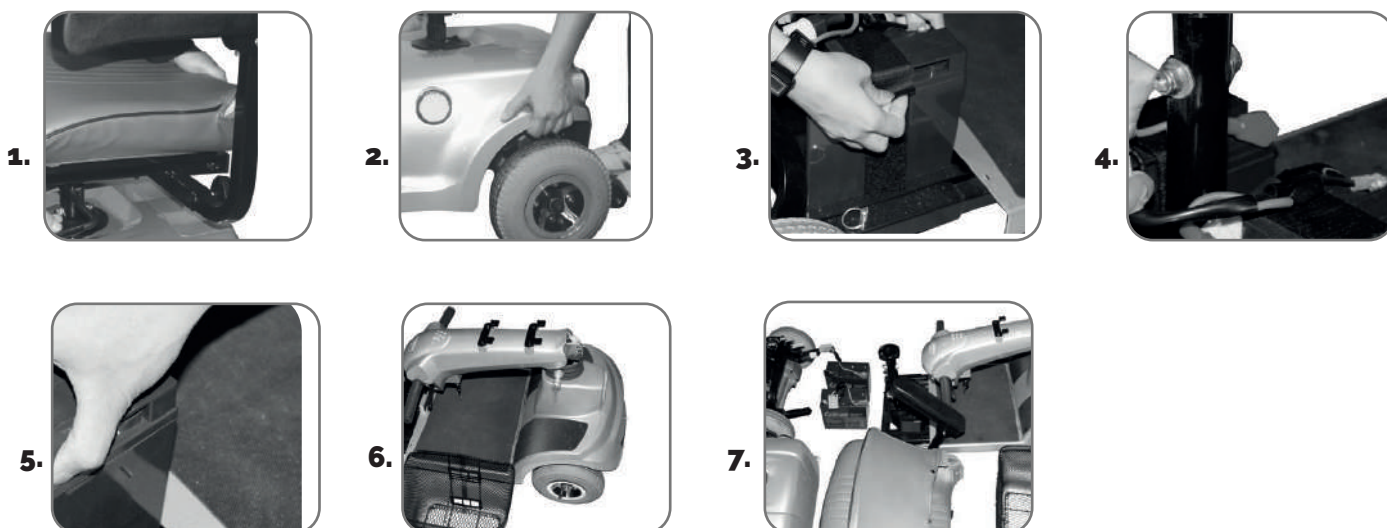All maintenance and repair of your scooter should be done by an authorized dealer.

The following areas required periodic inspection:

- Tire pressure between 30-50 psi.
- All electrical connections are firmly attached.

**SPECIFICATIONS**

|                        |                               |
|------------------------|-------------------------------|
| Overall Length         | 116 cm / 45.7"                |
| Overall Width          | 58 cm / 22.8" (Armrest Width) |
| Overall Height         | 95 cm / 37.4"                 |
| Front Wheels           | 23 cm / 9"                    |
| Rear Wheels            | 23 cm / 9"                    |
| Weight W/ Batteries    | 67.5 kg / 148.9 lbs           |
| Max. Speed             | 8 kmph / 5 mph                |
| Weight Capacity        | 135 kg / 300 lbs              |
| Ground Clearance       | 4 cm / 1.6"                   |
| Grade Climable         | 12 degree                     |
| Curb Climable          | 6 cm / 2.4"                   |
| Turning Radius         | 141.5 cm / 55.7"              |
| Brake                  | Electro-Mechanical            |
| Seat Type              | Swivel Light Weight Foldable  |
| Seat Width             | 46 cm / 18"                   |
| Motor Size             | 450W, 3700 r.p.m              |
| Battery Size           | (2) 12V, 36Ah                 |
| Weight of Battery Pack | 22.5 kg / 49.6 lbs            |
| Travel Range           | 34 km / 21 Miles              |
| Battery Charger        | 5A Off Board                  |

**TROUBLESHOOTING**

To check the Self-Diagnostic Warning Light, turn on the key and count the number of flashes

| Flash | Description                | Meaning                                                                                                                                                                                                                                                                                                                                                                                                                              |
|-------|----------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1     | Low Battery                | The batteries are running low.<br>• Recharge the batteries                                                                                                                                                                                                                                                                                                                                                                           |
| 2     | Low Battery Fault          | Battery charge is empty.<br>• Recharge the batteries.                                                                                                                                                                                                                                                                                                                                                                                |
| 3     | Battery voltage too high   | Battery voltage is too high. This may occur if overcharged &/or traveling down a long slope.<br>• Turn the scooter power off then on again<br>• If traveling down a slope, reduce your speed.                                                                                                                                                                                                                                        |
| 4     | Current limit time-out     | The motor has been exceeding its maximum current rating for too long. This may be due to a faulty motor<br>• Check the motor and associated connections and wiring.<br>• Turn the controller off, leave for a few minutes and turn back on again.                                                                                                                                                                                    |
| 5     | Brake fault                | Either a park brake release switch is active or the park brake is faulty.<br>• Check if the lever is in Automatic Mode (D).<br>• If this flash code does not appear until the throttle has been moved out of neutral, check for a short in the park-brake circuit.<br>• If this flash code appears at power-up, check for an open circuit in the park-brake circuit.<br>• Check the park brake and associated connections and wiring |
| 6     | Out of neutral at power-up | The Throttle is out of neutral when turning the controller on.<br>• Ensure the throttle is in neutral when turning the scooter on.<br>• The Throttle may require re-caPiscition.<br>• Check the its associated connections and wiring.                                                                                                                                                                                               |
| 7     | Speed Pot Fault            | The throttle, speed limit pot or their associated wiring may be faulty.<br>• Check the throttle and speed pot and associated connections and wiring.                                                                                                                                                                                                                                                                                 |
| 8     | Motor Voltage Fault        | The motor or its associated wiring is faulty.<br>• Check the motor and associated connections and wiring.                                                                                                                                                                                                                                                                                                                            |
| 9     | Other error                | The controller may have an internal fault.<br>• Check all connections & wiring.                                                                                                                                                                                                                                                                                                                                                      |

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CHIEN TI ENTERPRISE CO., LTD
Queralto S.A., Polígono Industrial El Pino,