MANUAL DE INSTRUCCIONES/ INSTRUCTIONS MANUAL/ ROLLATOR PATERNA

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ENGLISH

Read this instructions in order to familiarize with your new rollator well before use. The model Paterna has a weight capacity of 110kg.

How to use the rollator:

1. Lock the brakes on the rollator by pushing the brake handles down.
2. Place both feet side-by-side in the middle of the rollator frame.
3. Release the brakes by pulling the brake handles up.
4. Grasp the hand grips on either side of the rollator.
5. Push the rollator forward to a comfortable distance.
6. Your toes should be even with the back wheels of the rollator.
7. Step forward toward the inside of the rollator frame with your weaker leg.
8. Step forward with your stronger leg while supporting yourself with the hand grips. Both feet should be side-by-side between the wheels of the rollator. Continue walking by repeating Step 5 and Step 7.

Tips and Warnings

Keep a part of your body, like your toes, within the rollator frame at all times. When standing from a sitting position, push up from the seat instead of pulling up on the rollator handles. Apply the brake before sitting and make sure the brake is applied before standing to use the rollator. When turning, take small steps and keep your hips parallel to the front bar of the rollator. Avoid twisting your body away from the rollator. Don't sit on the rollator seat without activating the brakes and don't use this device as a temporary wheelchair. The rollator can tip over causing injury. Ensure all wheels of the rollator are on level ground before walking. Don't hunch forward over the rollator. Don't use the rollator on an escalator or stairs. Don't place heavy objects in the basket because this can cause the rollator to tip over. Remember this product should only be used on flat and smooth surfaces.

Cleaning and maintenance

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth. Do not use solvents, bleach, synthetic detergents, wax polishes, aerosol sprays or other abrasives that may damage the product.

Notice

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Indications

- 1) The rollator can be used to assist the elderly or people with any type of mobility problems, with walking or shopping.
- 2) The user's weight should not exceed 110KG; otherwise it will be deformed. The carrying weight of the shopping basket should not exceed 10KG;

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD
SUNGO Europe B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.