EN This transit wheelchair model Júpiter from Mobiclinic is designed for occasional use by a single occupant weighing up to 100 kg and is suitable for indoor and outdoor use. The Júpiter transit wheelchair can be manually moved forwards and backwards.

Fold and unfold the wheelchair:

Weight capacity 100 kg

To unfold the wheelchair, please follow the steps below:

1. Take the wheelchair out of the bag and lift the backrest.
2. Hold the aluminium bar between the rear wheels with your foot and at the same time pull the seat strap until you hear the following sound the seat strap until you hear a click to fix the ends.
3. Engage the safety levers on both sides.
4. Lower the armrests on both sides.
5. Unfold the aluminium footrest, and the wheelchair will be completely unfolded.

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i To fold the wheelchair, do the same process in reverse, making sure that the side folds are always facing downwards.

IMPORTANT ASPECTS TO BE TAKEN INTO ACCOUNT

The user or caregiver should check the safety of the wheelchair at regular intervals.

1. Check the handgrips and non-slip rubber pads and make sure they are firm and secure.
2. For correct adjustment, check the wheel locks and ensure that the large wheels are locked when they are activated. are locked when activated.
3. Check that all bolts, nuts and joints are securely in place.
4. If other conditions affecting safety are present, discontinue use of the wheelchair.
5. Do not stand on the footrests when getting out of the wheelchair.
6. Keep the wheelchair away from fire.
7. Remember to activate the wheelchair lock before getting into or out of the wheelchair.
8. Do not lean forward in the wheelchair unless both feet are flat on the floor. This is especially important for users with a cast.

Activating and deactivating the brake system of the Saturn wheelchair

Image 1. To temporarily brake the chair, press the brake lightly. To advance, release the brake.

Image 2. To activate the locking system, press down on the brake. To unlock, lift the brakes upwards and you can continue to move forward.

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

INDICATIONS

The wheelchair is used as a mobility tool for disabled people, patients and elderly people who have difficulties in action. Beginners should be accompanied by assistants to avoid accidents due to improper operation.

CLEANING AND MAINTENANCE

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth. Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage your product.

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD
SUNGO Europe B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.