MANUAL DE INSTRUCCIONES

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ENGLISH

This wheelchair is designed for frequent or occasional use by a single occupant of up to 100 kg (Opera)/110 kg (Sphinx, Pyramid, Obelisk) and can be used indoors and outdoors. The wheelchair can be manually moved forwards and backwards.

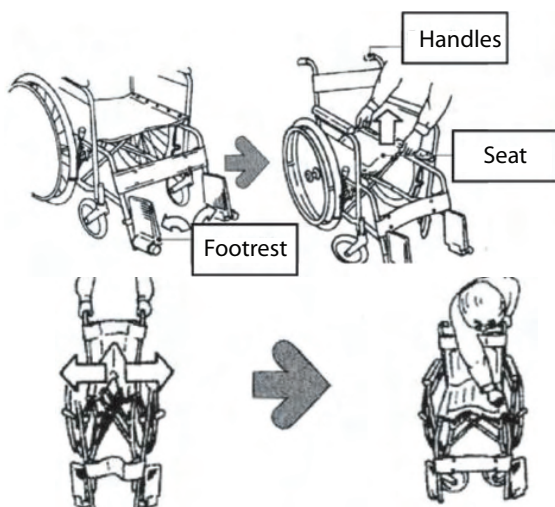
Notice

Notice to the user and/or patient: any serious incident that has occurred in relation to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications

The wheelchair is used as a mobility tool for disabled people, patients and elderly people and patients.

Folding and unfolding the wheelchair: To fold the wheelchair, simply lift the centre edge of the seat upholstery from below. To unfold, tilt the wheelchair slightly to one side to lift the wheels off the floor. Then press down on one or both seat rails. Before folding the chair, completely remove the footrests or make sure they are fixed and in an upright position.

Locking the wheels: Push the wheel locks forward until they engage in the locked position. Do not attempt to sit down or stand up without having activated the brakes. Always activate both brakes simultaneously.

Adjusting the length of the footrests: The position of the footrests (standard - flat - or raised) can be adjusted to suit the length of the user's legs. The footrest should be adjusted to support the weight of the

thighs. If the footrest adjustment is too long, the thighs will be pressed under the front of the seat upholstery. If the footrest adjustment is too short, the user's knees will rise and the buttocks will have to bear excessive weight. Both situations lead to an increased risk of bedsores. The user's knees and hips should be at approximately the same level. The lower edge of the footrest should be at least 7 cm above the floor to allow sufficient clearance when going over ramps or slopes. To do this, loosen the adjusting screw with a spanner and fold the foot plates inwards or outwards to achieve the desired position. Then tighten the screw carefully.

Using the rear support pedals and brakes: Use the rear support pedals to raise the front wheels, e.g. when climbing a kerb. To do this, push the rear support pedals down with one foot. Do not lift the small front wheels by pushing down on the handles, as this could damage the wheelchair.

- Climbing a kerb: Approach the kerb from the front. The caregiver should use the rear support pedals to raise the front wheels and lower them onto the kerb once the kerb has been raised. Finally, the caregiver should push the wheelchair forward, lifting it slightly to raise the kerb if necessary.

- Getting off a kerb: Place the front wheels in front of the kerb edge. The caregiver should use the support pedals to raise the front wheels and tilt the occupant back slightly. While keeping the wheels up, lower the wheelchair slightly towards the kerb.

Warning: When lowering a kerb, the front wheels should be raised to prevent the occupant from falling. These are recommendations only; they may not be suitable for all cases.

Cleaning and maintenance

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth. Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage the product.

Important points to note:

The user or caregiver must check the safety of the wheelchair on a regular basis.

1. Check the handgrips and non-slip rubber pads and make sure that they are solid and secure.
2. For a correct fit, check the wheel locks and ensure that the large wheels are locked when they are activated.
3. Check all bolts, nuts and joints to ensure that they remain solid.
4. If the wheelchair is fitted with tyres, check that the tyre pressure is correct.
5. Check that the adjusted length of the footrest is correct.
6. If other conditions affecting safety are present, discontinue use of the wheelchair.
7. Remember to engage the wheelchair lock before transferring to or from the wheelchair.
8. Do not lean forward in the wheelchair unless both feet are flat on the floor. This is especially important for users with heavy casts with raised footrests.

Make sure that the brakes are activated before getting up or sitting in the wheelchair.

Do not stand on the footrests when getting out of the chair, as the wheelchair may tip over.

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD
SUNGO Europe B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.