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INSTRUCTION MANUAL ELECTRIC SCOOTER VIRGO

NOTICE

Notice to the user and/or patient: Any serious incident that has occurred in connection to the product must be communicated to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

INDICATIONS

- Only one user can ride the scooter.
- The lowest speed must be used when carrying out the following actions: Going up, going down or crossing a slope or sloping terrain and also on uneven ground, ramps and soft surfaces.
- Keep away from flammable objects while charging the device.

CONTRAINDICATIONS

- Never try to go up or down a slope when the surface is slippery or there is a risk of skidding.
- Do not ride on a slope over 6 degrees.
- Avoid shifting the center of gravity when turning left or right.
- Direct exposure to rain or humidity will stop the scooter from functioning properly, it may cause the scooter to oxidise.
- Never put the scooter in neutral when you are on a slope.
- Do not try to modify or manipulate the scooter in regards to its mechanism or structure.
- Do not ride the scooter on stairs or escalators.
- Do not use the device while it is charging.

The Mobiclinic Virgo scooter helps to solve, in a comfortable and autonomous way, mobility problems for people who have difficulty walking. The Virgo scooter is designed for indoor and outdoor use.

OPERATING YOUR SCOOTER

Before operation, please remove the Lock Nut (A) and press button of Circuit Breaker (L) to activate the system.

1. Before operating your scooter, check the following:

- Check that the immobiliser lever is in D mode.
- Check that the speed control is in low speed (turtle) mode.

2. Sit on scooter and turn on key (C). The Battery Gauge light should be lit in the green, yellow, and red color zones. The Self-Diagnostic Warning Light should not be blinking.

3. When your hands rest comfortably on the handlebars, the thumb control levers should be within easy reach. The right lever moves the scooter forward, the left one moves it backward. When you release both levers the scooter will stop.

4. Steer the scooter by turning the whole steering column the way you want to go.


5. Practice driving where there are no obstacles. Start at the slowest speed and drive forward and backward; make some turns. As you get more comfortable you can increase the speed by turning the speed dial toward the picture of the rabbit.

6. If two Battery Gauge LED Lights be lit, you should plan to recharge the batteries very soon.

7. If the scooter stops, locate the circuit breaker (L) on the top of the battery pack. Push it and try the scooter again.

8. When you are finished riding, turn off the key before getting off.

9. If you are finished riding for the day, immediately recharge the batteries.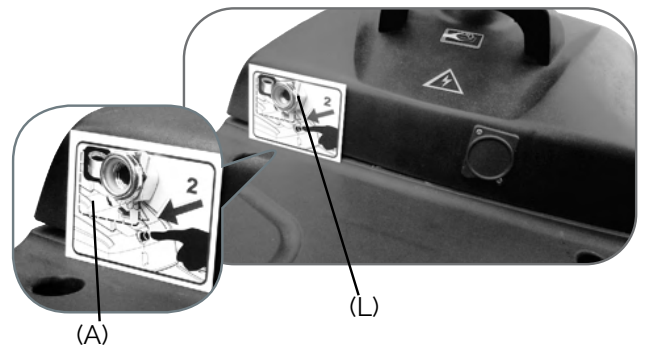

OTHER OPERATING INFORMATION



Hill climbing: You may need to use a higher speed. Turn to lower speed before going downhill.

Down slopes: To proceed down steep slopes slowly, set speed control in proximity of the turtle. This enables driver control, as the closer the speed control is set toward the turtle, the slower it will become. However, this scooter will not self-accelerate down hills due to the automatic braking taking effect should you attempt to drive too fast.

If the scooter breaks down and must be moved, get off the scooter, push the Free-Wheeling Lever to N, move the scooter slowly to a safe location, and move the lever back to D.

IDENTIFYING THE PARTS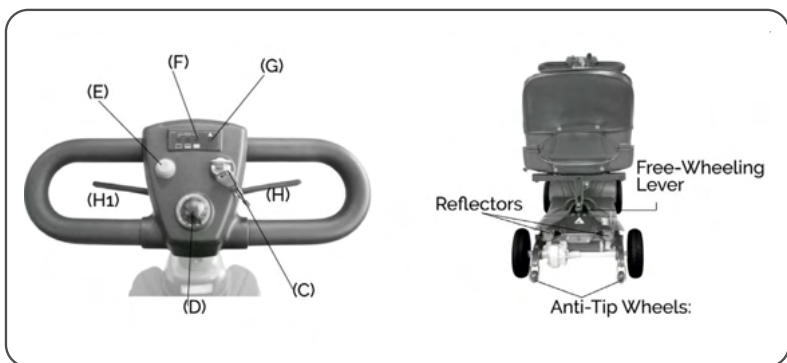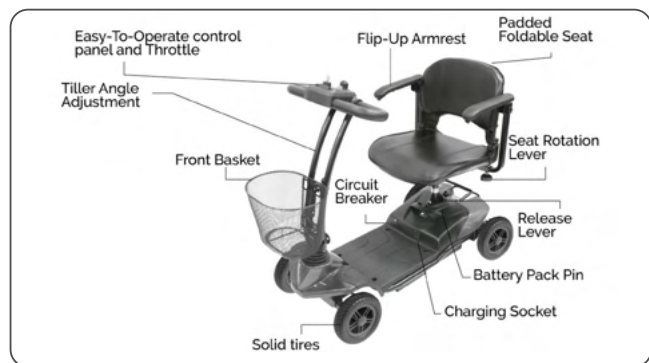

FUNCTION OF PARTS

Top control panel

1. Speed Control Knob: The Rabbit means fast and Turtle is slow. By turning this, you control the total speed transferred to the thumb controls.
2. Self Diagnostic Warning Light: Flashing indicates problems with the scooter.
3. Battery Gauge: This light is yellow. Below you will find three color ranges: red, yellow and green. When the light is on in all three color ranges, the batteries are fully charged. When the light is on only in the red, or red and yellow areas, the batteries need to be recharged.

Beneath top control panel

1. Throttle: Pushing the right Throttle moves the scooter forward. Pushing the left Throttle moves the scooter backward. (This can be reversed if required by local dealer.) Releasing both engages the automatic brake. This is also your accelerator. The further you depress them, the faster you go. (Subject to the position of the Rabbit/Turtle control).

At base of steering tiller

1. Adjust the steering handlebars: Push down on the handlebars to adjust them. Pull up to lock the turn of the steering column at an angle that is comfortable for you.

Under the seat

1. Seat adjustment lever:

- Handlebar clearance adjustment: pull up on the lever on the front of the seat to adjust the handlebar clearance.
- Rotation adjustment: press the lever backwards to rotate the seat; press it forwards to lock it in the desired position.
- Armrest adjustment: Loosen the two thumbscrews to adjust the width between the armrests. Tighten them again to secure them in the desired position.

Rear body

1. Rear Cover: Protects batteries from damage.

2. Anti-tip wheels: Helps keep the scooter from tipping over.

3. The Free-Wheeling Lever: It will let you change the Mode of your scooter (Neutral Mode or Drive Mode). The Neutral Mode (N) will let you move the scooter without energy, you will be able to push it, whereas the Drive Mode (D) will let you drive the scooter and immobilize it when you want to park it.

You will find some stickers next to the lever, they will help you to know the Mode of your scooter.

If you would like to change from Neutral Mode to Drive Mode, you must turn off the scooter. After changing the Mode, you will be able to turn on again your scooter.

CIRCUIT BREAKER (L) FREE-WHEELING LEVER (L1)

If scooter's circuit system malfunctions or over loaded, the circuit system will automatic shut down the power to ensure driver's safety. After the power is off, press button (L) to regain power. If you want your scooter to be immobilized, there is a lever in back side (L1)

The Free-Wheeling Lever (L1) will let you change the Mode of your scooter (Manual Mode or Automatic Mode). The Manual mode will let you move the scooter without energy, you will be able to push it, whereas the Automatic Mode will let you drive the scooter and immobilize it when you want to park it. You will find some stickers next to the lever, they will help you to know the Mode of your scooter. If you would like to change from Manual Mode to Automatic Mode, you must turn off the scooter. After changing the Mode, you will be able to turn on again your scooter.

ASSEMBLING YOUR SCOOTER

First, lift the lever (I) to disengage the pin. There are three levels, so you can adjust the steering fore-aft to the most comfortable angle. Release lever (I) and ensure the pin is fully engaged to lock the steering column in position. In order to set the seat (K), lift lever (J) upward to disengage pin. Simultaneously, rotate seat (K) (or lift to disassemble the seat) to the most comfortable angle. Release lever (J) and ensure the pin is fully engaged to lock the seat in position (379mm/390mm/410mm).

To fit the armrests, remove the plugs (L3) and locate the two holes behind the seat. Adjust the armrests so that they are fixed when bolted to the seat, but allow the width to be adjusted and folded down. Secondly, to fit the reclining armrests, remove the plugs (L3) underneath the seat. Underneath the seat you will find two holes (L4), provided in a way that secures these by screwing them into the chair. Secure the screws (L5). Finally, place the basket (for carrying objects) at the front and the rear view mirror in its place.

ELECTROMAGNETIC INTERFERENCE AND WARNINGS

Powered wheelchairs and motorized scooters may be susceptible to electro- magnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones. The interference (from radio wave sources) can cause the motorized scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the motorized scooter control system.

1. Do not operate hand-held transceivers (transmitters-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the motorized scooter is turned ON.
2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
3. If unintended movement or brake release occurs, turn the motorized scooter OFF as soon as it is safe.
4. Be aware that adding accessories or components, or modifying the motorized scooter, may make it more susceptible to EMI.


20 volts per meter (V/m) is a generally achievable and useful immunity level against EMI (as of May 1994). The higher the level, the greater the protection. This product has an immunity level of at least 10 V/m.

CHARGING THE BATTERIES

Batteries must be charged before using the scooter for the first time, it should be charged at night and recharged after each day use.

First, insert battery charger cord into Charging Socket (M) on battery pack and plug other end of power cable into a standard electrical wall outlet.

The scooter must be turned off to be charged. Please, do not replace the original batteries, contact your dealer if you need to replace them. If at any time battery charger light flashes green over 40 minutes, this indicates abnormal charging occurred, so you should check if charger plug is correctly positioned or if the scooter is turned off.

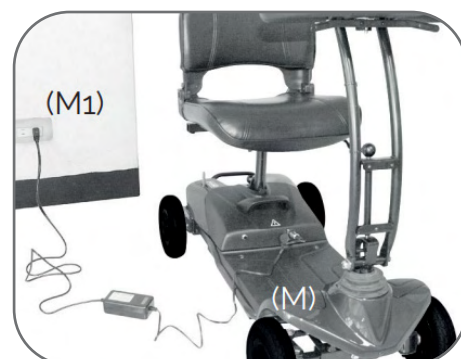
DISASSEMBLING YOUR SCOOTER

1. Remove seat by pulling up Seat Rotation Lever (J). Then lift away from scooter.
2. As indication, Push Release Lever (N) backward and pull Battery Pack Handle (O), to remove batteries from scooter. Caution: batteries are heavy.

3. Lift up front & rear frame's fixation pin (P); and hold and pull rear frame as arrow indicates direction to disassemble front and rear frame.

4. Lift the Lever (I) to bend down the steering to the lowest hole position and fold down the steering column as shown in the following picture.

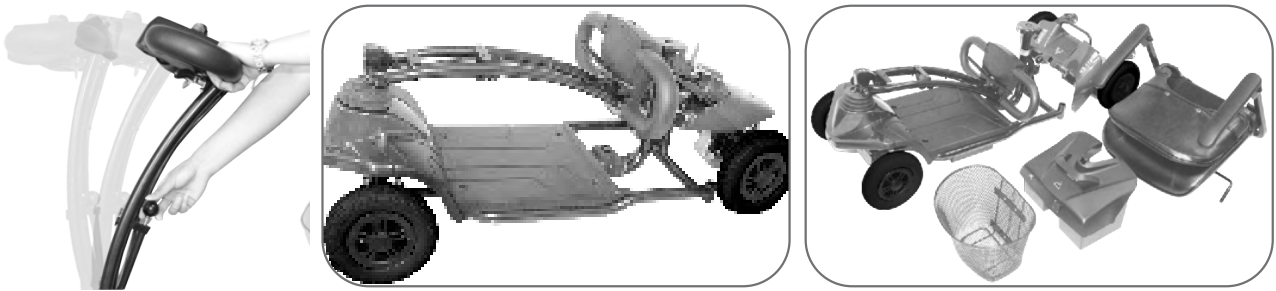CLEANING AND MAINTAINING YOUR SCOOTER

If your scooter is dirty, use a damp or lightly soapy cloth to wipe it down. Do not use running water to wash or rinse the scooter in order to protect the electrical parts. Polish with an automotive liquid polish.

All maintenance and repair of your scooter should be done by an authorized dealer.

The following areas required periodic inspection:

- Tire pressure between 30-50 psi.
- All electrical connections are firmly attached.

Specifications

Overall Length	128 cm / 50.4"
Overall Width	61cm / 24"
Overall Height	111 cm / 43.7"
Front Wheels	26 cm / 10"
Rear Wheels	26 cm / 10"
Weight W/ Batteries	87.9 kg / 194 lbs
Max. Speed	8 kmph / 5 mph
Weight Capacity	135 kg / 300 lbs
Ground Clearance	4 cm / 1.6"
Grade Climbable	12 degree
Curb Climbable	7 cm / 2.75"
Turning Radius	152 cm / 59.8"
Brake	Electro-Mechanical
Seat Type	Swivel Mid-Back W/ Headrest & Seat Sliding Mechanism
Seat Width	45.5 cm / 18"
Motor Size	500W, 3400 r.p.m
Battery Size	(2) 12V, 36Ah
Weight of Battery Pack	22.5 kg / 49.6 lbs
Travel Range	30 km / 18.6 Miles
Battery Charger	5A Off Board

TROUBLESHOOTING

To check the Self-Diagnostic Warning Light, turn on the key and count the number of flashes

Flash	Description	Meaning
1	Low Battery	The batteries are running low. •Recharge the batteries
2	Low Battery Fault	Battery charge is empty. • Recharge the batteries.
3	Battery voltage too high	Battery voltage is too high. This may occur if overcharged &/or traveling down a long slope. • Turn the scooter power off then on again • If traveling down a slope, reduce your speed.
4	Current limit time-out	The motor has been exceeding its maximum current rating for too long. This may be due to a faulty motor • Check the motor and associated connections and wiring. • Turn the controller off, leave for a few minutes and turn back on again.
5	Brake fault	Either a park brake release switch is active or the park brake is faulty. • Check if the lever is in Automatic Mode (D). • If this flash code does not appear until the throttle has been moved out of neutral, check for a short in the park-brake circuit. • If this flash code appears at power-up, check for an open circuit in the park-brake circuit. • Check the park brake and associated connections and wiring
6	Out of neutral at power-up	The Throttle is out of neutral when turning the controller on. • Ensure the throttle is in neutral when turning the scooter on. • The Throttle may require re-caVirgotion. • Check the its associated connections and wiring.
7	Speed Pot Fault	The throttle, speed limit pot or their associated wiring may be faulty. • Check the throttle and speed pot and associated connections and wiring.
8	Motor Voltage Fault	The motor or its associated wiring is faulty. • Check the motor and associated connections and wiring.
9	Other error	The controller may have an internal fault. • Check all connections & wiring.

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CHIENTI ENTERPRISE CO., LTD
MedNet EC-REP GmbH
Queralto S.A., Polígono Industrial El Pino,