MANUAL DE INSTRUCCIONES

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TROYA / TROYA PLUS / TROYA PLUS ULTRA

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

INDICATIONS

This product is intended for disabled or elderly people who have difficulty moving.

CONTRAINDICATIONS

Not recommended for people who are too weak to sit up. People who have difficulty in operating the wheelchair due to various disabilities.

1. CONTENTS OF THE BOX

1 Wheelchair

2 Accessories bag

Joystick

Charger

Tools

Anti-tipper

3 Model **Troya** (1 Battery, 17km autonomy)

Model **Troya PLUS** (2 Batteries, 34 km autonomy)

Model **Troya PLUS ULTRA** (3 Batteries, 51 km autonomy)

2. MAIN STRUCTURE

3. HOW TO FOLD OR UNFOLD THE CHAIR

Method 1. To open the wheelchair:

Method 2. To open the wheelchair:




Method 3. How to fold the chair:

4. ELECTRICAL DIAGRAM (depending on the model)


Controller LH-C

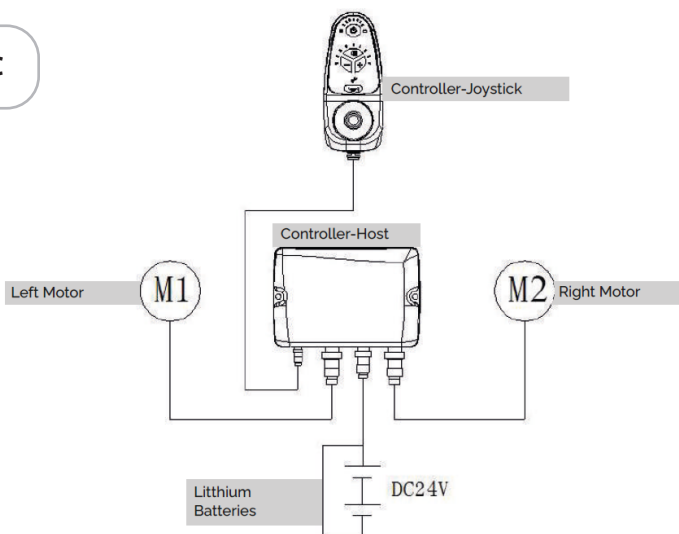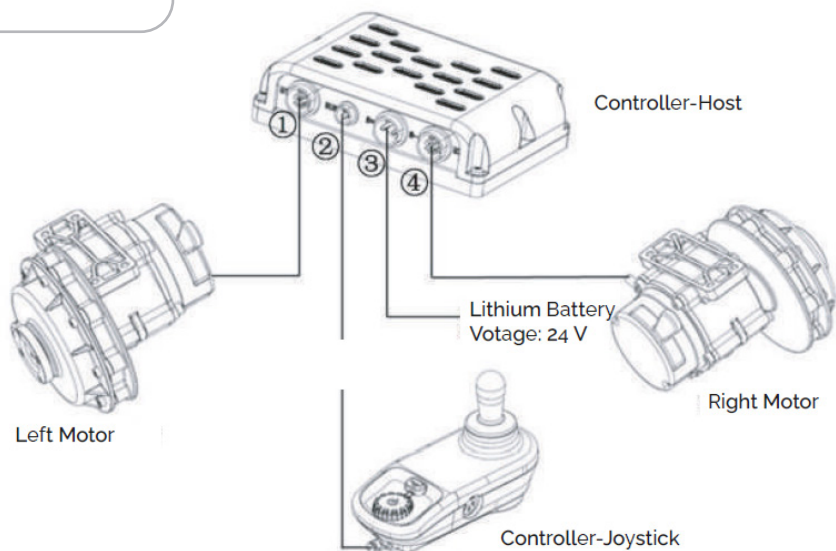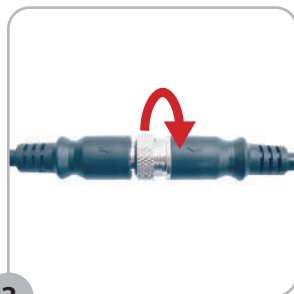Controller SY-C20**5. HOW TO INSTALL AND CONNECT THE JOYSTICK**

6. HOW TO INSTALL AND REMOVE BATTERIES

Install the lithium batteries:

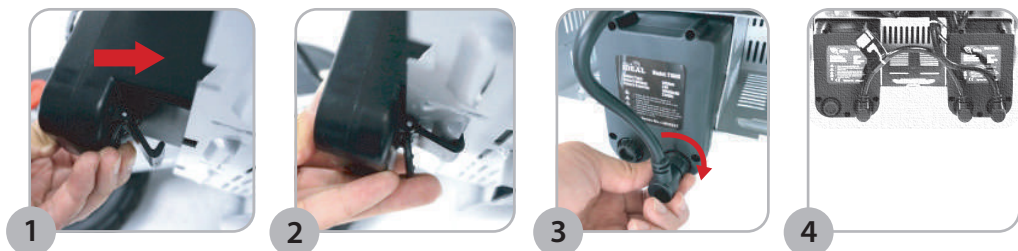Remove the lithium batteries:


7. HOW TO STEER AND CONTROL THE WHEELCHAIR

7.1. Changing the driving mode

The manual mode (M) allows the user to move the wheelchair without using the battery, i.e. by pushing the wheelchair. While the automatic mode (D) will allow you to both drive the wheelchair and keep it stationary when you want to park it.

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
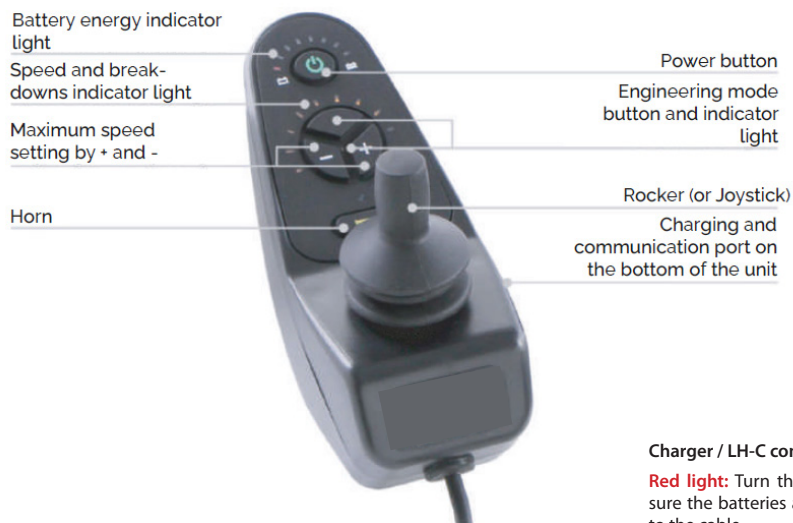Controller LH-C * Square wave**Charger / LH-C controller indicator light:**

Red light: Turn the power on and make sure the batteries are securely connected to the cable.

Orange light: Loading.

Green light: Fully loaded.

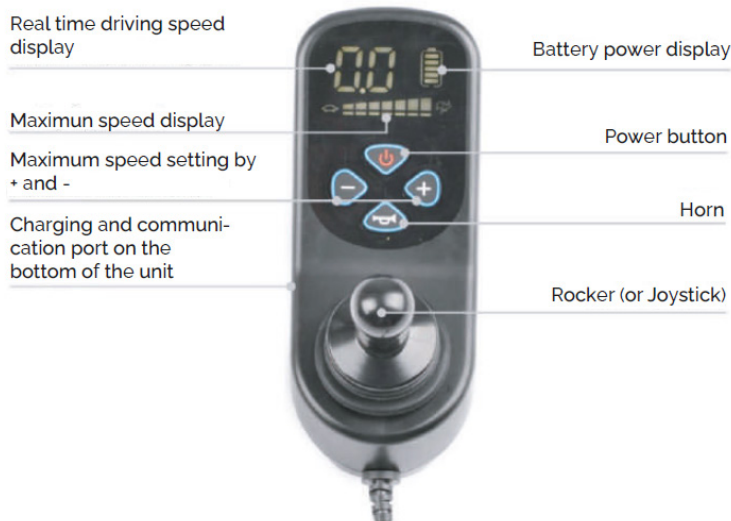Controller SY-C20 * LCD sine wave

8. MORE FUNCTIONS

8.1. Folding armrests

8.2. Folding footrests

8.3. Height-adjustable backrest

9. CHARGING OF LITHIUM BATTERIES

Charging time is usually 2 to 8 hours.

Method 1. To load via the joystick:

1

Connect the charger

2

Connect the joystick

3

Connect to the mains
socket

Method 2. For charging by cable (lithium batteries are charged directly) (Cable not included, sold separately):

1

2

3

CAUTION:

1. For indoor use only, do not expose to rain.
2. Disconnect power before removing the battery charger.
3. Place the charger in a well-ventilated area during charging.
4. Use charger only with 24V lithium batteries.
5. Charge the battery every 3 months if it is not used for a long time.

10. TECHNICAL CHARACTERISTICS



PARAMETERS	
Engine	LH-C: 250W*2, DC 24V / SY-C20: 220W*2, DC 24V
Maximum speed	8,5 km/h
Braking distance	≤ 1,0 m
Max. slope capacity	15°
Turning radius	85 cm
Rear wheel	30,48 cm / Maximum pressure 2,5 bar
Front wheel	20,32 cm
Net weight (without battery)	24,8 kg
Maximum user weight	120 kg
Lithium battery	24V10Ah/pc
Net weight of battery	1,7 kg/pc
Autonomy	LH-C: 17 km/1 battery - 34 km/ batteries - 51 km/ 3 batteries SY-C20: 20 km/1 battery-40 km/2 batteries-60 km/ 3 batteries
CD changer output	24V/24
AC charger input	100-240V, 50/60Hz

MEASUREMENTS	
Seat width	450 mm
Seat depth	440 mm
Seat height	500 mm
Wide backrest	440 mm
Backrest width	470 mm
Distance from seat to armrest	210 mm
Ground clearance	65 mm
Wheelbase	600 mm
Total length	1040 mm
Total width	640 mm
Total height	910-1060 mm (height adjustable)
Dimensions folded	640x340x740 mm (0,16m ³)
Carton box size	730x420x840 mm (0,26m ³)

11. SOLUTIONS FOR COMMON FAILURES (depending on the model)

LH-C CONTROLLER

When the speed indicator light flashes on the controller, it indicates that the power wheelchair is in an abnormal condition. The meaning of the flashing status and the recommended method of elimination are shown in the table below.

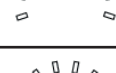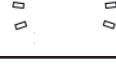Speed indicator light Flashing status	Description of the fault and warning	Solution
First light flashing 	Fault in the behaviour of the left engine	1. Check if the left engine compartment is damaged. 2. Check whether or not the left engine cable is connected.
Second light flashing 	Fault in the right-hand engine compartment	1. Check if the right engine compartment is damaged. 2. Check whether or not the right engine cable is connected.
Third light flashing 	Low battery level	Charge the battery (1st step) and then check the battery level again.
Fourth light flashing 	Left engine failure	Check the left motor and its connections
Fifth light flashing 	Right engine failure	Check the right motor and its connections
Sixth light flashing 	Left brake failure	Check the left engine brake and its connections
Seventh light flashing 	Right brake failure	Check the right engine brake and its connections
Eighth light flashing 	Fallo en el controlador	90% probability, the malfunction is caused by the power tube of the control unit, it is suggested to replace the control unit or the main controller, or contact after-sales service.
Ninth light flashing 	Fallo en la conexión del joystick y controlador	1. Check the connection of the joystick cable to the controller. 2. The controller is damaged.

Tenth light flashing 	Motor overload	Release the control bar to resume.
First and tenth light flashing 	High battery voltage	Check the output voltage, it must not be higher than 35V.
All lights flashing 	The rocker is not in the centre of the joystick.	1. Do not touch the rocker arm when it is switched on. 2. Turn off, shake the rocker, and try to bring it back to the centre, if the problem persists, please contact after-sales service.

SY-C20 CONTROLLER

When the fault code appears on the controller display, it indicates that the power wheelchair is in an abnormal condition. The meaning of the display and the recommended method of elimination are shown in the following table.

Fault code	Indicator status	Alarm sound	Fault description and warning	Method of elimination
C1	Long blink	Long time alarm	Communication failure	Check if the wiring harness is correctly installed and connected
C2	Long blink	Long time alarm	The joystick is not in the home position	Switch off the power, release the joystick to the centre position and then switch it on
C3	Long blink	Long time alarm	Control conflict (If your wheelchair has front controllers and rear controllers)	Disconnect power to the front controllers and the rear controllers, and select one to turn on
C4	Long blink	Long time alarm	The brake lever has not been set to the electric state	Check if the engine is in running condition
C5	Long blink	Long time alarm	Overload warning (First stage)	Reduce the load on the wheelchair or release the joystick to the centre position, the wheelchair is reset
C5	Short blink	Short time alarm	Overload warning (Second stage)	
C5	Long blink	Long time alarm	Overload warning (Third stage)	
C6	Long blink	Long time alarm	Ignition failure	Restart
C7	Long blink	Long time alarm	Left and right-hand wheel overvoltage warning	The joystick returns to the centre position until the overvoltage warning is removed

Fault code	Indicator status	Alarm sound	Fault description and warning	Method of elimination
C8	Long blink	Long time alarm	Voltage warning on left and right wheel	Please charge the battery in time
CE	Long blink	Long time alarm	Failure in system parameters	Reboot or contact sales service
L1	Long blink	Long time alarm	Left wheel overcurrent fault	
L2	Long blink	Long time alarm	Excessive deflection of the current left wheel	
L3	Long blink	Long time alarm	Left wheel circuit over-heating warning	Switch off power, restart after drive wheel cools down
L6	Long blink	Long time alarm	Left wheel encoder failure	Reboot or contact sales service
L7	Long blink	Long time alarm	Left wheel brake failure	
L8	Long blink	Long time alarm	Left wheel phase failure	
R1	Long blink	Long time alarm	Right wheel overcurrent fault	
R2	Long blink	Long time alarm	Excessive deviation of the current right wheel	Switch off power, restart after drive wheel cools down
R3	Long blink	Long time alarm	Right wheel circuit overheating warning	
R6	Long blink	Long time alarm	Right wheel encoder failure	Restart or contact after-sales service
R7	Long blink	Long time alarm	Right wheel brake failure	
R8	Long blink	Long time alarm	Right wheel phase failure	

If the problems shown above cannot be solved, please do not disassemble the wheelchair by yourself. Please contact the after-sales service.

Made in P.R.C

ES - Peso máximo 120 kg

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IT - Peso massimo 120 kg

FR - Poids maximum 120 kg

PT - Peso máximo 120 kg

DE - Maximales Gewicht 120 kg

NL - Maximaal gewicht 120 kg

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SUZHOU IDEAL MEDICAL CO.,LTD

No. 88 Huachi Street, Suzhou Industrial Park Suzhou,
215000, Jiangsu, PRC

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan
den IJssel, The Netherlands

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Polígono Industrial El Pino, Calle Pino Albar, 24, 41016,
Sevilla (Spain)

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GRUPO R. QUERALTÓ S.A.