 **BAUERFEIND AG**  
Triebeser Straße 16  
07937 Zeulenroda-Triebes  
Germany  
P +49 (0) 36628 66-35 00  
F +49 (0) 36628 66-39 99  
E [info@bauerfeind.com](mailto:info@bauerfeind.com)

## ViscoHeel® K

Gebrauchsanweisung  
Instructions for use  
Notice d'utilisation  
Istruzioni per l'uso  
Instrucciones de uso

(de)	deutsch	2	(sr)	srpski	32
(en)	english	4	(sl)	slovenski	34
(fr)	français	5	(ro)	romanian	35
(nl)	nederlands	7	(el)	ελληνικά	37
(it)	italiano	9	(tr)	türkçe	39
(es)	español	11	(ru)	русский	41
(pt)	português	13	(et)	eesti	43
(sv)	svenska	15	(lv)	latviešu	44
(no)	norsk	16	(li)	lietuvių	46
(fi)	suomi	18	(uk)	українська	48
(da)	dansk	20	(kk)	қазақ	50
(pl)	polski	21	(he)	עברית	53
(cs)	česky	23	(ja)	日本語	54
(sk)	slovensky	25	(ko)	한국어	55
(hu)	magyar	27	(ar)	عربي	58
(hr)	hrvatski	28	(zh)	中文	60
(mk)	македонски	30			

(de) deutsch

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en english

## Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

### Intended purpose

ViscoHeel K is a medical device for the feet. It is a viscoelastic heel cushion. It is used as a supination or pronation cushion. ViscoHeel K can be used to strategically correct varus and valgus misalignments of the heel bone as well as misalignments of the leg axis.

### Indications

- For inward and outward (varus and valgus) misalignment of the heel

### Risks of using this product

#### ⚠️ Caution\*

ViscoHeel K must only be worn in accordance with the specifications in these instructions for use. To ensure the optimum fit of the ViscoHeel K, the size required must be determined exactly. Only a specialist<sup>1</sup> may carry out the (initial) fitting of the product and provide instructions.

Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you. If you notice any unusual changes or your

symptoms get worse, stop using the product and please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids. No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed.

### Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up. The foot orthosis is not suitable for people who have diabetes (diabetic foot syndrome) or rheumatism (rheumatoid foot deformities), as it does not provide soft enough cushioning for these types of foot complaints. This foot orthosis is not suitable for patients who are allergic against silicone.

### Application instructions

#### Inserting the ViscoHeel K

Use the ViscoHeel K in shoes with a stiff heel counter. Discuss the use of the ViscoHeel K with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

The ViscoHeel K heel cushions are used to correct varus or valgus misalignments of the heel. The heel cushions are marked with the letters A and B.

- To correct a varus misalignment of the heel, insert the heel cushions as follows: heel cushion A in the right shoe, heel cushion B in the left shoe.
- To correct a valgus misalignment of the heel, insert the heel cushions as follows: heel cushion A in the left shoe, heel cushion B in the right shoe.

Unless otherwise prescribed by your physician, place **both** ViscoHeel K heel cushions in your

shoes so they fit snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

### Cleaning instructions

Note: Never expose the ViscoHeel K to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the ViscoHeel K. If necessary, you can clean your ViscoHeel K heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

### Notes on reuse

This product is intended to be fitted to one patient only.

### Places of use

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

### Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoHeel K have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

### Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist<sup>1</sup>. Before first using our medical device, please seek the advice of a physician or specialist<sup>1</sup> as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist<sup>1</sup> as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist<sup>1</sup>, please contact your physician or distributor, or contact us directly.

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

### Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2024-09

\* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

<sup>1</sup> A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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Bauerfeind Ibérica, S.A.
Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.