Strapin

Vendaje de tobillo
Ankle bandage

EMO
especialista
en ortopedia

Strapin

Vendaje de tobillo

Ankle bandage
Bandage à la cheville
Bandagem de tornozelo
Knöchelbandage
Bendaggio alla caviglia

EMO
especialista
en ortopedia

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REGULATIONS: This medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. A Risk Analysis (UNE EN ISO14971) has been carried out, minimizing all existing risks. Class I medical device, non-sterile, without measurement function.

POSITIONING: Surround the ankle with the elastic band. Finally: adjust the tension and fix the end with velcro on the surface of the tape.

PRECAUTIONS: If an allergic reaction, irritation or swelling occurs or if there is continuous pain, remove the product and consult with the prescribing doctor or health professional. Do not expose to a flame or heat source at very high temperatures.

WARNINGS: The orthosis must be used following the indications or recommendations of the prescriber or health professional. Follow the manufacturer's directions for proper orthosis placement. For a personalized adaptation, consult a specialist. Do not use directly on open or red scars without being properly protected.

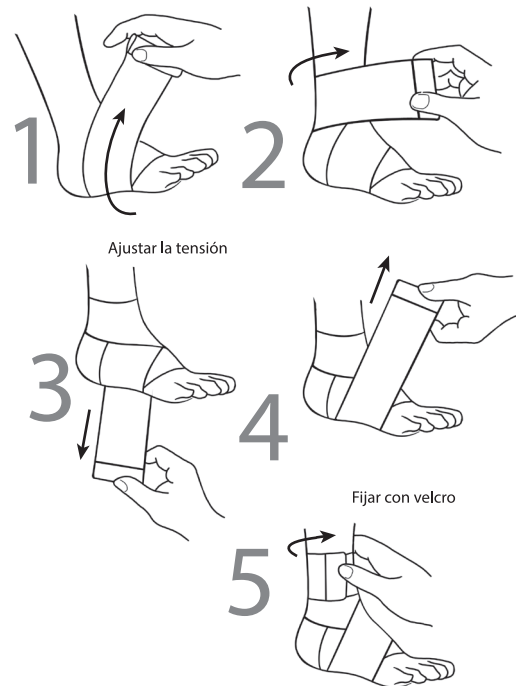
To dispose of the product, use the solid waste container. For the disposal of the container, depending on the material of the container, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country.

Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

WASHING AND MAINTENANCE: Wash with cold water, neutral soap and allow to air dry. Store at room temperature, preferably in the original container.

QUALITY AND GUARANTEE CERTIFICATES: All the materials used have been tested, comply with the applicable European regulations and offer maximum safety and comfort during use. Deko-Text Standard 100 Certificate: ecological textile label that guarantees the absence of substances harmful to health and certifies the absence of harmful substances in all the materials with which the orthosis is made. Guarantee within the framework of legal provisions.

Date of publication of the last revision V1 11/2021

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Única
Largo / Ancho
80 / 7,5

FEATURES

Breathable elastic bandage
Micro-velcro closure
Length 110 cm.
Latex free

EFFECTS

Support and compression bandage

INDICATIONS

Ankle support
Bandage in eight
Plantar fasciitis
Open foot
Support in chronic laxity

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MD Producto sanitario
Medical device

Reciclable
Recyclable

Mantener alejado de la luz
Keep away from light

Punto verde
Green point