U279 | Alza WC con reposabrazos abatibles  
مقعد مرحاض مع مساند للذراعين قابلة للطي  
WC-Aufsatz mit klappbaren Armlehnen  
Raised Toilet Seat with Folding Armrests  
Réhausse WC avec Accoudoirs Rabattables  
Rialzo per WC con poggiaabbraccia pieghevoli  
Alteador WC com repousa braços rebatíveis  
Сиденье для унитаза со складными подлокотниками

(ES)

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(ENG)

**WARNINGS:**

- Read carefully these instructions before the patient uses this daily living aid. For any doubt, consult always the establishment where you have acquired it before using it.
- Keep these instructions for future reference.
- The toilet lift is designed to raise the height of the toilet and facilitate its use when there is a limitation in mobility.
- This product is intended for domestic use only and is not suitable for use in hospitals or nursing homes.
- This aid living day should never be used for any other use different to the intended one. Do not use this product as a staircase, do not stand up on it. Do not use it as furniture. Do not allow children to play with this product. Do not modify the product nor add other elements on it.
- You should check regularly and before its use that the fixing elements are firmly secured. Check carefully the state of the components of the equipment. In the case that one of them appears not to be in good condition of use you should substitute it by an original spare part.
- The height is of 10 cm. Maximum patient weight: 150 kg. Respect always the maximum weight.

**ASSEMBLY AND INSTALLATION:**

1. Take all the components off the packaging. Verify that all the accessories are included.
2. Attach the armrests to the lift using the screws.
3. Adjust the 3 limits of the raised toilet seat to your toilet seat, tighten the screws until it stays stable.

**CARE AND MAINTENANCE:**

- Wash and dry the seat after each use. Remove the dirtiness with a wet cloth and neutral soap, if necessary. Then dry.
- Do not use aggressive or abrasive chemical cleaners. They can damage the seat.
- In case of contamination with infected material, use the usual ways of disinfection.
- At the end of its lifetime, get rid of the seat in a responsible way, preferably by taking it to a recycling centre. In any case, respect the local laws to that effect.

**NOTE:** Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient resides.

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PRIM, S.A
BestCare International LTD.
Lotus NL B.V