ASIDERO AUTOINSTALABLE CON VENTOSA  
SELF-MOUNTED GRAB BAR WITH SUCTION CUP  
POIGNÉE D'APPUI AUTOINSTALLABLE À VENTOUSES  
BARRA DE APOIO AUTOINSTALÁVEL COM VENTOSA

U230 - U231  
(UNI-500/UNI-350)

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# (ENG) Self-mounted grab bar with suction cup

## WARNING:

 **Make sure you read these instructions in full before using the product**

- The self-mounted grab bar is not designed to hold or support the full weight of your body. Do not put all of your body weight onto the grab bar.
- The self-mounted grab bar is designed to assist your natural movements as you enter or exit the shower.
- Every time you go to use the grab bar, always check the indicators in order to make sure the suction cup is firmly attached and is working correctly.
- Do not attach the grab bar to non-fixed surfaces.
- Maximum load supported: 100 lbs (45 kg) when positioned laterally, 150 lbs (68 kg) when positioned vertically.

This product is designed to be used only on smooth, non-porous surfaces (✓): tiles, glass, porcelain or glass fibre. It cannot be used (X) on panels, wood, or surfaces that have been painted or wallpapered.

## MOUNTING

1. Clean the surface carefully using a cotton pad impregnated with alcohol. Then allow the surface to dry. Check the condition of the suction cups. Do not use the product if they are damaged, cracked or broken.
2. Lift the two levers, then position the product so that both suction cups - in their entirety - are resting firmly on a smooth surface. Do not place the suction cups on top of joins, edges or gaps.
3. Hold one end and press down on the lever with the palm of your hand. You should feel resistance when the suction cup adheres to the surface.

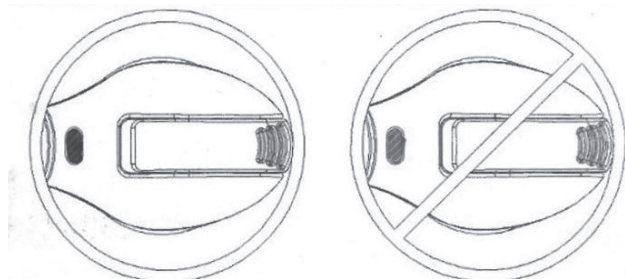
## REMOVAL

1. Lift both levers.
2. If the product remains attached, gently lift the edge of the suction cup using your finger.
3. Do not use any sharp or pointed implements, as they could damage the suction cup.
4. Keep both levers in the raised position until you wish to use the product again.

## Patented adherence indicator.

“GREEN” indicates that the product has been attached correctly, while “RED” indicates that the surface needs to be checked and the product re-attached. Do not place the suction cups on top of joins, edges or gaps.

“GREEN” = OK

“RED” = Re-attach

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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**PRIM, S.A.**
UNITREND CO., LTD.