Tracción Cervical
Zervikale Traktion
Cervicale Traction
Traction Cervicale
Traction Cervicale
Halstractie apparaat
Wyciąg Szyjny
Tracção Cervicale

G30

Instrucciones de adaptación · Anweisungen zur Anpassung · Fitting instructions
Instructions d'adaptation · Istruzioni per l'adattamento · Gebruiksaanwijzingen
Instrukcja obsługi · Instruções de adaptação

ES

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ENG

Indications:

This cervical device has been developed to make distraction of the cervical vertebrae. It is indicated for any process that requires cervical traction and specifically for: Cervical arthrosis. Cervical pains. Cervical contractures

Precautions:

- This traction cervical is a medical device. It should never be used for any other use different to the one prescribed by the doctor. Do not allow children to play with this device.
- Read carefully these instructions before using this product. Before the first use, it is advisable to assemble it without the patient until being familiarized with its assembly. For any doubt, consult always the doctor or the establishment where the product has been acquired before using it.
- Respect always these general instructions of use and the particular indications specified by your doctor, as the load to apply (weight in the water bag), application angle, time of each session, frequency of these sessions and complete duration of the complete treatment. The follow-on of the treatment should be revised by the doctor. If you notice some secondary effect, stop the treatment and inform immediately your doctor
- Before each use, revise carefully the state of all the components of the device. In case some were not under perfect use, consult the establishment where the equipment has been acquired for its inspection. Special attention should be paid to the rope which should not present chafing zones, knots or pleats and that it could move freely and without obstacles.
- Before the treatment, immobilise properly the door (with a wedge or a chock) on which you are going to install the device, and be careful that nobody moves the door while you are under the treatment.
- In order to avoid the risk of jam your fingers, do not manipulate the bracket of the device during its use, particularly avoid the holes and grooves.
- If the treatment is followed by a child or a patient with diminished physical or psychological abilities, it must be supervised permanently by a responsible adult.
- Avoid the use of creams during the treatment in order to avoid the premature deterioration of the head halter.
- You should also inform the doctor if you suffer of skin affection on the face.
- To avoid troubles during the treatment, prevent that the rope receives tugs or the abrupt fall of the water bag.

Assembly instructions:

The kit comes disassembled and it is composed of:

- (A) Sturdy antiskid plastic bracket, adjustable opening for 2 door thicknesses
- (B) Rope with buckle holes on each extremity
- (C) Padded head halter with Velcro closure
- (D) Notched plastic spreader bar
- (E) Graduated water bag with safety stopper
- (F) Support bar for water bag

Rope insertion

- Take the plastic bracket (A) and insert the rope (B) through the squared hole (A1) of the bracket base. It is necessary to compress the rope buttonhole for its insertion.
- Pass the rope (B) through the two grooves of the upper part of the plastic bracket.
- Introduce now the rope buttonhole downwards through the rectangular hole (A2) of the plastic bracket base.

Positioning of the plastic bracket on the door

- The plastic bracket has been designed to be hung over any inner doors. The bracket should lean on the upper edge of the door, the two rubber ends should be in contact with both sides of the door and the bracket should be on horizontal position once assembled.
- In case the door is too thick (+ 30 mm of thickness), the bracket would lean in oblique position upwards and it would be necessary to proceed as follows: remove the blue plastic clip (A3)

from its groove on the bracket base (A), pushing it laterally. Remove the black rubber end from the plastic clip (A3), pushing it laterally. Introduce the black rubber end (A4) in the free groove (A).

Water bag filling

- The graduation of the water bag allows to easily value the weight of the bag. Fill it in with water until the weight prescribed by your doctor. Close the security stopper with a slight pressure

Head halter positioning

- Remove the Velcro closures from the head halter (C).
- Introduce the head in the halter, with the rings (C1) upwards. The larger strap corresponds to the frontal part of the head. This strap should hold the chin and the sewed stitching should cover the chin.
- Finally, adjust and close the Velcro closures. If the assembly is made by the patient himself, we recommend to do it in front of a mirror.

Water bag assembly

- From the two rope buttonholes (B), insert the one coming out the squared hole (A1) of the plastic bracket, which means, the nearest to the door, through the central hole (E1) of the upper edge of the water bag (E).
- Insert the support bar (F) through the upper edge of the water bag (E). The central groove (F1) of the bar should be downwards. Introduce the bar through the bag channel until it is completely in, in order that the bar goes through the rope buttonhole.
- Make sure that the buttonhole is positioned correctly on the central groove (F1) of the bar.

Head halter assembly

- From the two rope buttonholes (B), introduce the one coming out from the rectangular hole (A2) of the bracket, which means, the more far from the door, through the spreader bar (D) until the central groove (D1) of the lower edge of the spreader bar.
- Hook the two metallic rings (C1) of the head halter in the appropriate bar notches (D2), regulating this way the width (C1) as it may be required.
- Caution: Do not loose the water bag suddenly, but gradually and carefully, until the rope (B) is tense.
- Caution: Verify that the rope (B) can move freely avoiding any circumstance that limits the mobility if the components of the device. Now you can achieve the treatment under the conditions and during the time prescribed by your doctor.

End of the treatment

- When time prescribed has been lapsed, hold the water bag (E) while you loose the head halter (C) from the spreader bar (D).
- Disassembly the device following the inverse process to the one described above and keep it until next session, taking care specially of the rope (B) that can be wound but never knotted neither with pleats.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.