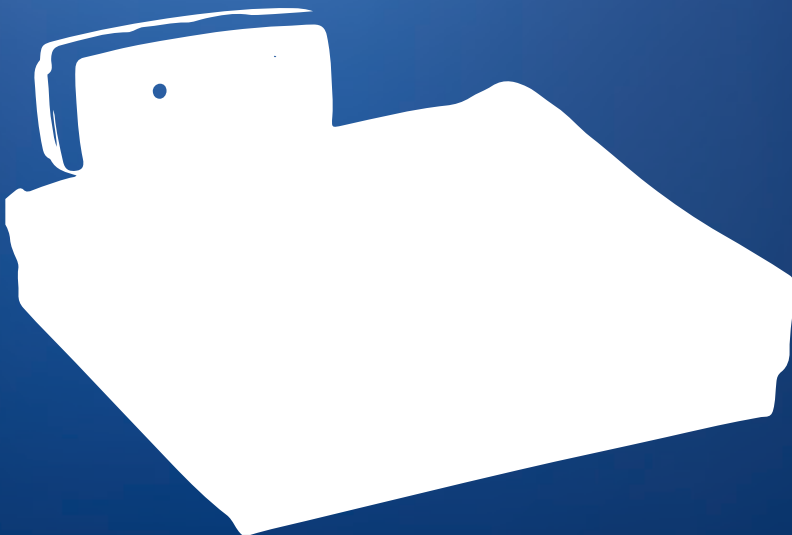
MANUAL DE INSTRUCCIONES

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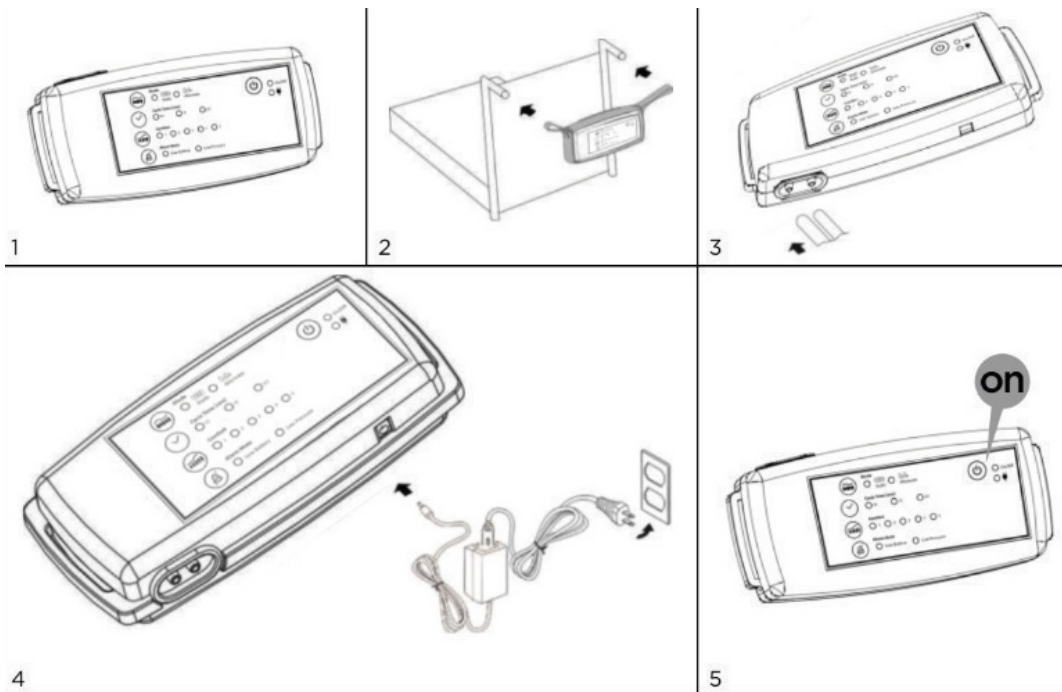
5. INSTALACIÓN

NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. INDICATIONS

The cushion is designed to prevent and heal pressure ulcers. The soft surface provides long-term positioning support and takes the strain off the buttocks and back, making users feel comfortable.

2. TIPS AND PRECAUTIONS

- To avoid explosion, do not over-inflate.
- To avoid any damage to the cushion, keep away from sharp objects.
- Keep away from corrosive objects.
- The equipment is not suitable for flammable anaesthetics and mixtures of air, pure oxygen or nitrous oxide.
- Not suitable for patients with spinal cord injury, unstable fracture, unstable spinal cord injury or long bone traction.
- Never use this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
- Keep the cable away from hot surfaces.
- Never block the air openings of this product or place it on a white surface, such as a bed or sofa, where the air openings may be blocked. Keep air openings free of threads, hair and other similar particles.
- Never drop or insert any object into any opening or sleeve.
- Maximum weight supported: 135 kg.

3. MAINTENANCE AND CLEANING

1. Clean the cushion regularly with neutral detergent or alcohol.
2. Do not use hot water or steam sterilisation.
3. When not in use, store the cushion in a dry place and keep it away from corrosive objects or liquids.

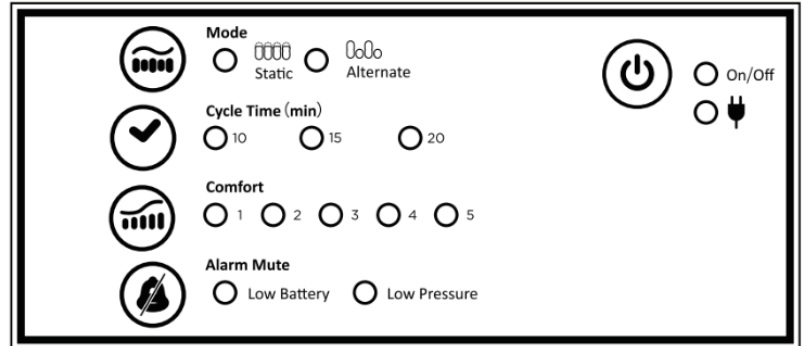
4. CONTROL PANEL

1. MODE OF THERAPY

Alternating mode: Continuously and sequentially inflates and deflates the air cells to remove constant pressure and facilitate enhanced cell perfusion.

Static mode: Provides a stable inflated surface of all cells at low pressure compared to the corresponding comfort level in alternating mode. The system then redistributes the patient's weight over a larger surface area.

2. CYCLE TIMES

To set the alternate pressure cycle time to 10, 15 or 20 minutes.

3. COMFORT SETTINGS

To adjust the comfort level according to the user's preferences or doctor's advice. There are 6 comfort levels, 1 being the softest and 5 the firmest. The default setting is comfort level 1.

4. ALARM SILENCE

Press the alarm silence button to temporarily suspend the alarms. If the problem persists, the alarms will sound again within 10 minutes to notify the caregiver until the problem is resolved.

5. LOW BATTERY CHARGE INDICATOR

A low battery alarm indicates approximately 1 hour of therapy remaining; charge the battery IMMEDIATELY to avoid interrupting therapy. When the battery is charged, the audible visual alarms will be disabled.

6. LOW PRESSURE INDICATOR

When the pressure is below the set pressure, the low pressure indicator will illuminate along with the audio alarm. Check if all connections are correctly made according to the instructions. If the pressure level is continuously low, check if there is any leakage.

7. SWITCHING ON AND OFF

Allows the pump unit to be switched on and off.

8. BATTERY CHARGE INDICATOR


When the indicator light illuminates, AC power is charging the battery. Typical charging time is approximately 4.5 hours. The LED light will turn off when the battery is fully charged.

 **NOTICE**

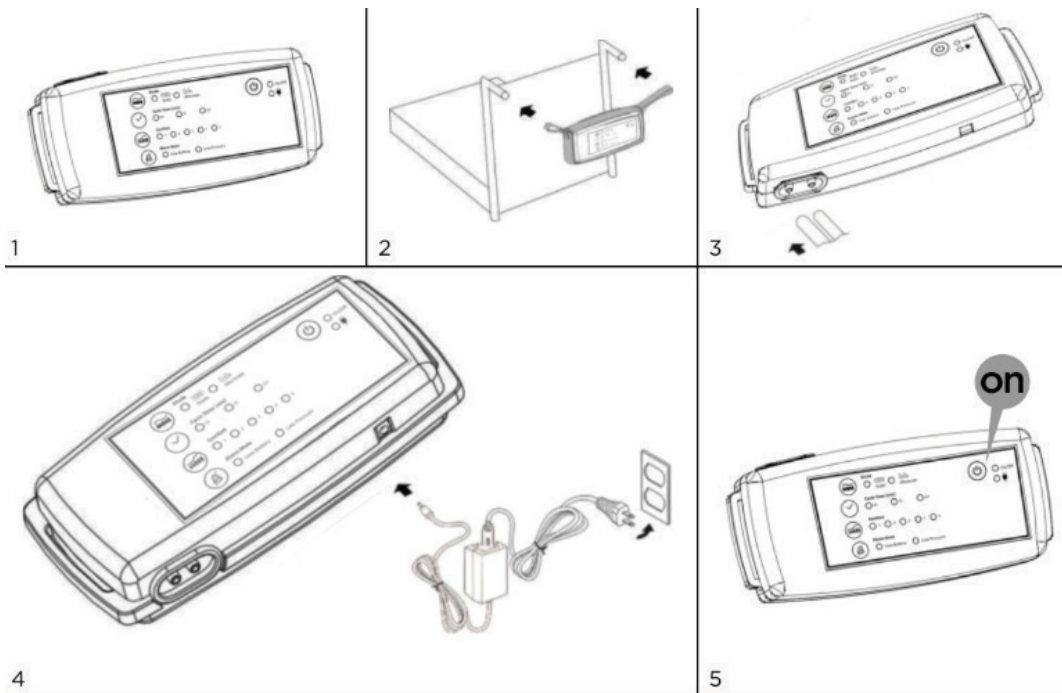If the charging time is more than 5 hours, but the battery charge indicator is still lit, DO NOT continue charging the battery and disconnect the adapter for safety..

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GUANGDONG YUEHUA MEDICAL INSTRUMENT FACTORY CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.