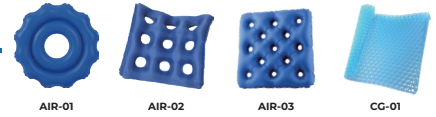
ES

Aviso:

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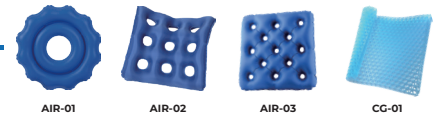
EN

Notice:

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Indications:

The cushion is designed to prevent and heal pressure sore. The soft surface provides longterm positioning support and take the strain off buttocks and back, makes users feel comfortable.

Contraindications:

To prevent from explosion, do not over-inflate. To prevent any damage on cushion, keep away from all sharps. Keep away from any corrosive objects. The equipment is not suitable for flammable anesthetics and air, pure oxygen or nitrous oxide mixtures. Does not apply to patients with spinal cord injury. Not suitable for unstable fracture, unstable spinal cord injury, long bone traction patients.

Maintenance:

-Wipe the cushion using a wet cloth; rinse in clear water then air dry. Do not use detergents or other solvents.
-Do not machine wash or tumble dry.

-Do not heat or steam autoclave.
-When not using the cushion, store it in cool and dry place. Keep away from the corrosive object.

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CO., LTD
SUNGO Europe B.V
Imported by/Importado por: Grupo R. Queraltó S.A.