MANUAL DE INSTRUCCIONES

Silla con inodoro MANZANARES

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. INDICATIONS

- It is designed for the elderly, patients and disabled as a commode chair.
- Weight capacity 100 kg.
- Check and ensure that all parts are tight and securely locked before using the wheelchair.
- Make sure that the wheels are locked before sitting in the chair.

2. TIPS AND PRECAUTIONS

- Must be used indoors on a flat surface. Do not drive it on a bumpy road.
- Avoid hitting the chair against obstacles.
- Do not drive the wheelchair in the raised area under the force of inertia. This could damage the wheelchair or cause an accident.
- Do not go up or down steps and do not climb stairs without the help of an assistant. Do not lift the wheelchair with the user seated in it, otherwise you will damage the wheelchair and cause injury to the user.
- Unless specified, do not adjust, assemble or disassemble the chair with the user seated.
- Do not lean your body out of the chair to prevent it from tipping over.
- Do not change the seating position when moving backwards, otherwise the chair may tip over.
- Do not drive the wheelchair on escalators.
- Do not step on the footrest when getting in and out of the wheelchair. Do not stand on the footrest. Otherwise it could tip over.
- Do not tilt the wheelchair or turn the wheelchair suddenly. Do not brake the wheelchair abruptly.

3. CLEANING AND MAINTENANCE

Use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth. Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage your product.

Chair for domestic use, please do not use it in a humid environment or leave it outside, if you want to extend its life of use.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD
EC REP SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.