MANUAL DE INSTRUCCIONES

Silla con inodoro GUADALQUIVIR

| | |
|-------------------------|-----|
| Manual de instrucciones | ES |
| Instruction manual | EN |
| Manuel D'Utilisation | FR |
| Manuale D'Istruzioni | IT |
| Anweisungen | DE |
| Instruções | PT |
| Gebruiksaanwijzing | NL |
| Bruksanvisning | SWE |
| Instrukcja obsługi | PL |
| Brugsanvisning | DK |

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

PARTS

- | | | |
|-----------|---------------|-------------------|
| 1. Bucket | 2. Tools | 3. Main structure |
| 4. Seat | 5. Seat cover | |

ASSEMBLY

1. Open the box and remove all components from the plastic bag.
2. Fitting the trough: unfold the main frame and place the trough on top or at the front of the frame.
3. Adjust the seat: position the seat with the smaller side facing the front and press the clamp to engage the seat to the frame.
4. Adjust the seat cover: place the seat cover on top of the seat with the smaller part also facing the front. Adjust the position and make sure that the two clips on the seat cover are parallel to the clips on the seat. Press down to lock it into the frame.

INDICATIONS

- It is designed for the elderly, patients and the disabled, as a commode chair.
- Weight capacity 100 kg.
- Check and ensure that all parts are tight and securely locked before using the wheelchair.
- Make sure the wheels are locked before sitting in the chair.

TIPS AND PRECAUTIONS

- Must be used indoors on a flat surface. Do not drive it on a bumpy road. Avoid hitting the wheelchair against obstacles.
- Do not drive the wheelchair on elevated areas with the force of inertia. This could damage the wheelchair or cause an accident.
- Do not go up or down steps and/or stairs without the help of an assistant. Do not lift the wheelchair with the user seated in it, otherwise you will damage the wheelchair and cause injury to the user.
- Unless specified, do not adjust, assemble or disassemble the chair with the user seated. Only adjust, assemble or disassemble the chair when the chair is unoccupied.
- Do not lean your body out of the chair to prevent it from tipping over.
- Do not change the seating position when moving backwards, otherwise the chair may tip over.
- Do not drive the wheelchair on escalators.
- Do not step on the footrest when getting in and out of the wheelchair. Do not stand on the footrest. Otherwise it could tip over.
- Do not tilt the wheelchair or turn the wheelchair suddenly. Do not brake the wheelchair abruptly.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD
EC REP **SUNGO EUROPE B.V.**
GRUPO R. QUERALTÓ S.A.