# MANUAL DE INSTRUCCIONES

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Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

## 1. COMPONENTS

1. Cube
2. Main structure
3. Seat
4. Seat cover
5. Tools

## 2. ASSEMBLY

1. Open the box and remove all components from the plastic bag.
2. To place the cuvette, unfold the main frame and place the cuvette on the back of the frame.
3. To fit the seat, position it with the smaller side facing the front and press the clamp to fit the seat into the frame.
4. To adjust the seat cover, place it on top of the seat with the smaller side also towards the front. Adjust the position and make sure that the two clips on the seat cover are parallel to the clips on the seat. Press down to lock it into the frame.

## 3. CLEANING AND MAINTENANCE

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth.

Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage the product.

## 4. INDICATIONS

It is used by elderly people and disabled people with mobility difficulties who cannot go to the toilet freely.

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FOSHAN KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD.
**EC REP** SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.