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ENGLISH

ASSEMBLY

1. Open the box, take out all parts from the polybag.
2. Fix the pot: unfold the main frame and put the pot either from top or from the front of the main frame.
3. Fixing the seat: place the seat with the small side facing the front and press down the clamp to fix the seat on the frame.
4. Fixing the seat cover: Place the seat cover on the top of the seat with the small side also facing the front. Adjust the position to make sure that the two clamps of the seat cover are parallel to the seat clamp. Press it down to fix it to the frame.

NOTICE

Notice to the user and/or patient: Any serious incident that has occurred in relation to the device must be informed to the competent authority of the Member State in which the user and/or patient is established.

INDICATIONS

- It is designed for elderly, patients and disabled, as a commode chair.
- Weight capacity 100 kg.
- Check and make sure all parts are tight and locked before using the chair.
- Make sure the wheels are locked before sitting in the chair.

TIPS AND PRECAUTIONS

- Must be used indoors on a flat surface. Do not drive on a bumpy road. Avoid hitting the wheelchair against obstacles.
- Do not drive the wheelchair in an elevated area with the force of inertia. This could damage the wheelchair or cause an accident.
- Do not climb up or down steps and do not climb stairs without the help of an assistant. Do not lift the chair with the user seated in it, otherwise it will damage the chair and cause injury to the user.
- Unless specified, do not adjust, assemble or disassemble the chair with the user seated. Only adjust, assemble or disassemble the chair when the chair is not occupied.
- Do not lean your body out of the chair to prevent the chair from tipping over.
- Do not change the seated position when moving backwards, otherwise it may cause the chair to tip over.
- Do not drive the wheelchair on escalators.
- Do not step on the footrest when getting in and out of the wheelchair. Do not stand on the footrest. Otherwise, it may tip over.
- Do not tilt the wheelchair or turn the wheelchair suddenly. Do not brake the wheelchair suddenly.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO.,LTD.
SUNGO EUROPE B.V
GRUPO R. QUERALTÓ S.A