Other languages

  
MOBICLINIC

## AMPLIFICADOR DE SONIDO EARPLUS2

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
## 1. WARNINGS

- 1. WARNING!** This sound amplifier is recommended for persons aged 18 years and over.
- 2. WARNING!** The sound amplifier should not cause pain at the time of insertion.
- 3. WARNING!** If you experience itching, redness, pain, swelling, inflammation or a burning sensation in or around the ears, stop using the product.
- 4. WARNING!** The earplug of the ear amplifier should not be uncomfortable or painful.
- 5. WARNING!** Remove the sound amplifier for MRI or other electromagnetic procedures.
- 6. WARNING!** If any part of the sound amplifier, such as the earplug, gets stuck in your ear and you cannot easily remove it with your fingers, seek medical help as soon as possible. Do not try to use tweezers or cotton buds, as they may push the part into the ear.
- 7. WARNING!** Do not share your sound amplifier with others.
- 8. WARNING!** People who start using sound amplifier sometimes need a few weeks to get used to them.
- 9. WARNING!** The sound amplifier, its parts and batteries are not a toy. Keep them out of the reach of children.
- 10. WARNING!** You may notice an accelerated build-up of earwax when using the sound amplifier. This is normal.

**11. WARNING!** Caution with high volume because of its auditory impact such as hearing loss with prolonged exposure.

## 2. SWITCH ON/OFF

To turn on the sound amplifier, the switch must be located where "ON" is indicated. The switch should be placed in the "OFF" position when the unit is not in use. This will ensure a longer battery life.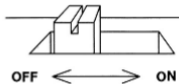

## 3. VOLUME

The sound amplifier has 6 levels of volume control. It is recommended to turn the device on at level 1 and from there adjust the volume according to the user's needs.

- Level 1 has no sound.
- Level 2 maximum gain  $30\pm 5$
- Level 3 maximum gain  $33\pm 5$
- Level 4 maximum gain  $36\pm 5$
- Level 5 maximum gain  $38\pm 5$
- Level 6 maximum gain  $40\pm 5$

## 4. BATTERY

Open the battery cover and insert the battery. Use a 1.5 V button cell battery.


## 5. CAUTION

This product is not waterproof, so care must be taken by the user to prevent water from entering the sound amplifier.

It is important to increase the volume control very gradually to avoid a sudden increase in sound.

## 6. INSTRUCTIONS FOR USE


1. Check that the battery is in place.
2. Set the volume to level 1.
3. Place the sound amplifier in your ear, making sure that the earpiece is just inside your ear before turning it on.
4. Turn on the switch at the end of the amplifier until you hear an airflow noise.
5. Gradually increase the volume to an appropriate level.

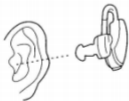
a. Correct form in terms of placement.

b. The amplifier must be carefully inserted into the ear to avoid possible damage.

## 7. PARTS OF THE SOUND AMPLIFIER
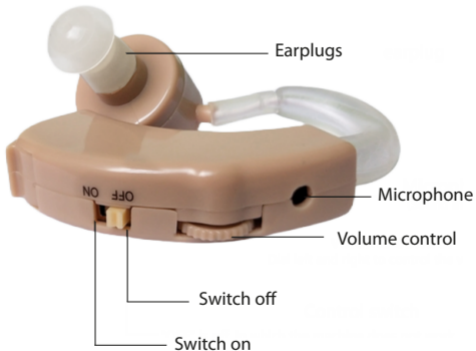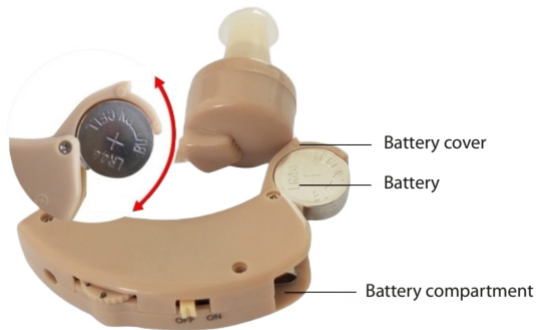
## 8. POSSIBLE SOLUTIONS

PROBLEM	CAUSE	SOLUTION
Sound amplifier does not work	Speaker/headset blocked	Clean the speaker/ear-piece opening
	Sound amplifier off	Switching on the sound amplifier
	The battery compartment is not properly closed	Close the battery compartment completely
	Battery depleted	Replacing the battery
	The battery is in the wrong place	Make sure the battery is correctly inserted
	Start-up delay	Wait a few seconds and check again

Sound amplifier too loud	Volume too high	Decrease the volume
Sound amplifier is not loud enough or is distorted	Volume too low	Increasing the volume
	Low battery	Replacing the battery
	Speaker/headset blocked	Clean the speaker/ear-piece opening
	Your hearing health has changed	Contact your Hearing Care Professional
No sound from the sound amplifier	The dome is blocked with wax	Clean/replace the dome
	Battery inserted incorrectly	Insert the battery correctly with the (+) side up
Sound amplifier sounds low / doesn't sound clear	Low battery	Replace the battery with a new one

Sound amplifier sounds low / doesn't sound clear	The dome is blocked with wax.	Clean/replace dome
	Volume control is not loud enough	Increase the volume of your sound amplifier to a comfortable level
Whistling	Sound amplifier not inserted correctly in the ear / Cerumen in the ear canal	Correctly fitting the sound amplifier
	Dome not properly inserted in the ear canal	Remove the sound amplifier and reinsert the dome into the ear canal
	Dome too small for your ear canal	Select a larger dome size to ensure that your dome seals completely
	Hair or clothing covering the microphone	Move hair or clothing away from the sound amplifier microphone

Whistling	Sound is too loud	Turn down the volume
	The dome is blocked with wax	Clean or replace the earpiece
Sound amplifier keeps falling out	Dome too small for the ear canal	Replace your dome with a larger dome

CE Made in P.R.C

GUANGDONG ENNO MEDICAL TECHNOLOGY CO., LTD.  
No. 8 Huaan Road, Zhongkai High-tech District, Huizhou City,  
Guangdong, China 516006  
[service@ennome.net](mailto:service@ennome.net)

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GRUPO R. QUERALTÓ S.A.