# MANUAL DE INSTRUCCIONES

## Andador INVICTO X1

Manual de instrucciones **ES**

Instruction manual **EN**

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Manuale D'Istruzioni **IT**

Anweisungen **DE**

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Bruksanvisning **SWE**

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

## 1. INDICATIONS

Movement aid for people with disabilities. Intended for rehabilitation after injuries. Helps to facilitate the movement of the elderly. Disorders of the musculoskeletal system. Active extension of the motor regime after long-term illnesses, injuries and the postoperative period.

## 2. MAIN PARTS OF THE PRODUCT

1. Handlebar height adjustment knob
2. Red retaining clip
3. Adjustment knob
4. Handle
5. Brake
6. Seat
7. Shopping bag
8. Front wheel
9. Rear wheel

## 3. BASIC PRODUCT PARAMETERS

1. Total length: 71.5cm
2. Total width: 60cm
3. Overall height: 79-87cm
4. Seat width: 45cm
5. Seat height: 52cm
6. Front wheel: 10
7. Rear wheel: 8
8. Maximum weight capacity: 100kgs

## 4. FOLDING THE WALKER

1. Lift the strap upwards to fold it.
2. Then lift the red retaining clip to lock and fold the product.
3. Release the red retaining clip to unfold the product

## 5. ASSEMBLING THE SHOPPING BAG

Insert the transport bag into the product frame.

## 6. HANDLE HEIGHT ADJUSTMENT

Press and hold the red release buttons. Simultaneously move the handle up or down. When the desired height is reached, release the red release buttons and the handle will automatically lock in the next possible height position.

## 7. BRAKING

To brake while in motion, pull both brake levers up as shown and always brake slowly and steadily.

Press both brake levers down to apply and lock the brakes while the rollator is stationary.

## 8. SAFETY AND MAINTENANCE

1. Before use, check the tightness and stability of the bolts and frame; users may not exceed the maximum weight of the product (100kg).
2. The effect of the brakes may be affected by abrasion of the wheels.
3. Check the brakes regularly.
4. Do not use strong corrosive cleaners to clean the surface of the product.
5. The product must not be left outside in rainy conditions.
6. Before sitting on the seat, lock both brakes to prevent the rollator from sliding.
7. Do not hang any heavy objects on any of the armrests, as this will cause the rollator to become unstable.

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**JIANLIAN HOMECARE PRODUCTS CO., LTD**
**LOTUS NL B.V.**
**GRUPO R. QUERALTÓ S.A.**