Other languages

MANUAL DE INSTRUCCIONES

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EARPLUS1

- Manual de instrucciones **ES**
- Instruction manual **EN**
- Manuel D'Utilisation **FR**
- Manuale D'Istruzioni **IT**
- Anweisungen **DE**
- Instruções **PT**
- Gebruiksaanwijzing **NL**
- Bruksanvisning **SWE**
- Instrukcja obsługi **PL**
- Brugsanvisning **DK**

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1. WARNINGS

- 1. WARNING!** This sound amplifier is recommended for persons aged 18 years and over.
- 2. WARNING!** The sound amplifier should not cause pain at the time of insertion.
- 3. WARNING!** If you experience itching, redness, pain, swelling, inflammation or a burning sensation in or around the ears, stop using the product.
- 4. WARNING!** The earplug of the ear amplifier should not be uncomfortable or painful.
- 5. WARNING!** Remove the sound amplifier for MRI or other electromagnetic procedures.
- 6. WARNING!** If any part of the sound amplifier, such as the earplug, gets stuck in your ear and you cannot easily remove it with your fingers, seek medical help as soon as possible. Do not try to use tweezers or cotton buds, as they may push the part into the ear.
- 7. WARNING!** Do not share your sound amplifier with others.
- 8. WARNING!** People who start using sound amplifier sometimes need a few weeks to get used to them.
- 9. WARNING!** The sound amplifier, its parts and batteries are not a toy. Keep them out of the reach of children.
- 10. WARNING!** You may notice an accelerated build-up of earwax when using the sound amplifier. This is normal.
- 11. WARNING!** Caution with high volume because of its auditory impact such as hearing loss with prolonged exposure.

2. READ BEFORE USE

1. Application: The sound amplifier is intended to amplify and transmit sound to the ear and thus compensate for mild to severe hearing impairments, this sound amplifier is intended for use by adults over 18 years of age.
2. Before using the hearing instrument for the first time, it is recommended to charge it for 4 hours.
3. Allow 4 to 8 weeks for your brain to adapt to the sounds with your new hearing amplifier.

3. INCLUDED PARTS

Sound amplifier
(Plug size M)

Protective cover

USB cable

Adapters ear
(L/S)


Cleaning brush

Tube

4. PARTS OF THE SOUND AMPLIFIER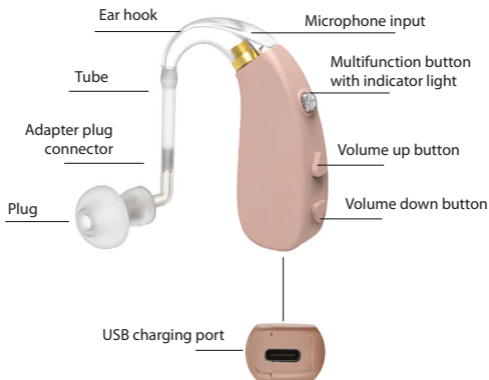

- SWITCHING THE HEARING AMPLIFIER ON/OFF

Press and hold the multi-function button on each sound amplifier for 2 seconds until the indicator light flashes.

- ON: The green indicator light flashes 3 times.
- OFF: The blue indicator light flashes 3 times.


- MODE SETTINGS

Briefly press the multifunction button to switch modes.

5. HOW TO CHARGE THE SOUND AMPLIFIER

1. Connect the larger end of the USB cable to the power supply.
2. Plug the smaller end into the USB charging port on the hearing amplifier.
3. Plug the power supply into a power outlet.

4. The indicator light illuminates blue when the charger is connected to the mains. When fully charged, the light will be green and will last for 20-24 hours of use.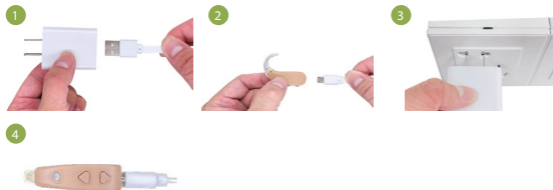

6. USING THE AMPLIFIER

- CHOOSING THE STOPPER

The size M earplug comes pre-installed in the hearing amplifier, if you feel that the medium size (M) is too loose, replace it with the large size (L), it will fit better in your ears.


- CUT THE TUBE TO YOUR EAR SIZE

Step 1: Remove the tube from the ear hook.

Step 2: Insert the earplug into the ear, and place the device over the ear, mark the point where the tube and hook meet. The tube should be cut with scissors.

Step 3: Push the tube over the grooves of the hook until it is securely fastened.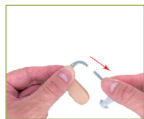


- REPLACEMENT OF THE SOUND AMPLIFIER TUBE

Step 1: Remove the tubing from the ear hook and adapter.

Step 2: Take the old pipe and align it with the cap and the new pipe, trim the new pipe to the same length as the old one.

Step 3: Push the tube over the grooves of the hook and adapter until it is securely fastened.

Tip: We recommend replacing the tube every 3-6 months.

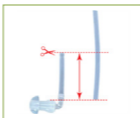- PUT THE SOUND AMPLIFIER IN THE EAR

Step 1: Adjust the direction of the earplug to fit either ear.

Step 2: Place the hearing instrument over the top of your ear.

Step 3: Carefully insert the earplug into the ear. The earplug should fit snugly and comfortably.

- ADJUSTING THE VOLUME

Press the up/down switch to increase/decrease the volume. You should hear beeps after each press of the switch. When you hear two beeps, it means you have reached the lowest or softest volume.

- CHANGE HEARING PROGRAMME

Press the multi-function button on the amplifier to change modes. Your hearing amplifier will beep to tell you which programme you are in.

Programme configuration	Beeps	Environment
P1: Normal mode	Beep	At home or in the office
P2: Medium noise reduction mode	Beep Beep	On the road, in the super-market or in the restaurant
P3: Noise reduction mode	Beep Beep Beep	In the open air, chatting with the people next to you

When the amplifier is switched off and on again, it is always in the last configuration you have made.

- HEARING LEVELS

Level 1 maximum gain 28 ± 5 .

Level 2 maximum gain 31 ± 5 .

Level 3 maximum gain 34 ± 5 .

Level 4 maximum gain 37 ± 5 .

Level 5 maximum gain 40 ± 5 .

7. MAINTENANCE

Keep your sound amplifier clean. Heat, moisture and foreign substances can cause poor performance.

1. Wipe daily with a tissue.
2. Clean regularly with a cleaning brush to remove debris. Use cleaning brushes to remove debris that clogs the microphone.
3. Avoid physical shocks such as dropping the hearing amplifier on a hard surface.
4. Avoid exposure to moisture or heat and store in a cool, dry place.
5. Never use water, solvents, cleaning fluids or oil to clean the hearing instrument.
6. The charging port should always be kept clean and dry.

7. Do not disassemble the hearing amplifier or insert cleaning tools inside the hearing amplifier.
8. Store the hearing amplifier in the protective case when not in use.
9. Away from direct sunlight or heat to avoid extreme temperatures.

8. PROBLEM SOLVING

PROBLEM	CAUSE	POSSIBLE SOLUTION
Amplifier does not work	Speaker / earphone blocked	Clean the speaker / earphone opening
	Amplifier off	Turn on the amplifier
	Battery depleted	Recharge the amplifier
	Start-up delay	Wait a few seconds and check again
Amplifier too loud	Volume too high	Decrease the volume
Amplifier is not loud enough or is distorted	Volume too low	Increasing the volume
	Low battery	Recharge the amplifier
	Speaker / earphone blocked	Clean the speaker / earphone opening
	Your hearing health has changed	Contact your Hearing Care Professional
Intermittent sound or no sound from the amplifier	Thin tube/dome is clogged with wax	Clean / replace thin tube or dome
	Moisture in the thin tube	Remove moisture from the thin tube

Amplifier sounds low / Amplifier sounds unclear	Low battery	Recharge the amplifier
	Thin tube / dome is clogged with wax	Clean/replace the thin tube or dome
	Volume control is not loud enough	Increase the volume of your amplifier to a comfortable level
Unclear and distorted performance	Reboot required	Place it in the charger until charging begins, then remove it. This will turn your amplifier off and on
Whistling	Amplifier is not inserted correctly in the ear. Cerumen in the ear canal	Correct positioning of the amplifier
	Dome not properly inserted in the ear canal	Remove the amplifier and reinsert the dome into the ear canal
	Dome too small for your ear canal	Select a larger dome size to ensure that the dome completely seals the ear canal
	Cracks in the thin tube	Replace the thin tube
	Hair or clothing covering the microphone of the amplifier	Move hair or clothing away from the amplifier microphone

Amplifier beeps	Sound is too loud	Turn down the volume
	The dome is blocked with wax	Clean or replace the earpiece
Amplifiers whistle in the charger	Incorrect orientation	Place the amplifiers back into the port with the handset resting in the charger reservoir. Check that the charging LED starts to glow
	Battery depleted	The battery in the charger has run out. Plug in the charger. If you do not have the cable with you, press and hold the amplifier's user control for 7 seconds to turn it off. This will conserve your amplifier's charge
Amplifier does not turn on	Battery fully discharged	Charging the amplifier
	The button has been inadvertently deactivated by pressing it for more than 7 seconds	Replace the amplifiers in the charger case and restart them
The amplifier indicator light does not turn on when the amplifier is placed in the charging case or on the USB cable	The amplifier is not inserted correctly into the charger	Correctly insert the amplifier into the charger
	Amplifier battery fully discharged	Wait 2-3 hours after placing the amplifier in the charger, irrespective of the behaviour of the LED

Indicator light of the amplifier switches off when removing the amplifier from the charger	Auto ON" function is deactivated	Switch on the amplifier
No LED indicator when amplifiers are inserted into the charging port	Battery depleted	Connect the micro USB to your charger and plug it into the power supply. The LEDs on the inserted battery will light up for a few seconds to indicate connection to the power supply. If not, please contact your hearing care professional
	Incorrect orientation	Place the amplifiers back into the port with the handset resting in the charger reservoir. The amplifier will charge in the correct port
Charging does not start (but the indicator light around the USB port of the charger is connected to the socket)	The amplifier is not correctly inserted into the charger	Correctly insert the amplifier into the charger
The amplifier keeps dropping out	Dome too small for the ear canal	Replace your dome with a larger dome. Remember to follow the instructions for selecting the correct dome size and thin tube length

9. TECHNICAL PARAMETERS

MODEL NUMBER	EARPLUS1
Maximum sound output	$\leq 130\text{dB} + 3\text{dB}$
Average sound power	$120\text{dB} \pm 4\text{dB}$
Average acoustic gain	$42\text{dB} \pm 5\text{dB}$
Equivalent input noise	$\leq 29\text{dB} + 3\text{dB}$
Frequency range	$F1 \leq 300\text{HZ}, F2 \geq 4500\text{Hz}$
Total harmonic wave distortion	$\leq 5\%$
Battery	Lithium
Working current	$\leq 5\text{mA}$
Net weight	4g

- WORKING CONDITIONS

1. Ambient temperature: $-10^{\circ}\text{C} \sim 40^{\circ}\text{C}$.
2. Relative humidity: $\leq 80\%$.
3. Atmospheric pressure: $86\text{kPa} \sim 106\text{kPa}$.
4. Operating voltage: DC 3.7V
5. Device safety level: Type B Part applied

CE Made in P.R.C

GUANGDONG ENNO MEDICAL TECHNOLOGY CO., LTD.
No. 8 Huan Road, Zhongkai High-tech District, Huizhou City,
Guangdong, China 516006
service@ennome.net

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GRUPO R. QUERALTÓ S.A.