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**IMPORTANT**

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the M12-M24 MARSELLA wheelchair! In this instruction manual you will find all the information you need to assemble, use and maintain the wheelchair safely. Please read all instructions carefully before using the product.

Follow the recommendations in this manual, as they are designed to improve safety and performance.

NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. PRODUCT FEATURES

| | |
|---------------------------------|--|
| PRODUCT NAME | Wheelchair |
| MODEL | M12 - M24 MARSELLA |
| MODEL REFERENCE | M12: (A-00449/46-PL) - (A-00449/46-AZ) M24: (A-00449/46-PL) - (A-00449/46-AZ) |
| MAXIMUM SUPPORTED WEIGHT | M12/M24: 242,50 lbs |
| NET PRODUCT WEIGHT | M12/M24: 39,24 lbs |
| PRODUCT DIMENSIONS | M12/M24: 32,28"*10,62"*37,00" |

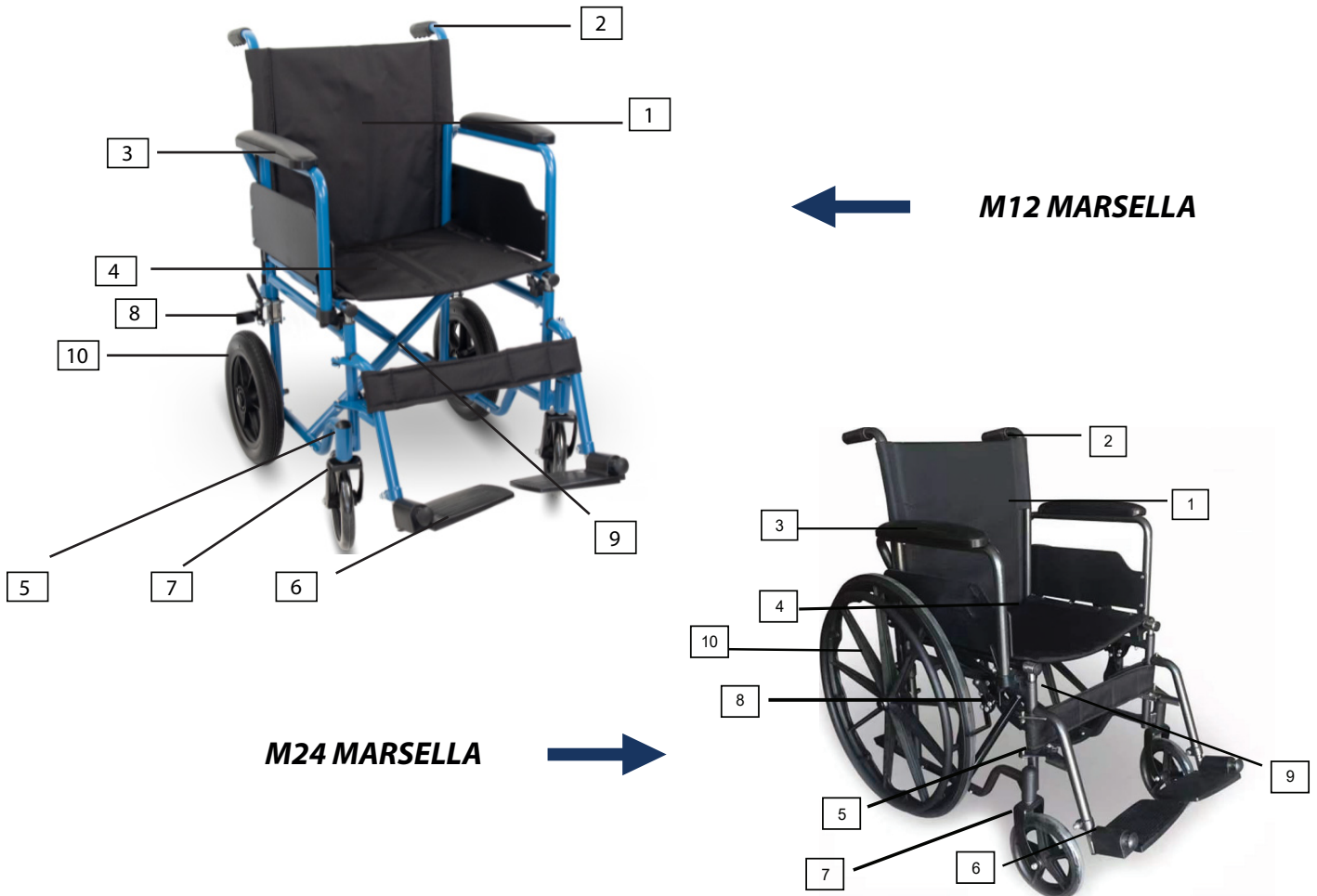
SPECIFICATIONS

| | |
|--------------------------|---|
| STRUCTURE | Single cross bar, powder coated |
| FRONT WHEEL | 8 "x1", PVC Tyres |
| REAR WHEEL | 24" alloy wheel (M12-M24), polyurethane tyres |
| ARMREST | Folding |
| FOOTREST | Swivelling and removable, plastic plates |
| SEAT AND BACKREST | Nylon with foam |
| FRONT FORK | Steel |

2. PARTS AND COMPONENTS

| NUMBER | NAME |
|--------|--|
| 1 | Backrest: you can fold the backrest by pressing on the two black plastic levers, then pulling backwards and pushing down on the top of the backrest. To return it to its original position, repeat the operation in reverse, making sure that the mechanism is securely locked |
| 2 | Handles |
| 3 | Armrests |
| 4 | Seat |
| 5 | Footrest support |
| 6 | Footrest: you can adjust its height by loosening the nut at the base of the adjustment tube, positioning the plates at the desired height and tightening the nut to secure them |
| 7 | Wheels and bearings: The wheels are fitted with puncture-proof tyres. The swivel fork allows the wheelchair to be steered. An adjustment locknut is located under the lid of the fork. If the lock nut is too tight, the wheelchair will be difficult to steer |

| | |
|----|--|
| 8 | Brakes: should not be used when the wheelchair is in motion or to reduce speed. They should always be applied at the same time. To do this, pull the lever upwards. To remove them, push the lever downwards |
| 9 | Cross bar |
| 10 | Quick release rear wheel |

3. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

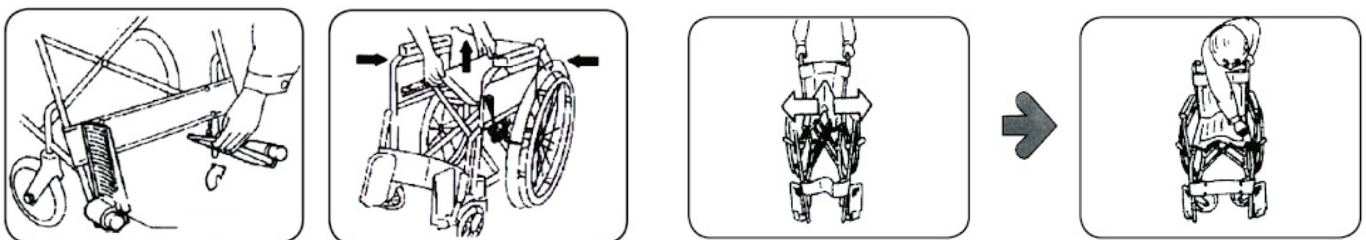
1. The wheelchair is used as a mobility tool for the disabled, patients and the elderly who have difficulties of action.
2. It is recommended that they are always accompanied at the beginning by co-workers in order to prevent possible accidents due to lack of experience in handling the equipment.

CONTRAINDICATIONS

1. Do not assemble or use the equipment without first reading and understanding this instruction manual. If you do not understand the instructions, consult a professional before mounting the equipment.
2. Before using the product, ensure that the parts are secure, the wheels are in good condition, the wheelchair is balanced and all wheels are in contact with the ground at all times during use.
3. Be aware of the risks when reaching for objects, leaning backwards or sideways, the use of hand brakes, the wheels and the weight limit.
4. The footrests are designed to support the feet while seated. Do not stand on them.
5. Do not use the wheelchair in moving vehicles.

4. ASSEMBLY INSTRUCTIONS

1. To unfold the chair, press down on the seat rails until the chair is fully unfolded. Make sure the seat rails are securely locked into the frame brackets before sitting down.
2. To fold the chair, pull the front and rear side of the seat fabric upwards.

M12 MARSELLA:**M24 MARSELLA:**

3. Footrest support:

- Place it on its side so that the footrest is perpendicular to the chair.
- Install the hinge plates onto the bolts on the chair frame.
- Push the brackets into the chair until they lock into place. The footrest will be facing into the chair when it is engaged.
- Repeat the process for the other bracket.
- To remove them, push the lever inwards and swing the footrest outwards.

4. Rear wheels:

- Grasp the rear wheel and press the quick release button.
- While pressing the button, push the axle against the chair until you hear a click.
- Release the button.
- Check that the wheel is securely fixed.
- Follow the reverse process to remove the wheels.

5. USAGE TIPS

Several activities require the user to reach, lean or transfer from the chair to another seat or vice versa. These movements can cause a change in the balance, centre of gravity and weight distribution of the wheelchair. The design of the wheelchair will allow for stability in performing daily activities when used correctly according to the recommendations:

1. Reaching / Leaning forward: Move the wheelchair as close as possible to the object you want to reach. Apply the brakes to lock the rear wheels. Only attempt to grasp objects that are within reach while in a normal seated position. Only extend your arms out of the seat of the wheelchair.

2. Reaching / Leaning backwards: Move the chair as close as possible to the object you want to reach. Only reach as far as your arm will reach without changing the normal seating position. Do not lean over the top of the seat back as this may cause the chair to tip over.

3. Reaching / leaning sideways: Move the wheelchair as close as possible to the object you want to reach. Turn the front wheels so that they point in the direction you are going to lean. Apply the brakes on the rear wheels. Do not carry your weight on the side you are leaning or you may tip your wheelchair over.

4. Transferring from one seat to another: First, make sure that the space between the two seats is as small as possible. Confirm that the brakes are applied. Turn the front wheels in the direction you want to move. Lift the footrests and fold the supports sideways. It is important to have professional supervision until the user is familiar with the process.

5. Overcoming obstacles:

- **Climbing a kerb from the front:** the front passenger should lean the wheelchair back until the front wheels are over the kerb. Move the wheelchair forward until the rear wheels touch the kerb and carefully lower the front wheels. Lift and push until the rear wheels are over the kerb. The wheelchair should not be lifted by the handlebars alone; use your foot to press down on the rear of the frame.
- **Back up a kerb:** the wheelchair should be positioned so that the rear wheels are close to the kerb. The attendant should lean the wheelchair back and pull the wheelchair until the rear wheels go up the kerb. The front wheels should only be lowered when the wheelchair is far enough away from the kerb.
- **Going up and down hills:** always go straight up and down hills to avoid the risk of rolling over.

6. MAINTENANCE AND CLEANING

1. Upholstery: Check the upholstery periodically for tears or looseness. Replace if damaged. It should be cleaned regularly with a mixture of neutral soap and water. Do not use solvent on the upholstery.
2. Metal parts: clean with wax products, such as car wax, to preserve the finish. Do not use abrasive cleaners that may damage the varnish.
3. Cross bar: Inspect periodically and lubricate the centre bolt with light machine oil.
4. Wheels and tyres: Check the wheels for wear or cracks and replace them if necessary. It is recommended that both front wheels are replaced at the same time. For tyres, check that the tyre pressure is correct and both tyres are even.
5. Bearings: The front wheels are attached to the swivel fork by means of a bolt and axle nut. They should be checked and tightened periodically, adjusting them so that they are secure but free to move.
6. Brakes: check frequently that they are working properly and that there is no wear.
7. Check that the wheelchair folds and unfolds properly and that the wheelchair moves in a straight line.
8. We recommend that you have your wheelchair serviced every twelve months. For more information about servicing or spare parts, please consult your local dealer.

7. CONDITIONS OF GUARANTEE

This product is covered by a 36 months warranty against manufacturing defects.

Thank you for trusting Mobiclinic with this M12-M24 MARSELLA wheelchair! We appreciate your confidence in our products and are committed to always offering you the best quality.

If you have any questions or need additional assistance, please do not hesitate to contact us through our customer service at **clientes@mobiclinic.com**. We appreciate your comments and suggestions for further improvement.

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INTCO MEDICAL (HK) CO., LTD
GRUPO R. QUERALTÓ S.A.