# MANUAL DE INSTRUCCIONES

## Andador rollator ESCORIAL

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Please read these instructions carefully so that you are familiar with your new rollator before use. The ESCORIAL model can support a weight of up to 110 kg.

## 1. STEPS FOR THE USE OF THE ROLLATOR

1. Press the brakes to activate them.
2. Place both feet together in the middle of the rollator frame.
3. Disengage the brakes by lifting the brake levers.
4. Grip both handlebars of the rollator.
5. Move the rollator forward at a distance that is comfortable for you. Your feet may be the same distance apart as the rear wheels of the rollator.
6. Once you are comfortable, move forward with the weaker leg first.
7. Then lean on the handlebars to move forward with the opposite leg.
8. Remember that both feet must be between the rollator wheels. Continue walking as indicated in steps 5 and 7.

## 2. TIPS AND PRECAUTIONS

1. Always keep at least the tip of your feet inside the rollator frame.
2. When standing up from a seated position, lean on the seat instead of pulling on the handlebars of the rollator.
3. Apply the brakes before sitting down and make sure the brake is applied before standing up.
4. Take short steps to turn and keep your hips parallel to the front bar of the rollator.
5. Do not turn your body away from the rollator.
6. Do not sit in the rollator without first applying the brakes or use it as a temporary wheelchair, as it could tip over and cause injury.
7. Make sure all wheels are on the ground before walking.
8. Do not lean over the rollator.
9. Do not use on conventional or escalators.
10. Do not place heavy objects in the basket as this may cause the rollator to tip over.
11. Remember that this product should only be used on flat, smooth surfaces.

## 3. CLEANING AND MAINTENANCE

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth. Do not use solvents, bleach, synthetic detergents, wax polishes, aerosol sprays or other abrasives that may damage the product.

## 4. INDICATIONS

1. The walker can be used to help the elderly or people with mobility problems to walk, do the shopping.
2. The weight of the user must not exceed 110 kg, otherwise it will be deformed. The weight of the shopping basket must not exceed 10 kg.

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FOSHAN KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.