
# MANUAL DE INSTRUCCIONES

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Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

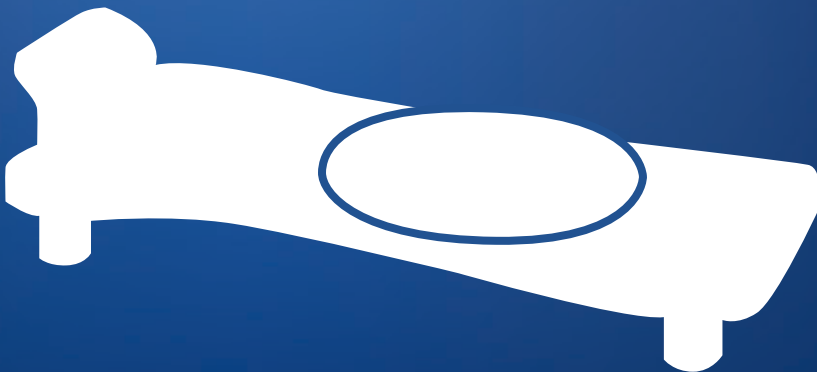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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Easily removable bathtub board mount design. Includes a round swivel seat panel, made of high strength materials, seat panel with holes to drain water from the surface and reduce the risk of slipping, grab handle for safety and a soap holder.

## 1. ASSEMBLY

1. Tighten the screws firmly into the threads. The handle must be facing the wall.
2. Place the bench on the bathtub.
3. Make sure that the inside walls of the bath are flat so that the bench can be securely fastened.
4. Adjust the position of the fixing screws to the inside width of the bath.
5. Tighten all screws securely.

## 2. INSTRUCTIONS FOR USE

- The bath board supports a maximum weight of 100kg. Do not overload the bench.
- The bath board pad helps to make it lightweight, easy to use and safe to access. Suitable for showering or bathing in the bath.
- Use the bench with care. Sit on it with the help of the handle and lift and insert your feet one at a time, never at the same time. Sit comfortably in the centre of the bench.
- There are openings in the bench to ensure water flow during use, reducing the possibility of slipping while sitting.
- For safety reasons, never sit on the edge of the bench that protrudes beyond the tub.

## 3. INDICATIONS

- The bath board is an auxiliary product suitable for people with musculoskeletal problems and people with disabilities.
- It facilitates daily activities related to personal hygiene for those for whom the conventional use of a shower or bathtub is difficult.

## 4. CONTRAINDICATIONS

- The use of the board is not recommended in case of perception or imbalance problems, inability to sit and/or weight over 100kg.

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**JIANLIAN HOMECARE PRODUCTS CO., LTD**
**LOTUS NL B.V.**
**GRUPO R. QUERALTÓ S.A.**