Andador / Rollator / Déambulateur OLIMPO

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ENGLISH

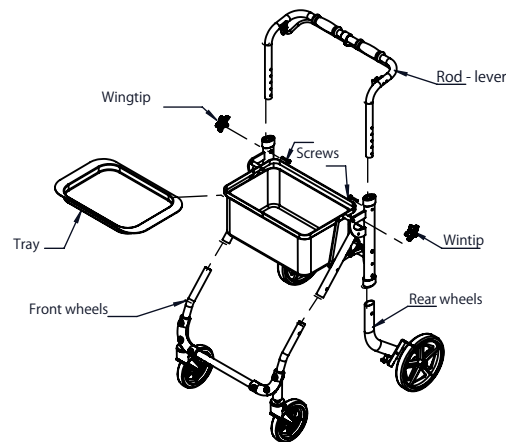
Notice

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Indications

The rollator can be used to help older people walk and shop.

Assembly

Use of the brake

The walker has brakes on the rear wheels that are operated by means of a lever rod under the hand rest. When the lever is actuated, the wheels are locked and released when the lever is released. When it is necessary to keep the walker braked for a longer period of time, pull the lever under the hand rest to the left.

Tips and precautions

Always keep a part of your body, such as your toes, inside the rollator frame. When standing up from a seated position, push yourself up from the seat rather than pulling on the handlebars of the rollator. Apply the brakes before sitting down and make sure the brake is applied before standing up to use the rollator again. Take short steps to turn and keep your hips parallel to the front bar of the rollator. Avoid turning your body away from the rollator. Do not sit on the rollator or use it as a temporary wheelchair if your rollator is not designed for this purpose, as it may tip over and cause injury. Remember that this product must be used on flat, smooth surfaces.

- It is designed to assist the elderly, disabled and people who have difficulty walking.
- Make sure all parts are securely tightened and all four wheels are in contact with the ground before use.
- To make it easier to climb curbs, this walker has two curb climbers on the rear wheels. In order to use them, it is necessary to hold down the brakes and then step on the curb climber with the foot.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V
GRUPO R. QUERALTÓ S.A