MANUAL DE INSTRUCCIONES

Taburete de baño FARO

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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Please read these instructions carefully before using this FARO bathing chair and keep them for reference.

To adjust the height of the chair, press the adjustment plunger on the leg and slide it to the desired height. Also ensure that the adjustment plunger is positioned securely into one of the holes. This process should be repeated for all four legs and they should all be at the same height for your safety.

1. INDICATIONS

- It is designed to assist patients, the elderly and/or the disabled.
- To be used indoors on a flat, smooth surface.
- Check regularly that all nuts and bolts are tight, and make sure that all four legs are level with the floor.
- Hold on to the seat on both sides to stand up.
- Maximum user weight: 100 kg.

2. TIPS AND PRECAUTIONS

- Unless specified, do not adjust, assemble or disassemble the shower chair with a seated user.
- Do not lean your body out of the chair to prevent it from tipping over.
- After use, wipe the shower stool with a dry cloth.
- Avoid direct contact with corrosive chemicals.
- Do not stand on the chair.

3. CLEANING

Remove dirt with a damp cloth and neutral soap and then wipe dry. Do not use harsh chemicals or abrasive cleaners to avoid damaging the product.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD
EC REP **SUNGO EUROPE B.V.**
GRUPO R. QUERALTÓ S.A.