MANUAL DE INSTRUCCIONES

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TROYA / TROYA PLUS / TROYA PLUS ULTRA

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

INDICATIONS

This product is intended for disabled or elderly people who have difficulty moving.

CONTRAINDICATIONS

Not recommended for people who are too weak to sit up. People who have difficulty in operating the wheelchair due to various disabilities.

1. CONTENTS OF THE BOX

1 Wheelchair

2 Accessories bag

Joystick

Charger

Tools

Anti-tipper

3 Model **Troya** (1 Battery, 17km autonomy)

Model **Troya PLUS** (2 Batteries, 34 km autonomy)

Model **Troya PLUS ULTRA** (3 Batteries, 51 km autonomy)

2. MAIN STRUCTURE

3. HOW TO FOLD OR UNFOLD THE CHAIR

Method 1. To open the wheelchair:

Method 2. To open the wheelchair:


Method 3. How to fold the chair:


4. ELECTRICAL DIAGRAM (depending on the model)

Controller LH-C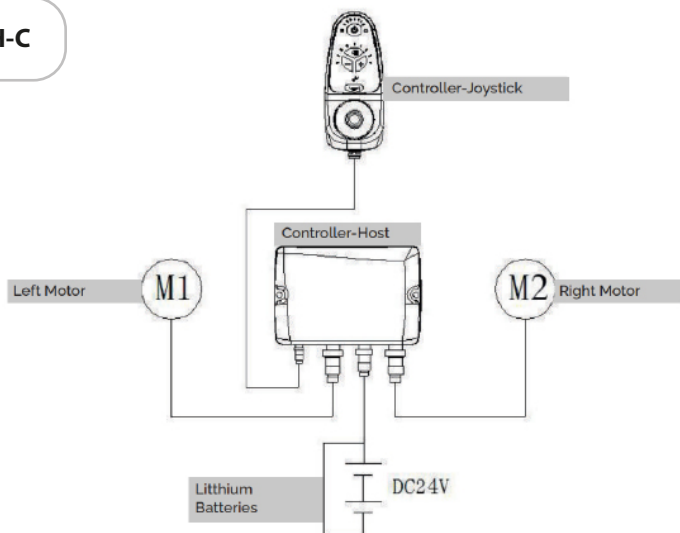

Controller SY-C20

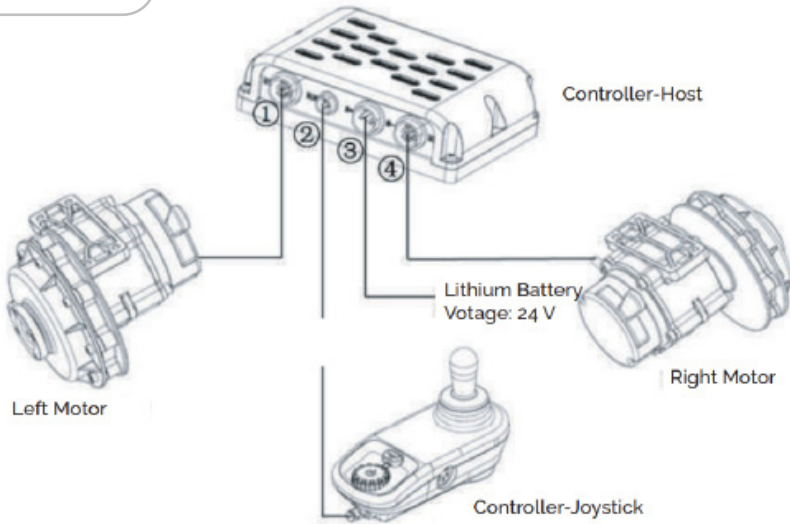
5. HOW TO INSTALL AND CONNECT THE JOYSTICK


6. HOW TO INSTALL AND REMOVE BATTERIES

Install the lithium batteries: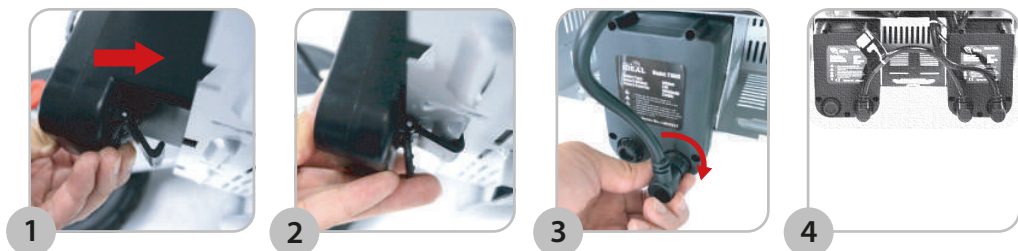

Remove the lithium batteries:

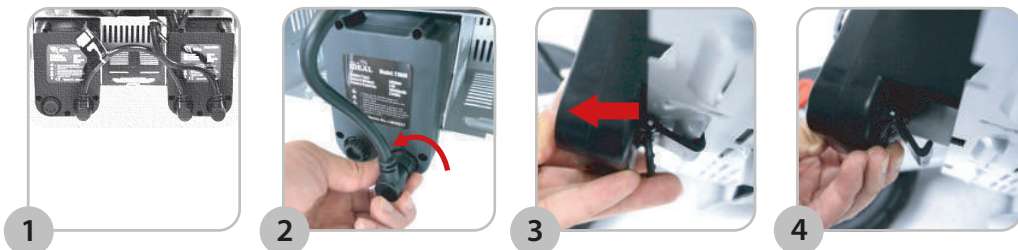7. HOW TO STEER AND CONTROL THE WHEELCHAIR


7.1. Changing the driving mode

The manual mode (M) allows the user to move the wheelchair without using the battery, i.e. by pushing the wheelchair. While the automatic mode (D) will allow you to both drive the wheelchair and keep it stationary when you want to park it.

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
Controller LH-C * Square wave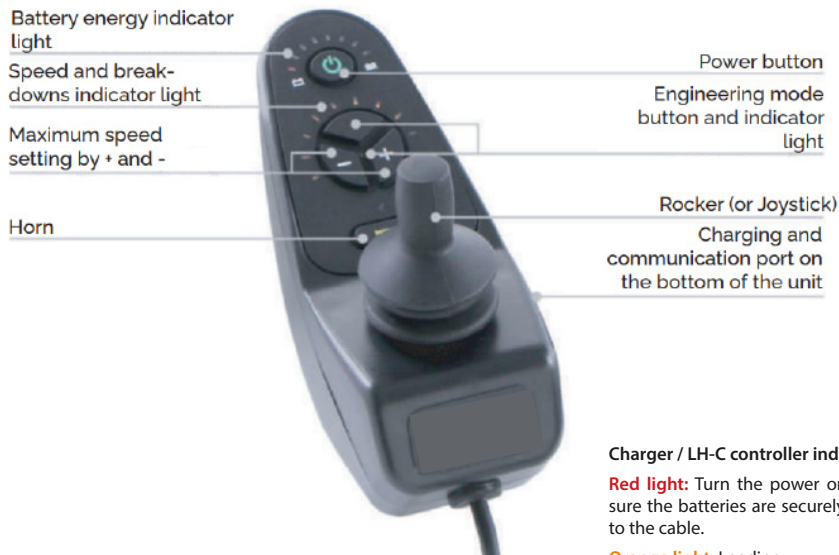


Charger / LH-C controller indicator light:
Red light: Turn the power on and make sure the batteries are securely connected to the cable.
Orange light: Loading.
Green light: Fully loaded.

Controller SY-C20 * LCD sine wave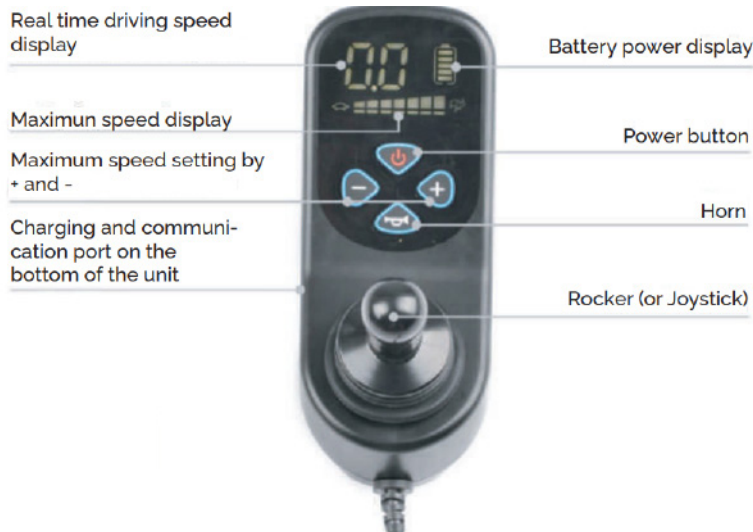

8. MORE FUNCTIONS


8.1. Folding armrests

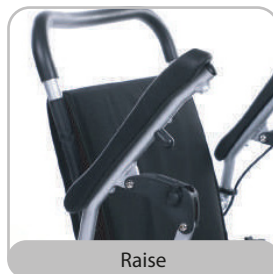8.2. Folding footrests

8.3. Height-adjustable backrest

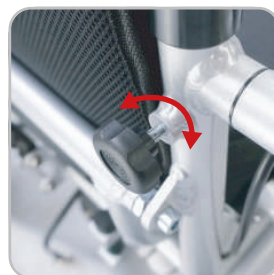9. CHARGING OF LITHIUM BATTERIES

Charging time is usually 2 to 8 hours.

Method 1. To load via the joystick:

1

Connect the charger

2

Connect the joystick

3

Connect to the mains socket


Method 2. For charging by cable (lithium batteries are charged directly) (Cable not included, sold separately):

1

2

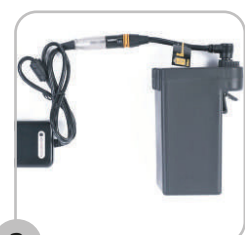3

CAUTION:

1. For indoor use only, do not expose to rain.
2. Disconnect power before removing the battery charger.
3. Place the charger in a well-ventilated area during charging.
4. Use charger only with 24V lithium batteries.
5. Charge the battery every 3 months if it is not used for a long time.

10. TECHNICAL CHARACTERISTICS

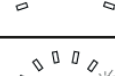
PARAMETERS	
Engine	LH-C: 250W*2, DC 24V / SY -C20: 220W*2, DC 24V
Maximum speed	8,5 km/h
Braking distance	≤ 1,0 m
Max. slope capacity	15°
Turning radius	85 cm
Rear wheel	30,48 cm / Maximum pressure 2,5 bar
Front wheel	20,32 cm
Net weight (without battery)	24,8 kg
Maximum user weight	120 kg
Lithium battery	24V10Ah/pc
Net weight of battery	1,7 kg/pc
Autonomy	LH-C: 17 km/1 battery - 34 km/ batteries - 51 km/ 3 batteries SY-C20: 20 km/1 battery-40 km/2 batteries-60 km/ 3 batteries
CD changer output	24V/24
AC charger input	100-240V, 50/60Hz

MEASUREMENTS	
Seat width	450 mm
Seat depth	440 mm
Seat height	500 mm
Wide backrest	440 mm
Backrest width	470 mm
Distance from seat to armrest	210 mm
Ground clearance	65 mm
Wheelbase	600 mm
Total length	1040 mm
Total width	640 mm
Total height	910-1060 mm (height adjustable)
Dimensions folded	640x340x740 mm (0,16m ³)
Carton box size	730x420x840 mm (0,26m ³)

11. SOLUTIONS FOR COMMON FAILURES (depending on the model)

LH-C CONTROLLER

When the speed indicator light flashes on the controller, it indicates that the power wheelchair is in an abnormal condition. The meaning of the flashing status and the recommended method of elimination are shown in the table below.

Speed indicator light Flashing status	Description of the fault and warning	Solution
First light flashing 	Fault in the behaviour of the left engine	1. Check if the left engine compartment is damaged. 2. Check whether or not the left engine cable is connected.
Second light flashing 	Fault in the right-hand engine compartment	1. Check if the right engine compartment is damaged. 2. Check whether or not the right engine cable is connected.
Third light flashing 	Low battery level	Charge the battery (1st step) and then check the battery level again.
Fourth light flashing 	Left engine failure	Check the left motor and its connections
Fifth light flashing 	Right engine failure	Check the right motor and its connections
Sixth light flashing 	Left brake failure	Check the left engine brake and its connections
Seventh light flashing 	Right brake failure	Check the right engine brake and its connections
Eighth light flashing 	Fallo en el controlador	90% probability, the malfunction is caused by the power tube of the control unit, it is suggested to replace the control unit or the main controller, or contact after-sales service.
Ninth light flashing 	Fallo en la conexión del joystick y controlador	1. Check the connection of the joystick cable to the controller. 2. The controller is damaged.

Tenth light flashing 	Motor overload	Release the control bar to resume.
First and tenth light flashing 	High battery voltage	Check the output voltage, it must not be higher than 35V.
All lights flashing 	The rocker is not in the centre of the joystick.	1. Do not touch the rocker arm when it is switched on. 2. Turn off, shake the rocker, and try to bring it back to the centre, if the problem persists, please contact after-sales service.

SY-C20 CONTROLLER

When the fault code appears on the controller display, it indicates that the power wheelchair is in an abnormal condition. The meaning of the display and the recommended method of elimination are shown in the following table.

Fault code	Fault description and warning	c Method of elimination
C1	Communication failure	Check if the power cable is intact and the connector is correctly connected.
C2	The joystick is not in the home position	Turn off the power, release the joystick back to the centre position, then restart.
C4 + 	Left brake or controller failure	Check whether the left motor crank is switched to the drive state, the power cable is intact and the connector is properly connected, then restart. If the problem is still not solved, connect the left and right interchangeable motors, if C4 appears and  means that the left brake is defective; if C4 and  are still displayed, it means that the controller is defective.
C4 + 	Right brake or controller failure	Check if the crank of the right motor is switched to the drive state, the power cable is intact and the connector is properly connected, then restart. If the problem is still not solved, switch on the left and right motors indistinctly, if C4 and  appears, it means that the right brake is defective; if C4 and  still appears, it means that the controller is defective.

C5	Overload warning	Switch off the power and switch the engine to the manual state, check if the engine can turn smoothly, if so, switch back to the driving state and restart.
C6	Ignition failure	Disconnect the power and restart.
C7	Overvoltage warning	Disconnect power, reconnect the battery cable and restart. Try replacing the controller or battery if the problem is still not resolved.
C8	Low voltage warning	The battery is low, please charge it in time. If the battery is fully charged and still low voltage alarm, it means that the controller is defective.
L1 / R1	Left or right motor overcurrent warning	Disconnect the power supply and put the motor into manual state, check if the motor can rotate smoothly, if not, disconnect the corresponding motor connector and the drive controller and check again, it means the motor is short-circuited if it still cannot rotate smoothly, on the contrary, it means the controller is faulty if it can rotate smoothly.
L2 / R2	Controller failure	Please contact the after-sales service.
L3 / R3	Circuit overheating warning	Disconnect the power and wait for it to cool naturally, then restart to check for a fault, if the fault still exists, contact after-sales service to replace the controller.
L6 / L7 / L8	Left engine or controller failure	Check if the power cable of the left motor is intact and the connector is properly connected. If the problem persists after the check, connect the left and right motors indistinctly and restart, it means the left motor is faulty if R6 / R7 / R8 is displayed, it means the controller is faulty if L6 / L7 / L8 is still displayed.
R6 / R7 / R8	Right engine or controller failure	Check if the power cable of the right motor is intact and the connector is properly connected. If the problem persists after the check, connect the left and right motors interchangeably and restart, it means the right motor is faulty if L6 / L7 / L8 is displayed, it means the controller is faulty if R6 / R7 / R8 is still displayed.

NOTICE

Switch off the wheelchair, leave it for a few minutes and restart it. If the fault persists, you should switch off the wheelchair and contact your dealer. Make a note of the information on the control panel display and report it to your dealer.

Do not use the wheelchair until the problem has been resolved or you have received further instructions from your service provider.

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SUZHOU IDEAL MEDICAL CO.,LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.