MANUAL DE INSTRUCCIONES

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Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

INDICATIONS

The product is recommended for people with mobility difficulties, who can operate the device with one hand.

The chair can be used indoors and outdoors. Consideration should be given to the environmental conditions and surroundings where it is to be used.

- Maximum weight supported (CENIT Model): 110 kg.

1. ASSEMBLY AND USE

1. To unfold the wheelchair, stand the wheelchair upright and press and hold both sides of the seat support until it opens fully (Figure 1). To fold, lift the seat cushion from the centre (Figure 2).

Figure 1

Figure 2

2. Attach the footrest (Figure 3). The height of the footrest can be adjusted (Figure 4). To remove it, simply pull it out.

Figure 3

Figure 4

3. Install the anti-tip wheel and fit the fixing bolt (Figure 5).

Figure 5

Figure 5.1

- To install the batteries, with the wheelchair open, hook the pins on both sides of the wheelchair onto the support rods (Figure 6).

Figure 6

Figure 6.1

- Install the controller. To do this, insert the bracket into its corresponding location on the armrest, adjust the location of the controller and tighten the adjusting bolt. Connect the wires according to the picture (Figure 7). The one marked LEFT connects the left motor and the one marked RIGHT connects the right motor.

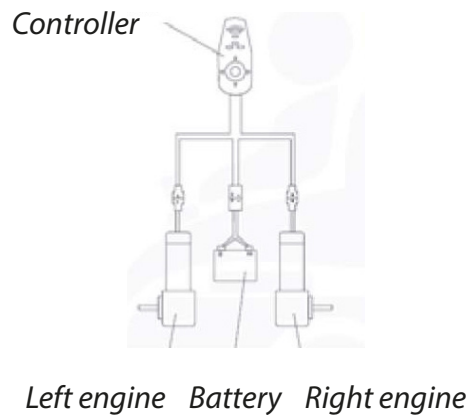
Figure 7

- Attach the protective strap by hooking it to the footrest bars (Figure 8).

Figure 8

- To start using the wheelchair, set the rear levers to Automatic Mode. If the levers are in Manual Mode, you will be able to move the wheelchair manually, but not use it as a power chair.

Automatic mode

Manual mode

Once you have completed the installation steps described above, the wheelchair is ready for operation. To optimise battery performance, it is recommended to first fully charge the batteries before using the wheelchair for the first time.

2. LOAD

The charging port of the powerchair is located on the bottom of the controller. The wheelchair uses two 12V batteries connected in series. To charge them, first connect the charger to the controller and then connect it to the power supply to start charging. When the battery is discharged, the indicator will be illuminated red; once the battery is full, it will turn green. If it is lit and flashing, the battery is charging.

1. Avoid using the wheelchair outdoors in low temperatures.
2. Stop using the wheelchair immediately if a low voltage alarm appears.
3. To avoid excessive discharge and damage to the battery in normal use.
4. If the battery is overcharged, the battery life will be reduced.
5. If the wheelchair is not to be used for an extended period of time, allow the wheelchair to fully discharge and disconnect the controller plug from the battery.
6. Both batteries should be replaced at the same time.
7. Do not disassemble the battery, as the acidic liquid it contains can be toxic to the human body.

3. CONTROL FUNCTIONS

4. TECHNICAL SPECIFICATIONS

Maximum speed	6 km/h
Approximate autonomy	20 km
Batteries	24V (2x12V)
Weight without batteries	42 kg
Charging time	6-8 h
Maximum inclination	8°
Operational environment	-5~40°C

5. CLEANING AND MAINTENANCE

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth.

Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage your product.

Store the product in a clean, dry place, away from rain, snow, ice, salt and water.

6. FAULT INDICATOR

If a fault should occur with your wheelchair when switching it on or during a journey, the protection system will automatically activate and send fault signals by means of a flashing light on the hand control. At this point, you should not use your wheelchair until the problem has been resolved.

Number of flashes	Failure	Solution
1	Bad contact for battery charging or too low batteries	Check the cables to verify that the battery is properly charged. Charge the battery.
2	Bad contact of the left motor circuit	Check the connection and wiring between the control unit and the left-hand motor.
3	Short-circuit in the left motor	The left motor is short-circuited, consult a technician
4	Bad contact of the right motor circuit	Check the connection and wiring between the control unit and the right-hand motor.
5	Short-circuit in the right-hand motor	The right-hand motor is short-circuited, consult a technician.
6	The controller has stopped working for security reasons	An external signal has interfered with the controller, check if the charger is connected. Do not use the controller while the battery is charging.

7	The steering lever of the control unit is faulty	Move the lever to the neutral position and restart the control unit
8	The controller has an internal fault	Consult a technician
9	The right and left brakes or their circuitry is faulty	There is a short circuit or damage to the electromagnetic brake circuits. Consult a technician.
10	Battery overcharge	Check the battery, its connections and wiring

7. PRECAUTIONS

1. The power wheelchair is suitable for indoor and outdoor use.
2. It should not be used on bumpy roads or climbing steps frequently.
3. It is not suitable to climb long slopes for long periods of time as this can cause the motor to overload and the battery to drain quickly.
4. If you need to stop the wheelchair, reset the control lever and it will stop automatically.
5. Do not disassemble the battery under any circumstances.
6. If you need to fold the wheelchair, the battery must be disconnected and removed.
7. Caution! Flammable material, avoid exposure to heat sources such as flames or sparks. Avoid transporting the batteries next to flammable or combustible objects.
8. In case you need to brake in an emergency situation, pull the steering lever in reverse and the wheelchair will stop instantly thanks to the electromagnetic brake system.
9. Keep tools and other metal objects away from the edges of the battery, as short circuits or electric shocks may occur in case of contact and cause injury.
10. Extreme caution must be exercised when reversing. The drive wheels could hit an obstacle and cause the wheelchair to lose control and tip over.

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FOSHAN KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD.
EC **REP** SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.