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EN Mobitape Kinesiology Tape

The skin-Friendly Mobitape Tape is very well and is elastic (40-60%) due to its wavy structure of the hypoallergenic acrylic hypoallergenic. An optimal fit of the structures on the human body, such as skin, fascia, muscles, meridians, etc. are secured and stimulated by our tape. It is very resistant, waterproof and tear-proof in its texture.

EXPLANATION OF THE SYMBOLS

Keep in a dry place. Do not use twice. Read an observe instructions for use. Lates free. Protect from sun.

APPLICATION

Measure the desired length and cut the tape with scissors. Please consider rounding the corner! Remove the tape from the paper base without touching the adhesive surface by gluing the base (approx. 5cm) and then glue the rest of the tape. After applying the tape, the body is placed in the pretension, this depends on the goal of the application. You can use Mobitape tape for sports injuries, joint pain, back pain, oedema, muscle tension and more indications and counter indications. On-line resources with more than 40 video instructions for different positioning techniques.

WARNING NOTICES

Never use the tapes on open wounds or as a bandage. Before every painting, it is important that a thorough finding of a specific therapy is carried out. We recommend that our tape should only be applied by trained therapists and doctors. The effect of kinesiology tapes is not scientifically proven, our statements are based on our many years of experience.

MATERIAL

97% cotton; 3% spandex

WARNING

Never use the tapes on open wounds or as a bandage. Before every painting, it is important that a thorough finding of a specific therapy is carried out. We recommend that our tape should only be applied by trained therapists and doctors. The effect of kinesiology tapes is not scientifically proven, our statements are based on our many years of experience.

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