# MANUAL DE INSTRUCCIONES

## Andador rollator HÉRCULES

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Please read these instructions carefully to get to know your new rollator before using it. The HÉRCULES model can support a weight of up to 100 kg. Steps for using the rollator:

1. Activate the rollator brakes by pressing the brake handles.
2. Place both feet together in the middle of the rollator frame.
3. Deactivate the brakes by lifting the brake handles.
4. Grasp both handlebars of the rollator.
5. Move the rollator forward to a distance that is comfortable for you. Your toes may even be at the same distance as the rear wheels of the rollator.
6. Move forward into the rollator frame with your weaker leg.
7. Step forward with your more agile leg while leaning on the handlebars. Both feet should be between the rollator wheels. Continue walking repeating steps 5 and 7.

Please insert the horizontal frame in the correct direction, you must insert both sides at the same time. Do not insert one side, then the other.

## 1. TIPS AND PRECAUTIONS

Always keep a part of your body, such as your toes, inside the rollator frame. When standing up from a seated position, push yourself up from the seat rather than pulling on the handlebars of the rollator. Take short steps to turn and keep your hips parallel to the front bar of the rollator. Avoid turning your body away from the rollator. Do not sit in the rollator or use it as a temporary wheelchair, as it could tip over and cause injury. Make sure all wheels are on the ground before walking. Do not slouch into the rollator. Do not use on escalators or stairs.

## 2. INDICATIONS

- Movement aid for people with disabilities.
- Intended for rehabilitation after various types of injuries.
- Aid to facilitate the movement of the elderly.
- Disorders of the musculoskeletal system.
- Active amplification of the motor regime after long-term illnesses, injuries and the postoperative period.

## 3. CONTRAINDICATIONS

The product is not intended for use by individuals with severe balance problems or impaired perceptual function.

A walker may be contraindicated for patients with limited upper extremity function, motor control,

cognitive function or balance deficits, but this determination should be made by the issuing physician who can weigh the risks and benefits for the individual patient.

#### 4. MAINTENANCE AND CLEANING

Maintenance and cleaning of a rollator is important to ensure its durability, safety and optimal functioning. Here are some tips:

##### MAINTENANCE:

1. **Regularly check parts and components:** Examine the rollator's wheels, brakes, frame and grips to make sure they are in good condition. If you find any damaged or worn parts, repair or replace them immediately.
2. **Adjust the wheels:** Make sure the wheels are tight and turn freely. Sometimes they may require tightening or loosening nuts and bolts.
3. **Keep the brakes in good condition:** Brakes are an essential part of a rollator. Check that they are working properly and that the brake pads are not worn. If necessary, replace the pads.
4. **Lubrication:** Apply lubricant to the pivot points and joints to keep the rollator smooth and easy to manoeuvre. Make sure there is no excess oil, as this can attract dirt and dust.
5. **General inspection:** Carry out regular inspections for damage or wear and tear. This will allow you to identify problems before they become safety hazards.

##### CLEANING:

1. **Clean regularly:** Clean the walker regularly to prevent the build-up of dirt, dust and germs. Use a soft, damp cloth to clean the surface of the frame and wheels.
2. **Wheels and tyres:** Make sure the wheels and tyres are clean. If they get dirty outdoors, remove dirt and debris from the wheels to prevent them from getting inside and damaging components.
3. **Grips and contact surfaces:** Clean grips and surfaces that come into contact with your hands regularly, as they can accumulate dirt and oil from the skin.
4. **Proper storage:** When not in use, store the rollator in a clean, dry place to prevent corrosion and premature wear.

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JIANLIAN HOMECARE PRODUCTS CO., LTD
LOTUS NL B.V.
GRUPO R. QUERALTÓ S.A.