# MANUAL DE INSTRUCCIONES

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Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Please read these instructions carefully so that you are familiar with your new rollator before use. The rollator is used by people who need support when walking due to lack of balance, disability or during a post-operative period. These models can support a weight of up to 100 kg.

## 1. INSTRUCTIONS FOR USE

1. To activate the brakes, press the handlebars upwards.
2. Place both feet together in the middle of the rollator frame.
3. Disengage the brakes by lifting the brake levers.
4. Grip both handlebars of the rollator.
5. Move the rollator forward to a distance that is comfortable for you.
6. Advance towards the inside of the rollator frame with your weaker leg.
7. Move forward with the more agile leg while leaning on the handlebars. Both feet should be between the rollator wheels. Continue walking repeating steps 5 and 7.

## 2. TIPS AND PRECAUTIONS

- Always keep a part of your body, such as your toes, inside the rollator frame.
- When standing up from a seated position, push yourself up from the seat instead of pulling on the handlebars of the rollator.
- Take short steps to turn and keep your hips parallel to the front bar of the rollator. Avoid turning your body to the opposite side of the rollator.
- The rollator must be braked when seated. Do not use it as a wheelchair. It can only be used as a temporary rest while being braked.
- Do not apply force on one side only or the rollator will tip over.
- Do not walk up or down stairs or steps with the walker.
- During use, do not adjust the rollator. Assembly and disassembly of the rollator should only be done when the rollator is not in use.
- Do not walk on escalators with a rollator.
- Do not change the direction of the rollator suddenly when using the rollator on a slope.
- Must be used on level roads. Do not use on bumpy roads. Avoid hitting the rollator against obstacles.
- Do not walk with a rollator uphill with a slope of more than 7 degrees. Do not walk with a rollator uphill or downhill, the lateral inclination or decline of the slope must not exceed 3.5 degrees.
- Clean the rollator regularly with a clean, dry cloth.

## 3. INDICATIONS

- It is designed to help the elderly, the disabled and people with walking problems to walk.
- Make sure that all parts are tight and that all four wheels are in contact with the ground before use.

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**GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD**
**SUNGO EUROPE B.V.**
**GRUPO R. QUERALTÓ S.A.**