# MANUAL DE INSTRUCCIONES

Andador rollator

AUGUSTO

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Please read these instructions carefully to get to know your new rollator before using it. The AUGUSTO model can support a weight of up to 136 kg.

## 1. STEPS FOR THE USE OF THE ROLLATOR

1. Activate the rollator brakes by pressing the brake handles.
2. Place both feet together in the middle of the rollator frame.
3. Deactivate the brakes by lifting the brake handlebars.
4. Grasp both handlebars of the rollator.
5. Move the rollator forward to a distance that is comfortable for you. Your toes may even be at the same distance as the rear wheels of the rollator.
6. Move towards the inside of the rollator frame with your weaker leg.
7. Step forward with your more agile leg while leaning on the handlebars. Both feet should be between the rollator wheels. Continue walking repeating steps 5 and 7.

## 2. TIPS AND PRECAUTIONS

1. Always keep a part of your body, such as your toes, inside the rollator frame.
2. When standing up from a seated position, push yourself up from the seat rather than pulling on the handlebars of the rollator.
3. Apply the brakes before sitting down and make sure the brake is applied before standing up to use the rollator again.
4. Take short steps to turn and keep your hips parallel to the front bar of the rollator.
5. Avoid turning your body to the opposite side of the rollator.
6. Make sure all wheels are on the ground before walking.
7. Do not slouch towards the rollator.
8. Do not use on conventional or escalators.
9. Do not place heavy objects in the basket, doing so may cause the rollator to tip over.

## 3. INDICATIONS

1. It is designed to help the elderly, the disabled and people who have trouble walking.
2. Make sure all parts are securely tightened and all four wheels are in contact with the ground before use.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.