MANUAL DE INSTRUCCIONES

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LAGO

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. ADVICE AND PRECAUTIONS

- Make sure that the four supports are placed stably on the side walls of the bathtub. The ends of the supports have a plastic coating to prevent the chair from slipping.
- Before using the chair for the first time, check its stability. Then help the patient to sit carefully in the chair.
- To rotate the seat, pull the side lever upwards.
- Once the patient is in the correct position, use the lever to secure the chair.

2. SECURITY

The maximum weight of the chair is 100kg.

3. ASSEMBLY

- Remove the two parts of the chair from the box and place the lower part on a flat surface. Insert the short metal tube, located at the base of the upper part (the chair with backrest), into the metal coupling on the lower part. Tighten the wing nut until the frame is secure. To check the firmness and stability of the chair, shake it.
- The lever on the underside of the seat surface is used to swivel and lock the chair.
- The smaller levers on both sides are used to adjust it securely.

4. CLEANING

Because the chair will be in contact with the skin, it is important to clean it regularly to avoid possible infections. Clean it with a solution of mild detergent and a conventional disinfectant. Then rinse the seat with a sponge and warm water. Do not use abrasive products for cleaning.

5. INDICATIONS

- It is designed for the elderly, patients, the frail and disabled, as a shower chair.
- To be used indoors on a flat, smooth surface.
- Check regularly that all nuts and bolts are tight. And make sure that all four legs are touching the floor at the same level.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.