# MANUAL DE INSTRUCCIONES / INSTRUCTION MANUAL / MANUALE D'UTILISATION

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## ENGLISH

### CAUTION

Read these instructions carefully before the patient uses this Guadiana model toilet seat booster and keep them for future reference.

The Guadiana toilet seat booster is designed for sitting on the toilet seat and should never be used for any other purpose than the intended one. Do not use this product as a staircase, do not stand up on it and do not allow children to play with it.

Check regularly before its use that the fixing elements are firmly secured. Check the state of the components of the product carefully. If any part appears to be in bad condition, it should be replaced with an original spare part.

Height 10cm.

### NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the product must be reported to the manufacturer and the authorities of the Member State in which the user and/or patient is established.

### INDICATIONS

- This product is indicated for the elderly or people with mobility difficulties that cannot use the bathroom unassisted.

- Maximum user weight: 100 kg

### ASSEMBLY AND INSTALLATION

1. Take all the components out of the packaging and verify that all the accessories are included.

2. Take one of the armrests and place it in the lateral orifice on the posterior part of the toilet seat booster, insert and tighten the screw between the armrest and the raised toilet seat. Repeat the same procedure with the other armrest.

3. Adjust the 3 sides of the toilet seat booster to your toilet seat, tighten the screws until it is stable.

### CARE INSTRUCTIONS AND MAINTENANCE

- Wash and dry the seat after each use. Remove the dirtiness with a wet cloth and neutral soap and then dry. Do not use aggressive or abrasive chemical cleaners as they can damage the seat.

- At the end of its lifetime, discard the seat responsibly.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V
GRUPO R. QUERALTÓ S.A