MANUAL DE INSTRUCCIONES

Elevador de inodoro RÍO

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. INDICATIONS

The RÍO toilet lift is designed to make sitting on the toilet easier and should not be used for other purposes. It is recommended for elderly people or people with mobility difficulties who are unable to use the toilet on their own.

Maximum supported weight: 160 kg.

2. CONTRAINDICATIONS

Do not use it as a ladder, do not stand on it and do not allow children to play with it. Supervise children when using the lift.

3. ASSEMBLY

1. Unscrew the side screws until they protrude from the riser.
2. Place the raised seat on the toilet. It should not be necessary to remove the existing toilet bowl.
3. Tighten the side screws until the attachment is secure. Do not overtighten. Once in place, the riser should not move.

4. CARE AND MAINTENANCE

Wash and dry the lift after each use. Remove dirt with a damp cloth and mild soap, then wipe dry. Do not use harsh chemicals or abrasive cleaners to avoid damaging the hoist.

At the end of its service life, dispose of the hoist properly.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.