
MANUAL DE INSTRUCCIONES

Colchón antiescaras MOBI 3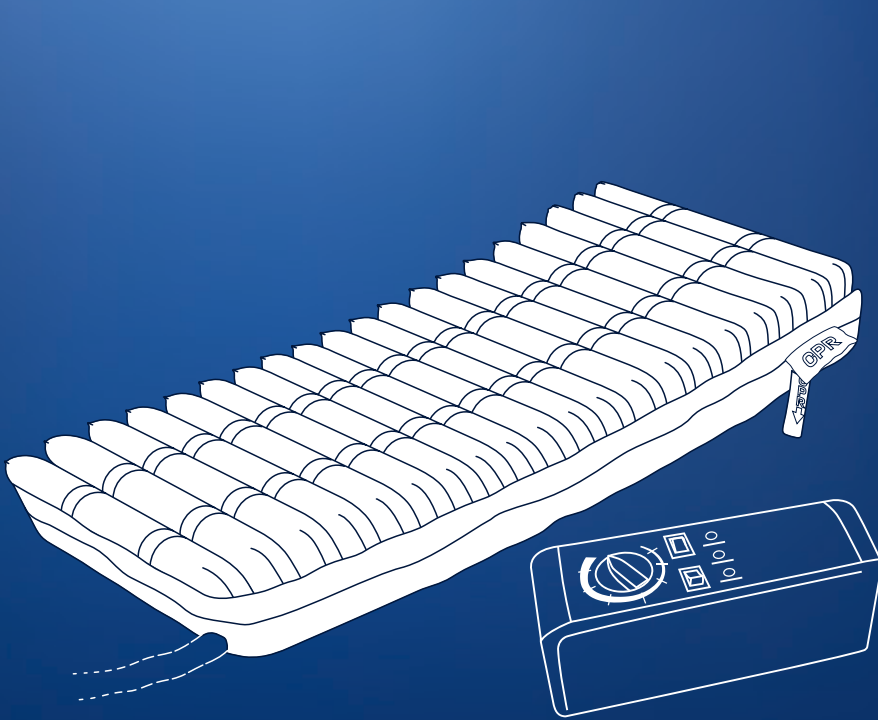

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

The device consists of two parts: the mattress and the motor. Thanks to its small compressor, the motor generates little noise and consumes little energy. The control panel is simple and very easy to use. The mattress relieves pressure through its alternating air cells that inflate and deflate approximately every 12 minutes. Constant pressure on a bony prominence is known to be the main cause of bedsores. As the mattress cyclically inflates and deflates, it forms a wave-like motion by alternating the patient's pressure points to allow the patient to remain in the most comfortable position.

1. USE

1. Place the mattress on the bed with the air sleeve towards the feet.
2. Use the integrated hooks to hang the compressor securely on the footboard of the bed. You can also place it on a flat surface.
3. Connect the air hoses of the mattress to the compressor.
4. Plug the compressor into an electrical outlet and make sure that the cord is not in danger of damage.
5. Turn on the switch on the compressor control panel to start inflating the mattress.
6. Once inflated, adjust the mattress using the compressor's dial.
7. Users can choose between "Alternating" and "Static" mode to meet the patient's requirements.
 - "Alternating" mode: The pump will inflate and deflate the cells periodically, which can promote blood circulation to prevent the occurrence of sores.
 - "Static" mode: The pressure between the cells is equalised. This mode is used for patient care and repositioning/transfer in and out of bed.

2. INDICATIONS

This product is intended for:

- To assist and reduce the incidence of pressure ulcers by optimising patient comfort.
- For long-term home care of patients suffering from pressure ulcers.
- For the treatment of pain according to medical prescription.
- If necessary, replace fuse with: T1A 250V. Input current: ~ 220V 50Hz, 0.1A.
- Supported weight: 180 kg.

3. CONTRAINDICATIONS

- This equipment is not suitable for use in the presence of a flammable anaesthetic mixture with air or with pure oxygen or nitrous oxide.

- Do not rotate or turn patients with a bone fracture or spinal injury.
- Do not smoke near the compressor.
- Do not bring the compressor near hot surfaces.
- There is a risk of explosion if used near flammable anaesthetics.

4. CLEANING AND MAINTENANCE

1. Ensure that the system is disconnected from the mains before cleaning.
2. Do not immerse in water.
3. Do not dry clean.
4. Do not use bleach or products containing bleach.
5. Do not iron.
6. Do not autoclave or use phenol for the cleaning procedure.
7. Do not wash air cells, clean with mild soapy water or disinfectant.
8. Wipe the compressor with a soft dry cloth.
9. Environmental conditions:
 - Operating conditions:
Ambient temperature: 10°C to 40°C
Relative humidity: 10% to 75%.
 - Storage and shipping conditions:
Ambient temperature: From -20°C to +70°C
Relative humidity: 10% to 100%.

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GUANGDONG YUEHUA MEDICAL INSTRUMENT FACTORY CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.