# MANUAL DE INSTRUCCIONES

## Andador EMÉRITA

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Please read these instructions carefully to get to know your new rollator before using it. The EMÉRITA model can support a weight of up to 100 kg. Steps for the use of the rollator:

1. Place both feet together in the middle of the rollator frame.
2. Grasp both handlebars of the rollator.
3. Move the rollator forward to a distance that is comfortable for you.
4. Advance towards the inside of the rollator frame with your weaker leg.
5. **Move forward with the more agile leg while leaning on the handlebars.**

## 1. ADVICE AND PRECAUTIONS

- Always keep a part of your body, such as your toes, inside the rollator frame.
- When standing up from a seated position, push yourself up from the seat instead of pulling on the handlebars of the rollator.
- Take short steps to turn and keep your hips parallel to the front bar of the rollator.
- Avoid turning your body to the opposite side of the rollator.
- Do not sit on the rollator or use it as a temporary wheelchair, as it could tip over and cause injury.
- Make sure all wheels are on the ground before walking.
- Do not bend towards the rollator.
- Do not use on conventional or escalators.

## 2. INDICATIONS

- Movement aids for people with disabilities.
- Intended for rehabilitation after various types of injuries.
- Aids in facilitating the movement of older people.
- Disorders of the musculoskeletal system.
- Active extension of the motor regime after long-term illnesses, injuries and the postoperative period.

## 3. CONTRAINDICATIONS

- The product is not intended for use by persons with severe balance problems or impaired perception.
- A walker may be contraindicated for patients with limited upper extremity function, motor control, cognitive function or balance deficits, but this determination must be made by the issuing physician who can weigh the risks and benefits for the individual patient.

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**JIANLIAN HOMECARE PRODUCTS CO., LTD**
**EC REP** **LOTUS NL B.V.**
**GRUPO R. QUERALTÓ S.A.**