# Manual de instrucciones

## Neb-1

- ES** - Compresor nebulizador
- EN** - Piston Compressor nebulizer
- IT** - Compresore nebulizzatore
- FR** - Compresseur nébuliseur
- PT** - Compressor nebulizador
- DE** - Inhaliergerät
- NL** - Vernevelcompressor
- SWE** - Kompressorn för nebulisatorn
- PL** - Kompresor nebulizatora
- DK** - Forstøverkompressor

-  ES- Lea las instrucciones antes de usar el producto.  
EN- Please read instructions before use.  
IT- Leggere le istruzioni prima di usare il prodotto.  
FR- Lisez les instructions avant d'utiliser le produit.  
PT- Leia as instruções antes de utilizar o produto.  
DE- Lesen Sie die Gebrauchsanweisung, bevor Sie das Produkt verwenden.  
NL- Lees de instructies voordat u het product gebruikt.  
SWE- Läs instruktionerna innan du använder produkten.  
PL- Läs instruktionerna innan du använder produktet.  
DK- Læs vejledningen, før du bruger produktet.

- Instrucciones en Español **ES**
- User manual in English **EN**
- Instructions en Français **FR**
- Istruzioni in italiano **IT**
- Anleitung auf Deutsch **DE**
- Instruções em Português **PT**
- Instructies in het Nederlands **NL**
- Instruktioner på Svenska **SW**
- Instrukcja w języku Polskim **PL**
- Vejledning på Dansk **DK**

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**INSTRUCTION MANUAL: Model NEB-1**

This compressor nebulizer is designed to deliver the prescribed medication solution to treat patient respiratory disorders, such as asthma, allergies and bronchitis. The nebulizer converts the medication solution into an aerosolized mist which is inhaled by the patient through the mouthpiece or masks. Read this manual thoroughly before using the nebulizer and save for future reference cleaning of the nebulizer is recommended after each aerosol treatment.

**NOTICE**

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

**INDICATIONS**

The compressor nebulizer intended to generate aerosols that are delivered directly to the patient to inspire and Treatment of respiratory diseases

**CONTRAINDICATIONS**

- Do not drop water or any other liquid on the nebulizer.
- Keep the device away from hot, burning or sparkling objects.
- Never block the air openings or place it on a soft surface such as a bed or a sofa.
- Do not use the device near other radio frequency devices, since they may interfere with the proper nebulizer operation.

**PRECAUTIONS AND WARNINGS**

- Keep the device away from your eyes during use. The vapour of medicines may be harmful.
- Do not use the device near flammable gases, oxygen or nitrogen 3. oxide.  
Before cleaning and maintenance work, the device must be switched 4. off and unplugged.
- Make sure that no liquids get inside the equipment.  
The use of nebulisers and accessories from other manufacturers may 6. reduce the therapy effectiveness and damage the device.
- Keep the device and its cable away from heat sources.
- Never cover the device while working.  
If different medicines are inhaled in succession during therapy, make 8. sure you wash the nebulizer with hot water after each use.

**OPERATING INSTRUCTIONS**

- Place the compressor on a stable surface, so you can easily use the device while sitting.
- Unwind the cable from the base and make sure that the device is in the off position.
- Connect the power cable
- Connect one of the ends of the hose to the compressor.
- Pour the medicine through the cover opening using a dropper or previously measured dose container. The marking "6cc" marked on the nebuliser cup is the maximum level.
- Assemble the T-shaped mouthpiece and the hose head and the hose into the top of the nebuliser. If a mask is used, connect the bottom of the mask to the top of the nebuliser cover.
- Connect the hose to the nebuliser.
- Press the on bottom to turn the device on.
- Put the mouthpiece between your teeth. Inhale and exhale through the mouthpiece.
- If a mask is used, place the mask over the mouth and nose.

**CLEANING AND MAINTENANCE**

**Cleaning:**

- Switch off the device and unplug it.
- Remove the hose from the air intake.
- TO CLEAN THE DEVICE: Rinse these parts thoroughly to remove all the detergent. Let them air dry.
- TO DESINFECT: Mix one part white vinegar and 3 parts hot water in a clean container. Put the mouthpiece, the cup and the masks for half an hour in the solution. Remove from solution and let air dry.
- It is no necessary to clean the hose. If necessary, clean the surface.

**Maintenance:**

- The filter of the compressor should be replaced when it turns into grey.
- Only authorised personnel must repair this product.  
CAUTION: The nebuliser may be damaged if used with a dirty filter or if the filter is replaced by other material such as cotton. Do not use it without a filter.

**SPECIFICATIONS**

Electrical requirements	230 V / 50Hz
Energy consumption	Under 60 W
Current	0.70 A
Dimensions (long x wide x high)	155 mm x 140 mm x 95 mm
Weight	1.2 kg
Sound pressure level	55 dB (A)
Maximum pressure of the compressor	From 35 Psi to 50 Psi (241 kpa to 345 kpa)
Operating pressure	9-16 Psi (62 kpa to 110 kpa)
litre flow range	5-8 L/Min
Nebulisation rate	≥ 0.25 ml/Min
Particle size	3 micron
Maximum capacity of the nebulised solution	6 ml
Maximum volume of residual medicine	0.5 ml
Temperature in operation	From +5 °C to +40 °C
Humidity in operation	From 0% RH to 85% RH
Storage and transport temperature range	From -25 °C to +70 °C
Transport/storage humidity	From 10% RH to 95% RH
Operational transport/storage atmospheric pressure	From 700 mb to 1,060 mb

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**SHENZHEN BI-RICH MEDICAL DEVICES CO.,LTD**
**GRUPO R. QUERALTÓ S.A.**