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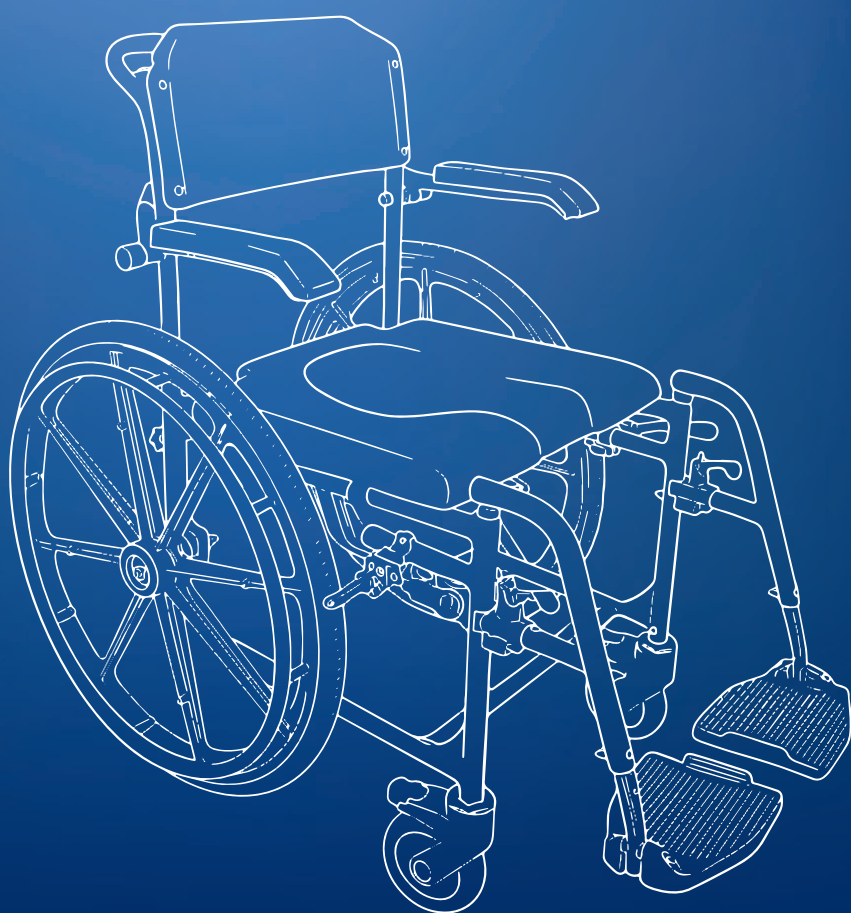
**IMPORTANT**

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Thank you for choosing the Caligula bathing wheelchair! This instruction manual is designed to guide you in the safe and effective use of your new wheelchair, ensuring your comfort and safety in the bathroom.

Our Caligula wheelchair is designed with high quality and durable materials, offering stability and ease of movement in wet environments. This product is specially designed to facilitate personal hygiene for people with reduced mobility, allowing safe and comfortable access to the shower or bath.

Please read this manual carefully before using your wheelchair for the first time. Correct understanding and proper use of the functions and features of your wheelchair are essential to avoid accidents and maximise its durability and functionality.

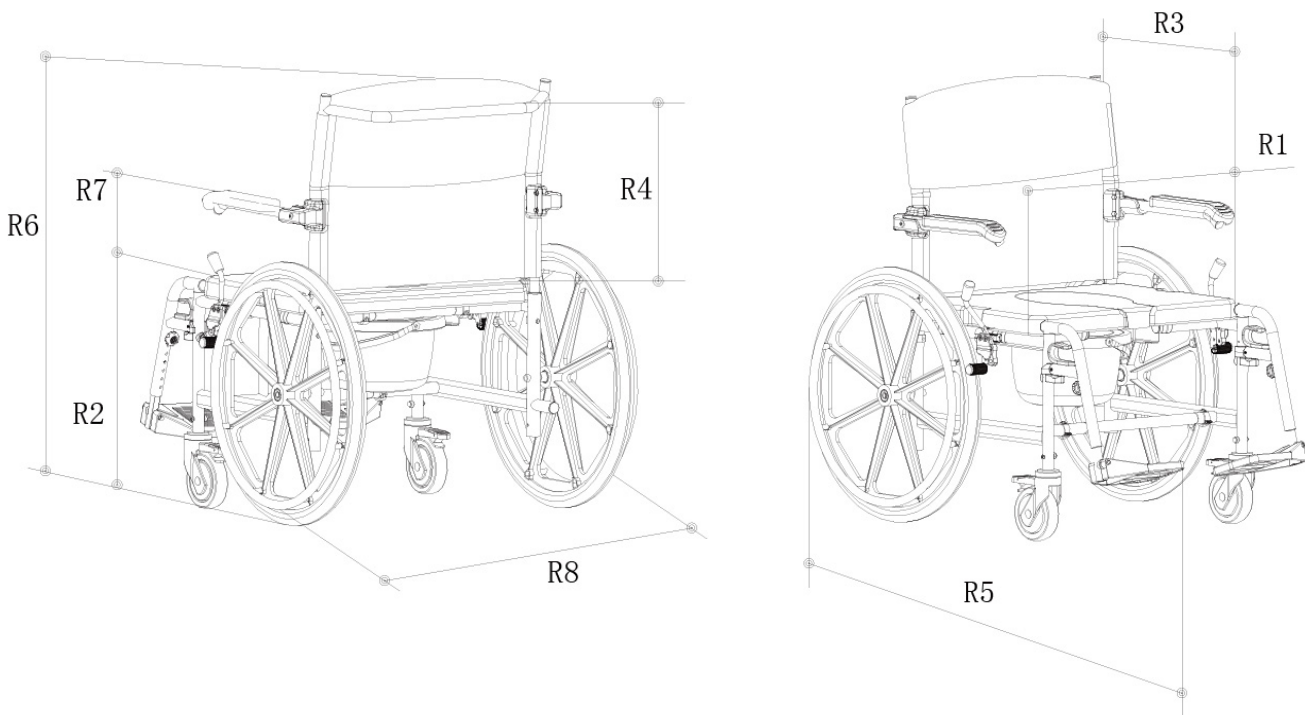
**Declaration of conformity with Regulation (EU) 2017/745:**

This product has the corresponding Declaration of Conformity in compliance with Regulation (EU) 2017/745.

**1. PRODUCT DESCRIPTION:****General description of the characteristics and technical specifications:**

- Model: Caligula
- Reference: A-00359/29
- Description: Aluminium bath wheelchairs
- Maximum supported load: 120 kg
- Materials: aluminium structure, seat, backrest and armrests in PU.

- Measurements:
  - Seat width (R1): 460 mm
  - Seat height (from the floor to the seat) (R2) : 510 mm
  - Seat depth: (R3): 400 mm
  - Backrest height (R4): 450-480 mm
  - Saddle length (R5): 1030 mm
  - Chair height (R6): 965/990 mm
  - Height of armrest (measured from the seat) (R7): 220-245 mm
  - Saddle width (R8): 630 mm

### Purposes of the product:

- **Facilitating personal hygiene:** The bathing wheelchair enables people with reduced mobility to access the bath or shower safely and comfortably, making it easier to perform personal hygiene tasks without assistance or with minimal assistance.
- **Promoting independence:** Helps people maintain their independence by enabling them to carry out bathing activities without the need for constant help from carers or family members. This contributes to greater autonomy and self-esteem.
- **Ensuring safety:** Designed specifically for wet environments, the bathing wheelchair provides stability and reduces the risk of falling or slipping. Its anti-slip features and water-resistant structure ensure safe use in the bathroom.
- **Ease of movement in the bathroom:** The wheels and locking mechanisms facilitate movement within the bathroom and allow the chair to be held in place when needed. This is especially useful for safe and stable transfers to the toilet or shower.

## 2. INDICATIONS FOR USE

This wheelchair is suitable for people with:

- Restricted or limited mobility (e.g. legs and/or hips)
- Balance problems in standing or walking due to permanent transient disabilities.
- The weight of which does not exceed 150 kg.

## 3. CONTRAINDICATIONS

This wheelchair is not suitable for the following cases:

- Users with open wounds, ulcers or infections on areas of the body that will come into contact with the chair, which could worsen the condition.
- Patients whose weight exceeds the maximum capacity of the chair, which could compromise the structure and safety of the device.
- People who cannot sit upright on their own and cannot be safely secured in the chair, which increases the risk of falls.
- Users with severe respiratory problems that may worsen in a humid or enclosed environment such as a bathroom.
- Situations where the wheelchair could be used incorrectly, such as use on unsuitable or dangerous surfaces outside the bathing environment for which it was designed.

## 4. SAFETY WARNING

- Never leave a person unattended while using the wheelchair in the shower or bath, especially if the user has limited mobility or balance problems.
- Use the wheelchair only for its intended purpose. Do not use it for carrying heavy objects or as a support for standing or sitting.
- Always apply the brakes when the wheelchair is in stationary use to avoid unexpected movements. Regularly check the brakes for proper function.
- Avoid sudden movements or extreme tilting while in the wheelchair to prevent falls or tipping.
- Use only original spare parts and accessories to ensure compatibility and safety.
- Strictly follow the assembly instructions, steps and methods of assembly.
- Before using this product, please check that all parts are correctly assembled.
- If you encounter mechanical errors or abnormal operation of the product, please contact the technical team.

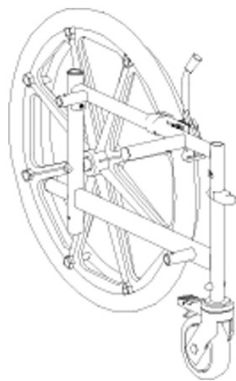
- The user's weight must not exceed the maximum weight capacity.
- This product is suitable for indoor use, with potty chair and wheelchair functions.
- Avoid exposure to sunlight and store in a cool, dry and ventilated place.
- Before pushing the product, ensure that the user's feet are placed on the footplate.


#### 4. PARTS AND COMPONENTS

This product allows installation without tools. Please review the following list of product parts before assembly.

(1) Left wheel: 1pc

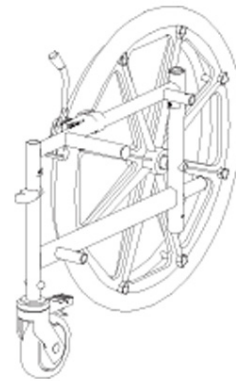(2) Right wheel: 1pc

(3) Backing: 1pc

(4) Connector tubes: 2pc

**(5) Left footrest: 1pc**

**(6) Right footrest: 1pc**

**(7) Seat: 1pc**

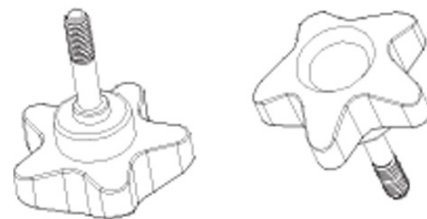
**(8) Seat cover: 1pc**

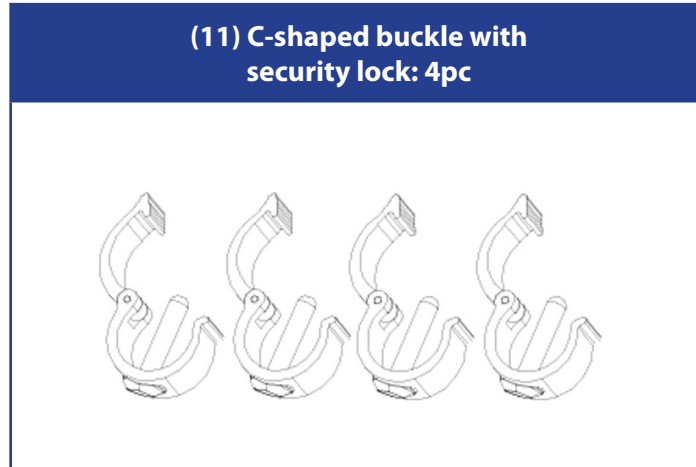
**(9) Bucket: 1pc**

**(10) Star-shaped height adjusting screws: 2pc**


## 5. ASSEMBLY INSTRUCTIONS

Steps to follow for assembly:

|  |  |
|--|--|
| <p>Prepare parts (2) x1, (11) x2 and (4) x1.</p> <ul style="list-style-type: none"> <li>• Insert part (4) to part (2) as shown in figure 1, while aligning the side holes of these two parts.</li> <li>• Insert part 11 into the hole and fasten the safety catch, as shown in figure 1. (Pay attention to detail E, which indicates to press the buckle closed to prevent it from falling out)</li> </ul> |  |
| <p>Prepare parts (1) x1 and (11) x2</p> <ul style="list-style-type: none"> <li>• Insert part (1) to part (4) as shown in figure 2, while aligning the side holes of these two parts.</li> <li>• Insert part 11 into the hole and fasten the safety catch</li> <li>• (Note: part (11) has a safety lock to prevent it from coming off during use.)</li> </ul>   |  |


Prepare parts (3) x1 and (10) x2

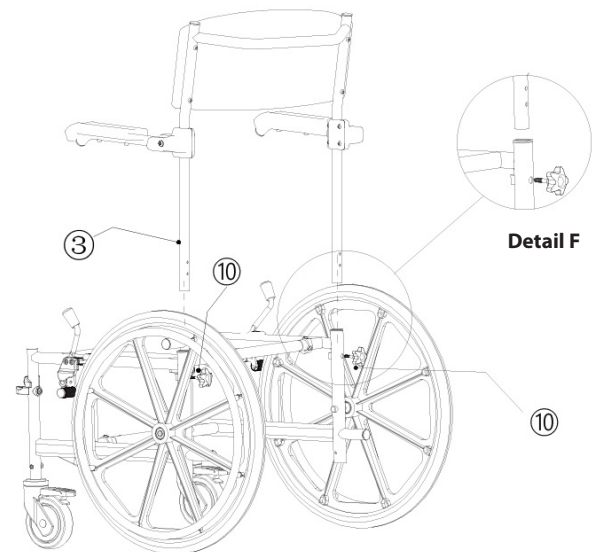- Insert part (3) to the nipples as shown in figure 3, while aligning the side holes of these two parts.
- (Note: there are two holes available at the end of part 3) which allows the height of the backrest and armrests to be adjusted at the same time. Please choose the correct position depending on the user's situation.
- After aligning the holes, fasten the two parts with part (10) as shown in figure 3. (Pay attention to detail F, which indicates that the adjustable positions on the backrest tube are 1 inch apart.)


Prepare part (7)x1 and part (8) x1.

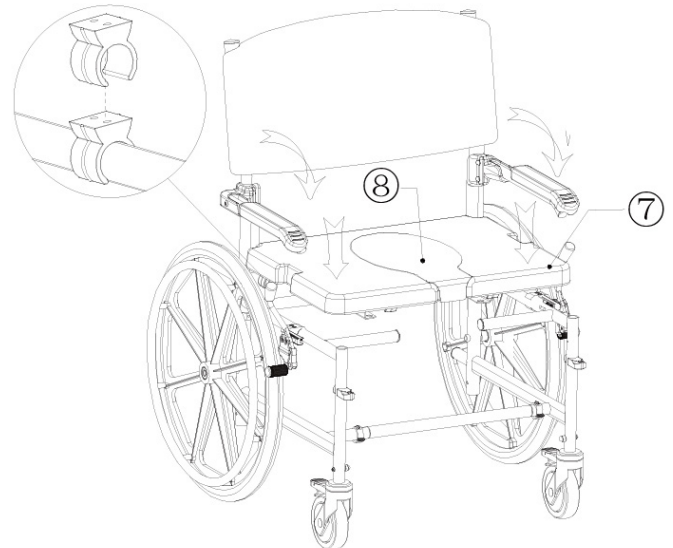- Align the U-shaped buckles at the bottom of the part (7) with the frame. Press both ends of the seat to secure the buckles in place.
- Place the part (8) in the central groove of the part (7)
- Push the armrests forward until they click into place as shown in figure 4.
- (Note: The armrest can be easily raised, making it convenient for users to transfer and push it onto desks)


Prepare part(5) x1 and part (6) x1

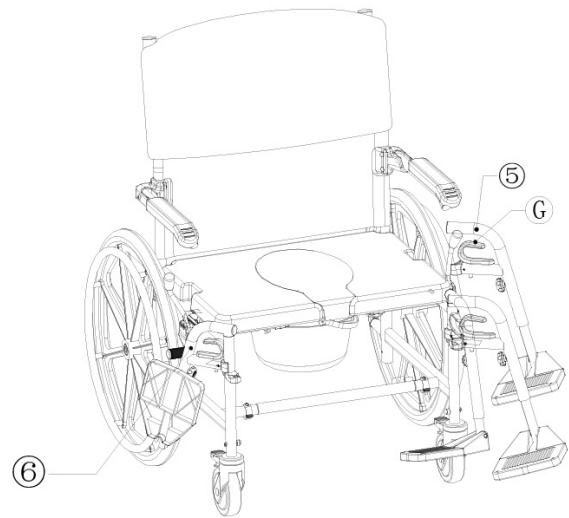- Align the part (5) with the position of the nozzle as shown in figure 5. Insert from one side and turn it to the front in the direction of the arrow, until it is fixed in the holder.
- (Note: The footrest can be foldable. While holding the handle G (attention to picture 5), you can turn the footrest to one side. As shown in part (6), lift it up to remove the footrest.


Prepare part (8) x1.

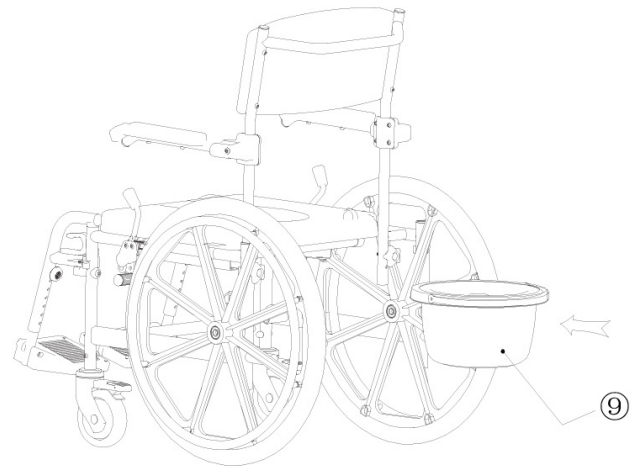- Align the part (8) with the slot and the button and press it in the correct position, as shown in figure 6


Description of functions:

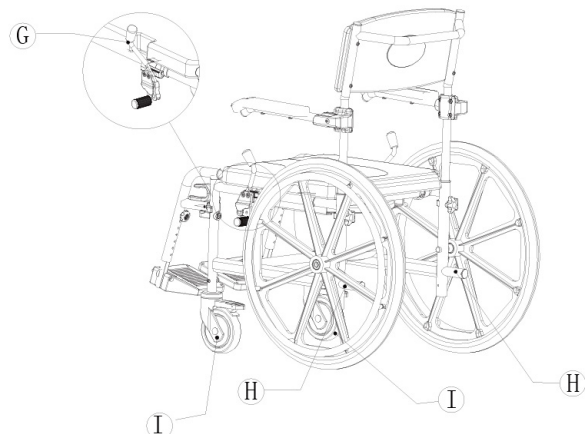
- It is equipped with 4 sets of universal silent castors, with horseshoe foot brakes, as shown in figure 7, which can be easily pushed and turned in a narrow space.

- There are pedals on both rear sides of the frame, which is convenient for the caregiver to lift the commode chair by foot and cross obstacles easily.
- The armrest can be easily raised, which is convenient for users to transfer and push towards the desks, as shown in picture 4.
- Swivelling and removable footrests, as shown in figure 5, which are also height adjustable. Note: As shown in picture 8, part (L) has two positions to be adjusted and part (K) has 7 positions to be adjusted. Select the suitable position according to the user, and then lock with part (J)

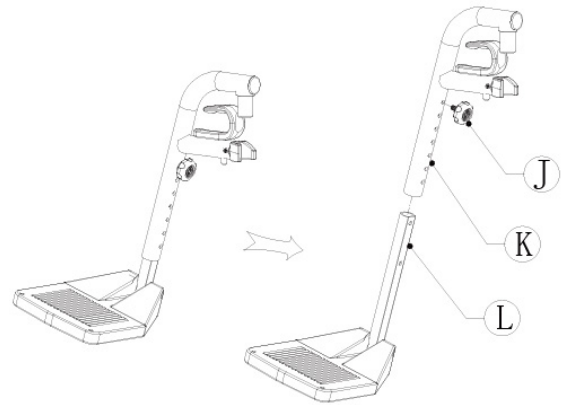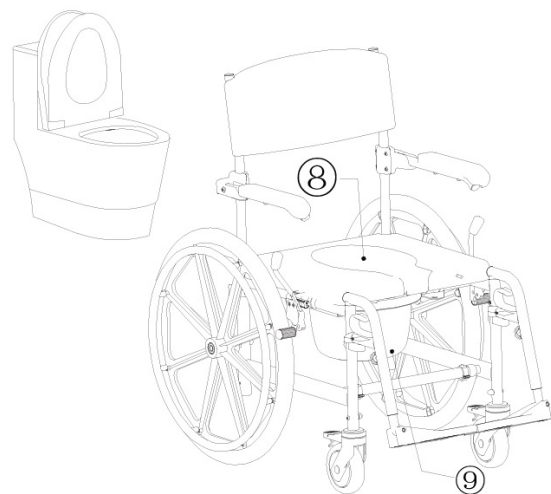- Aluminium brake set; push handle brake forward (as shown in figure 5)
- After removing part (8) and part (9), the product can be used in a medium sized toilet, as shown in figure 9.

## 6. INSTRUCTIONS FOR USE

### Brakes

- The rear wheels of the chair measure 24 inches and its soft push rims allow the user an easy and light movement, feeling comfortable and easy to handle.
- When the handbrake lever is pushed forward, the rear wheels come to a full stop.
- To release the brake and allow passage, the handbrake lever must be pulled back to allow the rear wheels to move again.

### Footrest

- The footrests can be swivelled to the side. This makes it easier for the patient to leave the bathing chair from the front.
- This can be achieved by turning the footrest manually to one side by hand.

### Armrests

- Both armrests can be raised. This makes it easier for the patient to leave the bathing chair from the side.

## 7. MAINTENANCE AND CLEANING

### Regular maintenance

- Visually check the chair for any damage, wear and tear or loose parts before each use.
- Make sure that the wheels move freely and the brakes work properly.
- Check all moving parts, such as armrests, footrests and adjustment mechanisms, to ensure that they operate smoothly.

### Cleaning

- Clean the wheelchair after each use in the bathroom to remove soap residue.
- Use a damp cloth with lukewarm water
- Avoid the use of aggressive chemicals that may damage the materials.
- On a monthly basis, carry out a more thorough cleaning of all parts, including wheels and brakes.

## 8. STORAGE AND TRANSPORT

- Store the wheelchair in a dry, clean place when not in use to avoid prolonged exposure to moisture, which can cause corrosion.
- Avoid storing the chair in locations exposed to extreme temperatures or direct sunlight.

## 7. TROUBLESHOOTING

### Assembly problems:

- **Missing Parts**
  - Problem: When opening the package, some parts required for assembly are missing.
  - Solution: Check the parts list in the manual and contact the manufacturer or supplier to order the missing parts.
- **Difficulty in Assembling Components:**
  - Problem: Parts do not fit correctly or do not fit according to instructions.
  - Solution: Make sure you are using the correct parts according to the instructions. If the problem persists, check the illustrations in the manual and make sure all parts are oriented correctly. If you still cannot assemble the chair, contact customer service.

### Problems of use:

- **Wheels not turning properly**
  - Problem: Wheels are blocked or do not turn smoothly.
  - Solution: Inspect wheels for obstructions such as hair, dirt or debris. Clean the wheels and lubricate the axles as necessary. Make sure the brakes are not applied while attempting to move the wheelchair.
- **Brakes not working:**
  - Problem: Brakes do not engage or hold the wheelchair in place.
  - Solution: Check the brake adjustment and ensure that the brakes are correctly fitted. Clean off any dirt that may be preventing the brakes from working. If the brakes are worn, consider replacing them.

- **Footrests and armrests loose:**

- Problem: Footrests and armrests are loose or not held in position.
- Solution: Ensure that the footrests and armrests are correctly installed and tightened. Tighten any loose screws or bolts. If the part still does not work properly, consult the manufacturer for replacement parts.

- **Damaged seat or backrest:**

- Problem: The seat or backrest shows signs of wear or damage.
- Solution: Regularly inspect the seat and backrest for damage. If significant damage is found, discontinue use of the chair until the seat or backrest has been repaired or replaced. Contact the manufacturer for replacement parts.

## 8. WARRANTY AND CUSTOMER SERVICE

This product is covered by a 36-month warranty against manufacturing defects.

Thank you for choosing Mobiclinic! We appreciate your confidence in our products. We are committed to providing you with the best quality and service. We know you will enjoy your new Caligula as much as we enjoyed creating it for you. If you have any questions or need assistance, please feel free to contact us at [clientes@mobiclinic.com](mailto:clientes@mobiclinic.com). We are here to help you.

**ES** - Peso máximo soportado 120 KG.  
**EN** - Maximum supported weight 264,555 lb.  
**IT** - Peso massimo supportato 120 KG.  
**FR** - Poids maximal supporté 120 KG.  
**PT** - Peso máximo suportado 120 KG.  
**DE** - Maximal unterstütztes Gewicht 120 KG.  
**NL** - Maximaal ondersteund gewicht 120 KG.  
**SW** - Maximal stödvikt 120 KG.  
**PL** - Maksymalna obsługiwana waga 120 KG.  
**DA** - Maksimal understøttet vægt 120 KG.

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XIAMEN NEAQ IMP.& EXP. CO., LTD,
SUNGO EUROPE B.V
GRUPO R. QUERALTÓ S.A.